MULTI-ADJUSTABLE AMPUTEE SUPPORT

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner’s manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION..... 3

COMPOSITION..... 3

RECOMMENDATIONS..... 3

WARNINGS 4

INSTALLATION 4

ADJUSTMENTS 5

MAINTENANCE..... 7

WARRANTY..... 8

FRANÇAIS 9

DESCRIPTION

The multi-adjustable amputee support properly aligns the tibial segment. The cushion's composition provides optimal comfort for the amputated limb.

The cushion is available in two types: flat or profiled

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Mounting hardware

Aluminium

Cushion

ABS

Plastazote

Soft memory foam

Viscoelastic foam

RECOMMENDATIONS

To ensure the longevity of your lower limb support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the lower limb support's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required lower limb support and make any necessary adjustments.
- Clean regularly and inspect the lower limb support's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the lower limb support's surface.

It is recommended to consult your doctor or therapist in the month following the installation of a lower limb support to make sure it is efficient.

ENGLISH

WARNINGS


- Physipro Inc. lower limb supports are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your lower limb support.
- Never push or lift wheelchair with a lower limb support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- New lower limb supports should be tested before prolonged use. To avoid redness due to pressure points, it is necessary to regularly check for redness in the various critical pressure areas, especially around bony prominences.
- Before use, let the lower limb support get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your lower limb support to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION

Required tools: 5 mm and 6 mm Allen keys

Installing the amputee support on the legrest

1. Loosen screws **A** of the mounting clamp and position clamp at the desired height on the legrest tube.
2. Firmly tighten screws **A**.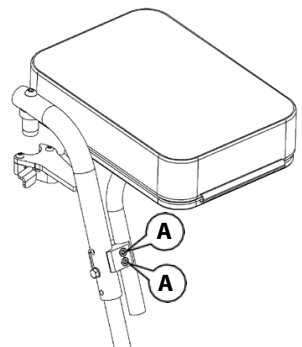

ADJUSTMENTS


With its multiple adjustments, the multi-adjustable amputee support enables optimal alignment of the tibial segment.

Required tools: 5 mm and 6 mm Allen keys

Height adjustment

9-inch (23 cm) adjustment range


1. Loosen screws **A** of the mounting clamp.
2. Lower or raise the amputee support tube until the cushion is positioned at the desired height.
3. Firmly tighten screws **A**.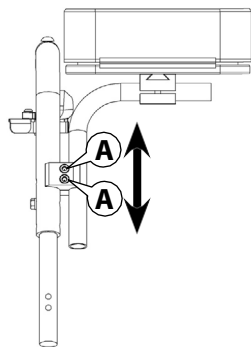

Width adjustment

5-inch (13 cm) adjustment range

1. Loosen screw **B** to enable anchor block **C** to move freely.
2. Adjust the position of the cushion and firmly tighten screw **B**.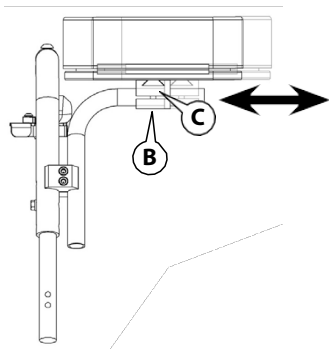


ENGLISH

Depth adjustment



1-inch (2,5 cm) adjustment range

1. Loosen screw **B** to enable anchor block **C** to move freely.
2. Adjust the position of the cushion until it is positioned at the desired depth.
3. Firmly tighten screw **B**.

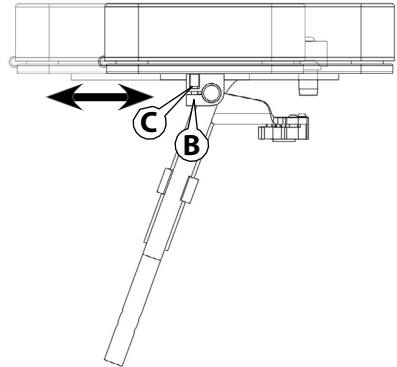**Angle adjustment**

0° à 90° adjustment range

1. Loosen screw **B** to enable anchor block **C** to move freely.
2. Adjust the position of the cushion until it is positioned at the desired angle.
3. Firmly tighten screw **B**.

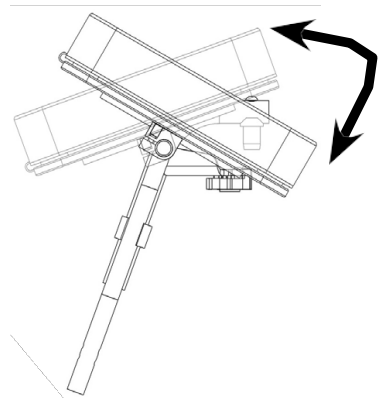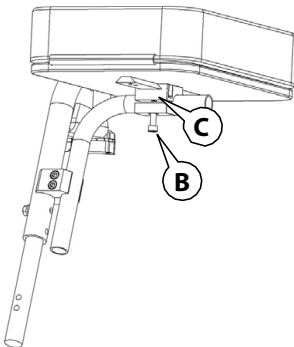MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your lower limb support. It is recommended to visually inspect all the components of the lower limb support during the cleaning process. A weekly cleaning is recommended.

Cleaning the cushion cover

1. Unfasten zippers and remove the cover.
2. Hand wash with a mild detergent and rinse.
3. Let air dry completely, then reinstall.

WARNING

- Never machine wash or dry.

Cleaning the cushion padding

1. Wipe padding lightly with a damp cloth, without soap or detergent.
2. Use a clean dry cloth to gently press on padding to absorb excess moisture, then let air dry completely.

WARNING

- Do not submerge padding in water.

Cleaning the hardware

The hardware can be cleaned with warm water and a mild liquid cleaner.

Storage

Lower limb supports must be stored in a dry and secure area.

WARNINGS

- Never store your lower limb support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your lower limb support near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your lower limb support is two (2) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH**WARRANTY**

The warranty period for the parts and components of the Multi-adjustable Amputee Support is **12 months**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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