Padded Y-style 4-point belt adult and pediatric

Owners manual

This manual contains important information about this product.
Please pass it to the final user upon delivery.

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ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our company by choosing one of our products. This user manual has been designed to allow the client to use **padded Y style 4 points belt**, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your **padded Y style 4 points belt** and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the company.

For Physipro Inc., your satisfaction remains a priority.

Safety Rules

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user.

Physipro recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a pelvic support belt is appropriate.

This product should be installed and fitted by a qualified rehab technician.

Ensure that the pelvic support belt is worn tightly fitted at all times. A loose belt may cause the user to slip down and create risk of strangulation.

Accidental release of the pelvic support belt can occur and cause the user to fall forward. If the user's movements or cognitive abilities could lead to accidental release, a healthcare professional should be present during the utilisation of these products.

The healthcare professional should regularly check for skin redness especially around bony prominences. If the user's skin develops redness, discontinue the use of this product immediately and see a doctor or therapist.

Ensure that the healthcare professional knows how to fasten and unfasten. Failure to do so may delay release in an emergency.

This product is NOT intended for transit transportation or personal restraint device. The pelvic support belt should only have positioning purpose in the wheelchair. Misuse of this product is unauthorized and may occur in damage or injury.

Padded Y style 4 points belt, adult and pediatric SC7590 and SC7591 (plastic buckle) SC7592 and SC7593 (metal buckle)



As it is fitted with Neoprene panels, this 4 point belt offers pelvic support on the iliac crest level and the crural level, thus keeping the spastic pattern under control. Also, the crural installed second strap holds the first strap, to keep their respective positions.

A magnetic lock can be added to our Y-style 4-point belt.

- o **Function** : to provide a secure closing mechanism of the support system.
- o **Safety Measures** :
 - Inspect the magnetic lock before use to ensure that the lock is well adjusted and adequately fastened. This component may cause injury if damaged. Do not use if damaged.
 - Do not submerge in any type of liquid. Liquids can cause a malfunction of the locking system.


If installed as the following, you will work on:

- Spastic pattern
- Pelvic rotation
- Pelvic Sliding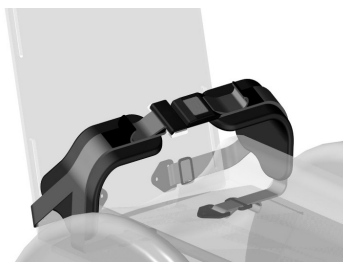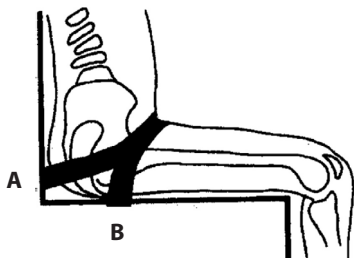

A - mounted on the backrest beam (low part)

B - mounted on the side tube (high part)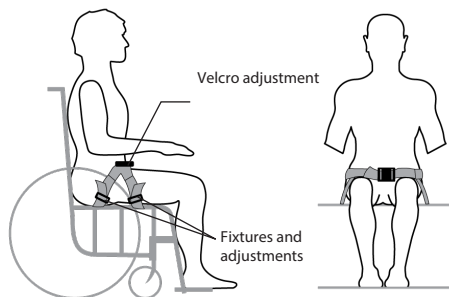


ENGLISH

INSTALLATION PRINCIPLES


At each end, the belt is fitted with anchor plates. These can be screwed on a rigid plate or simply on the wheelchair tubes (drill $\frac{1}{4}$ " or 6 mm M5 screw or existing hole) The belt location varies from one case to the next (morphology, weight), but the most important to know is that when it is appropriately installed, the belt reduces backward pelvic tip. A pre-adjustment can be done with the belt clips and final adjustments are made with the velcro. Velcro is much appreciated when frequently changing more cumbersome clothes like a coat, but should not be readjusted without checking the patient's installation depth in the wheelchair.

Belt SC7696 clip

The belt clips allow firmly fitting the strap ends without needing to drill the frame. The clips are fixed on tubes that have a 1 inch (2.5 cm) diameter.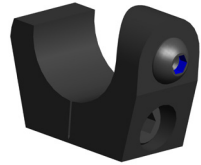

Padded Y style 4 points belt selection guide

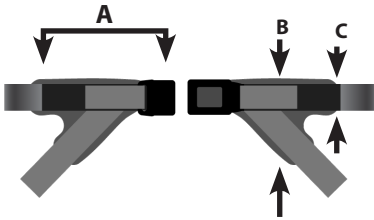	A	B	C	Maximum length
Child	7 po	5 po	1,5 po	35 po
Adult	9 po	6 po	2 po	62 po

Warranty

This product is covered with our twelve (12) month manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro Inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro Inc. or an authorized dealer to obtain your warranty services. Do not return your belt without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

Exclusions and Limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibility for damages incurred during shipping.

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