## Owners manual

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

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the Valeo Backrest, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional. Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your Valeo Backrest and extend its operational life. Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

*For Physipro inc., your satisfaction remains a priority.*


Comfortable due to its dispersion foam and dorsal straps, the liberty backrest favors the correction and balance of the trunk. It also acts in a way to prevent and minimize the risk of deformation.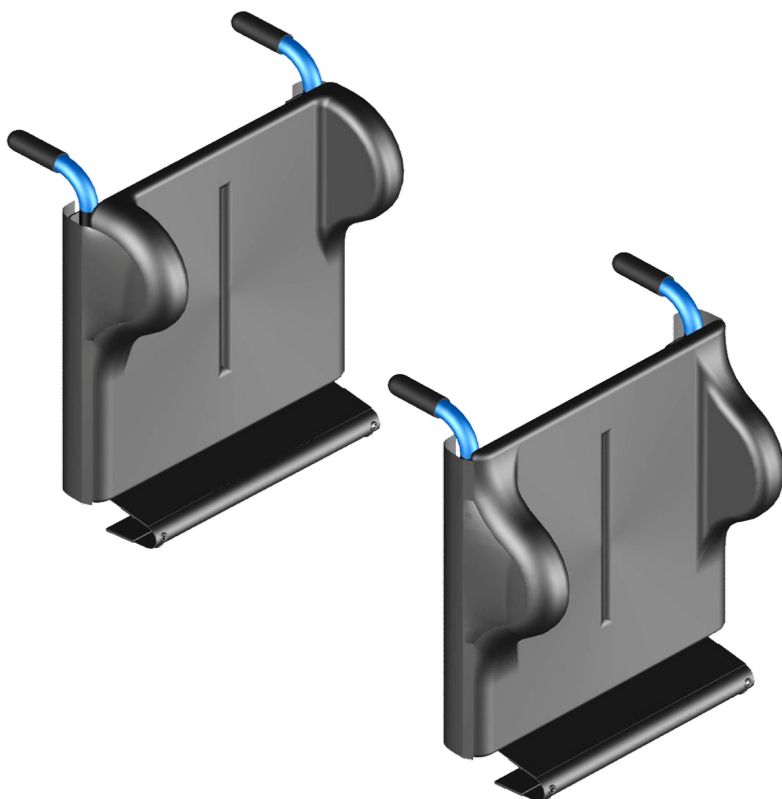

## Pieces included

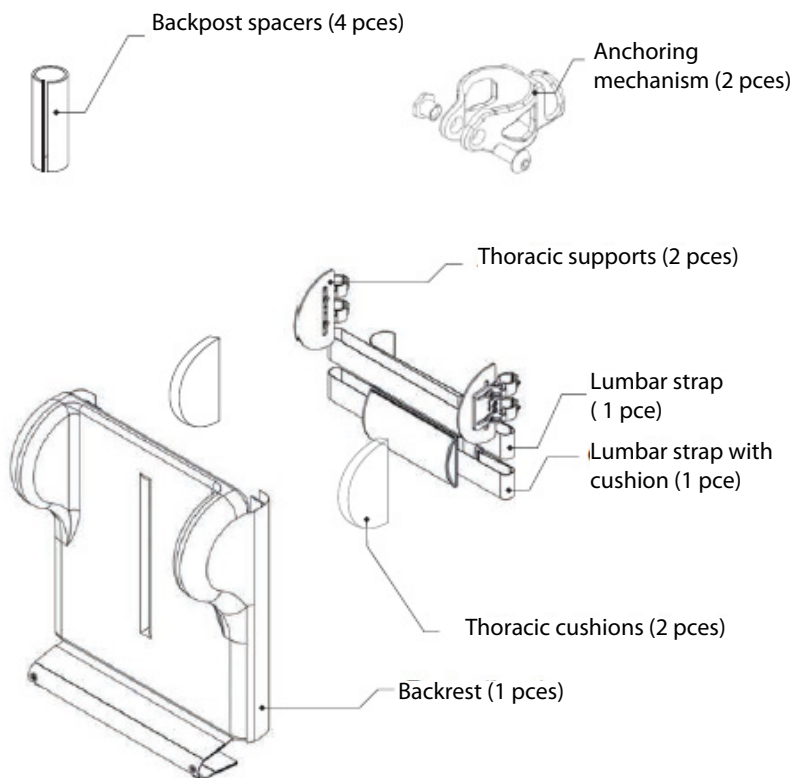
## Install the thoracic supports


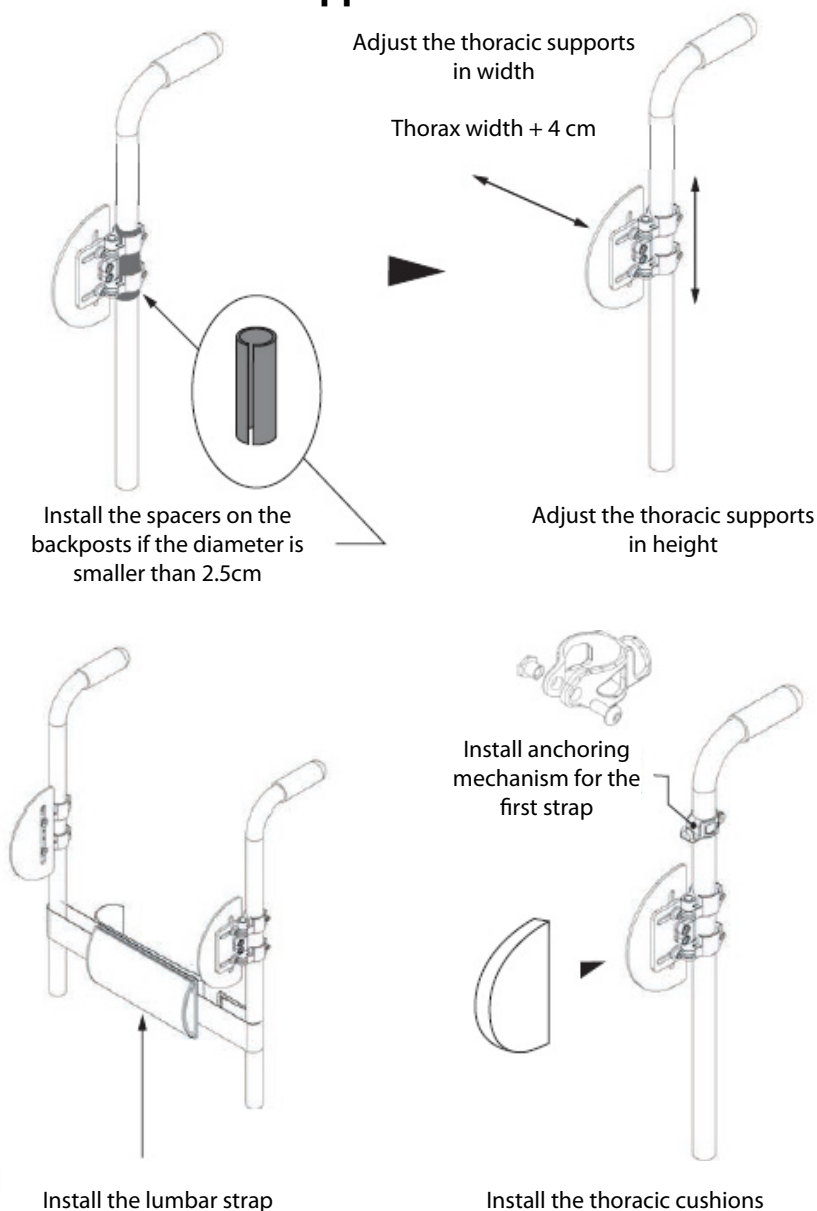
# ENGLISH



Install the backrest on the thoracic supports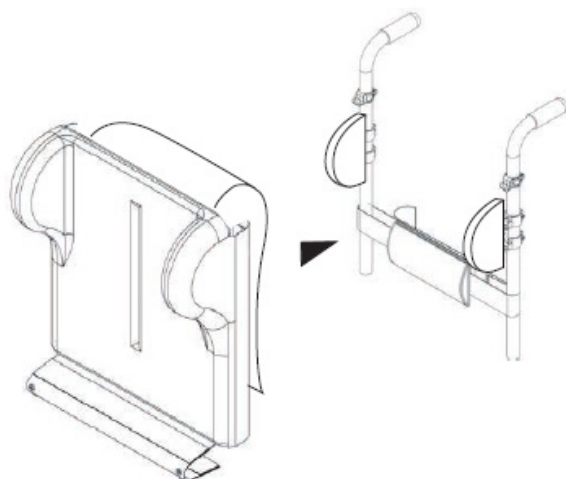


Insert each strap through the locking buckle and adjust the tension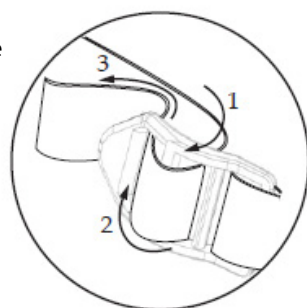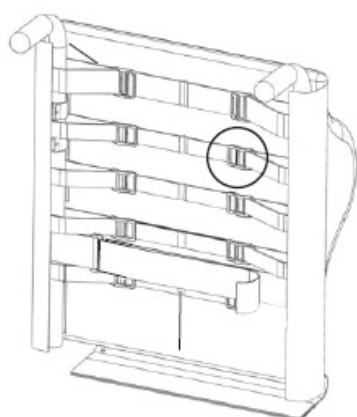

Fold the flap and close the two zippers

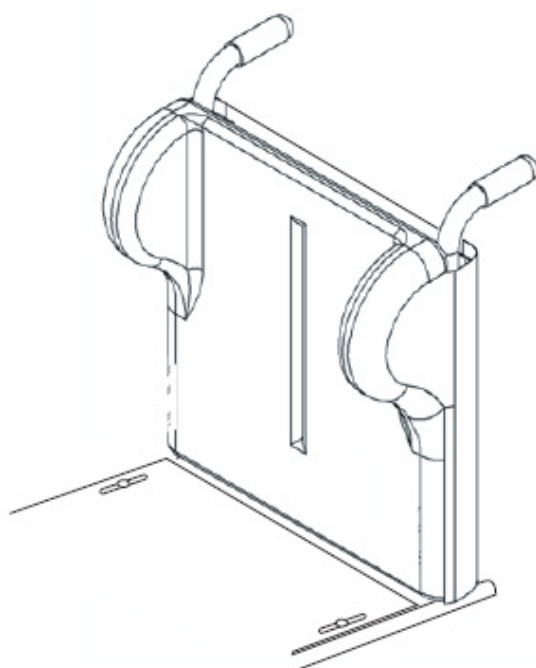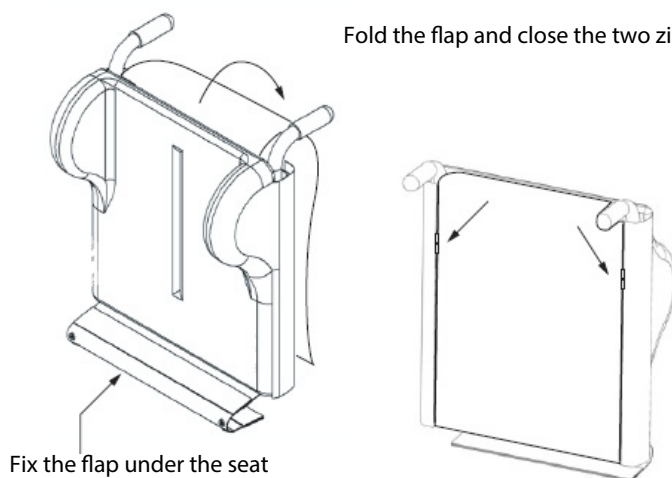
## ENGLISH
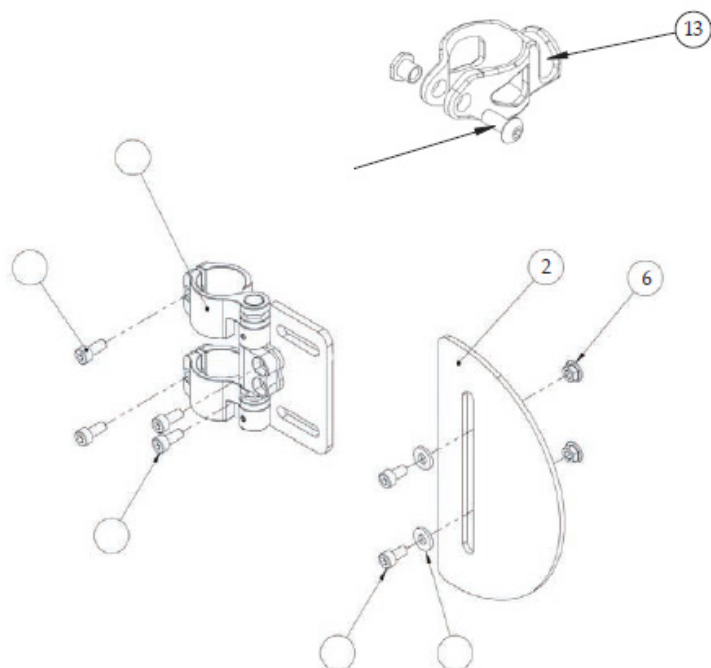
| no. | DESCRIPTION                                        | NO.<br>Produit |
|-----|----------------------------------------------------|----------------|
| 1   | Thoracic support anchoring mechanism (tube 2.5 cm) | D3845          |
|     | Thoracic support anchoring mechanism (tube 2.9 cm) | D3840          |
| 2   | Thoracic support plate - mini                      | D3823          |
|     | Thoracic support plate - small                     | D3825          |
|     | Thoracic support plate - medium                    | D3030          |
|     | Thoracic support plate - large                     | D3835          |
| 3   | M-5 16                                             | ST-51387       |
| 4   | M-5 10                                             | ST-51384       |
| 5   | M-5 Washer                                         | ST-51173       |
| 6   | Flange nut M-5                                     | ST-50332       |
| 7   | Anchoring mechanism spacer (tube 1.75 cm)          | D3862          |
|     | Anchoring mechanism spacer (tube 2.25 cm)          | D3861          |
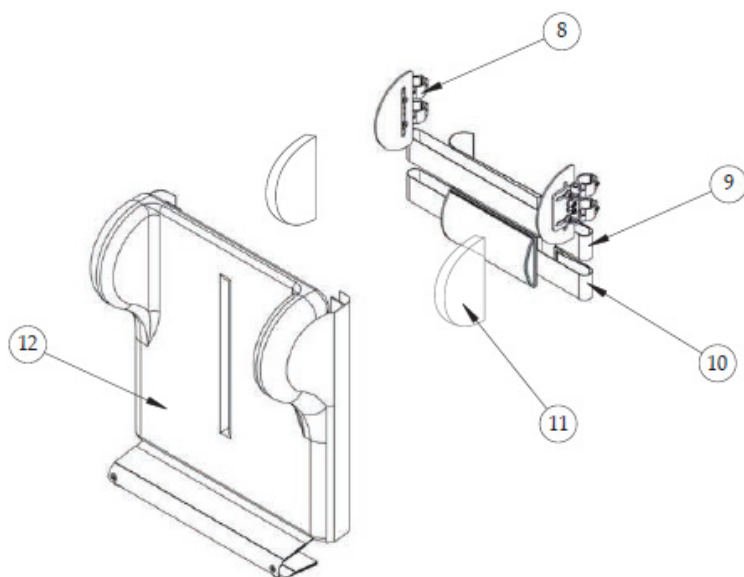
| no. | DESCRIPTION                             | NO. PRODUCT |
|-----|-----------------------------------------|-------------|
| 8   | Thoracic support Assembly - mini        | D3921       |
|     | Thoracic support Assembly - small       | D3822       |
|     | Thoracic support Assembly - medium      | D3823       |
|     | Thoracic support Assembly - large       | D3824       |
| 9   | Lumbar strap                            | D3866       |
| 10  | Lumbar strap with cushion               | D3865       |
|     | Lumbar strap with cushion - Customized  | D3867       |
| 11  | Thoracic cushion - mini                 | D3925       |
|     | Thoracic cushion - small                | D3926       |
|     | Thoracic cushion - medium               | D3927       |
|     | Thoracic cushion - large                | D3828       |
| 12  | Contour Liberty backrest - high profile | D3810       |
|     | Contour Liberty backrest - low profile  | D3815       |
|     | Contour Liberty backrest - Customized   | D3820       |
| 13  | Anchoring mechanism                     | D3030       |
| 14  | Button head M-5 16mm                    | ST-50044    |
| 15  | M5 Sleeve                               | PH-00145    |

**ENGLISH****Care and maintenance**

Liberty backrest maintenance is essential. By performing the maintenance in the recommended way, you will maximize its life cycle and ensure its safety of use. Do not forget to have your backrest checked up by a certified professional at least twice a year. Eliminate all stains (food and/or biological) as fast as possible.

| Check list                                                                                                                                                    | When receiving | Each week  | Each month | Every six month |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------|------------|------------|-----------------|
| <b>Backrest</b><br><br>Observe the condition of the aluminum;                                                                                                 | x              |            | x          |                 |
| <b>Anchors</b><br><br>Make sure the receptacles and the universal clamps of the backrest are solid;<br><br>Make sure the anchor slides and latches work well. | x<br><br>x     | x<br><br>x |            |                 |
| <b>Thoracic supports</b><br><br>Make sure that the thoracic supports are fastened properly;                                                                   | x              | x          |            |                 |
| <b>Clean up</b><br><br>Clean up the jointing.                                                                                                                 |                |            |            | x               |

## Warranty

The Liberty backrest, a product of Physipro inc., is covered with our twenty-four (24) months manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services. Do not return your backrest without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

## Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro inc. will not take any responsibilities for damages incurred during shipping.

## Terms and Conditions

The seat cushions and backrests are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment.

The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

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