User Manual
Manuel d'utilisation

This manual contains important information about this product. Please pass it to the final user upon delivery.

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ENGLISH

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the Mediclean wheelchair, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional. Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your Mediclean wheelchair and extend its operational life. Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company. ***For Physipro inc., your satisfaction remains a priority.***

Table of content

1. Components	3
2. Specifications	4
3. Recommendations	5
3.1 Safety Inspection Checklist	5
3.2 Safety Rules	6
3.3 Storage	6
4. Usage	7
4.1 Unfolding / folding the wheelchair	7
4.2 Brakes	8
4.3 Footrest adjustments	9
4.4 Safety belt	13
4.5 Accessories	14
5. Cleaning	15
5.1 Machine cleaning	15
5.2 Hand cleaning	15
6. Maintenance	16
6.1 Backrest cloth replacement	16
6.2 Upholstery cloth replacement	17
6.3 Brake adjustments	18
6.4 Rear wheel change	18
6.4.1 Rear wheel removal	18
6.4.2 Rear wheel insertion	18
6.5 Front wheel change	19
6.5.1 Front wheel removal	19
6.5.2 Front wheel insertion	19
6.6 Calfrest adjustments	20
6.6.1 Calfrest	20
6.6.2 Footrest rod	20
7. Warranty	21

1. Components

Figure 1 shows the Physipro Mediclean wheelchair’s main components.


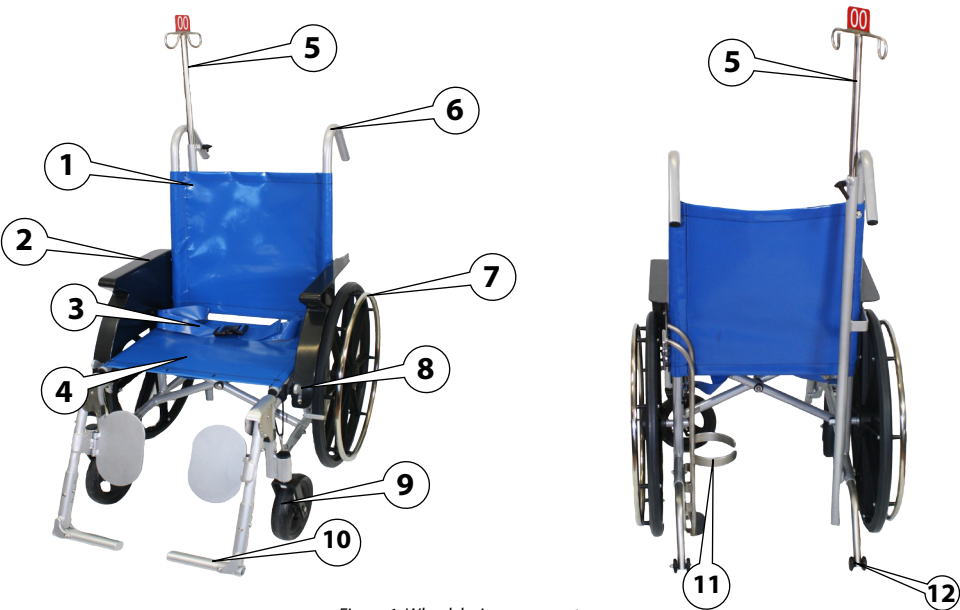
Figure 1, Wheelchair components

Description

- 1 Backrest
- 2 Armrest
- 3 Safety belt
- 4 Upholstery
- 5 IV pole
- 6 Push handle
- 7 Rear wheel
- 8 Brakes
- 9 Front wheel
- 10 Footrest
- 11 Oxygen bottle support
- 12 Anti-tip

ENGLISH

2. Specifications

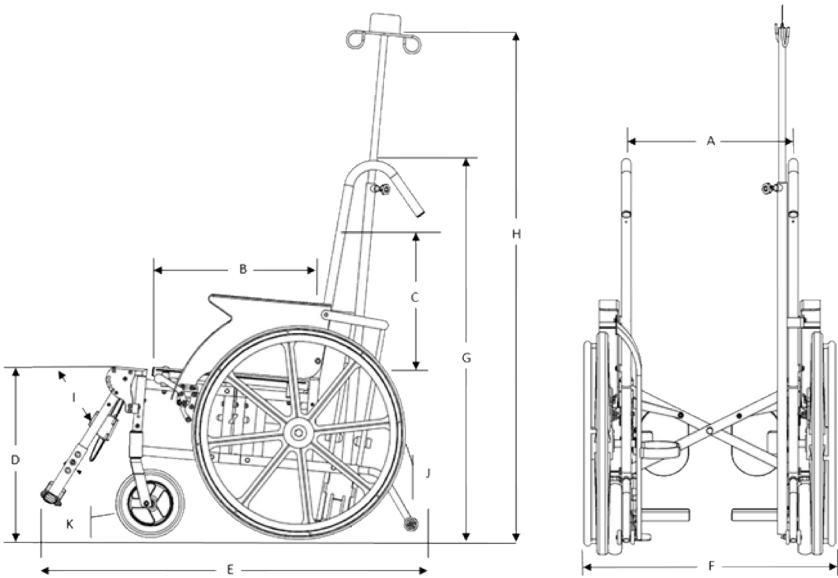A	Width	18"
B	Depth	16"
C	Backrest height	18"
D	Seat/ground height	18"
E	Overall length	41"
F	Overall width	26"
	Folded overall width	< 12"
G	Overall height	41"
H	IV pole height	38" to 72"
I	Legrests angle adjustment	0° to 60°

J	Back wheels diameter	24"
K	Front wheels diameter	8"
	Maximum weight	250 lb
	Total weight	47 lb

3. Recommendations

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the Mediclean wheelchair.

3.1 Safety Inspection Checklist

Maintenance procedures:

- ☐ Ensure that the wheelchair rolls straight and that all parts work smoothly.
- ☐ Check for noise, vibration, or a change in ease of use.
- ☐ Make sure that the wheel locks do not interfere with tires when rolling.
- ☐ Verify that the wheel locks are easy to engage.
- ☐ Make sure the anti-tip tubes lock in place.
- ☐ Ensure that the wheel locks prevent the wheelchair from moving when engaged.
- ☐ Ensure the seat and backrest are stable and secured to the frame;
- ☐ Inspect the seat and backrest for loose or broken hardware.
- ☐ Check for proper tire inflation pressure;
- ☐ Inspect tires for flat spots and wear.
- ☐ Ensure armrests are secured when locked;
- ☐ Check if footrests are positioned and secured properly;
- ☐ Ensure there is no heavy objects fixed to the backrest;
- ☐ Ensure the seat positioning strap is secured to the frame and adjusted to the client's needs.
- ☐ Check to see that both quick-release rear axles are locked.

ENGLISH

3.2 Safety

Several safety measures must be applied to ensure the safety of the user and the individuals who handle or use the wheelchair. Note that the following list is not exhaustive. Therefore, the user and the individuals who handle or use the wheelchair have the responsibility to handle it or use it cautiously.

- ✓ Never use the wheelchair with improper tire inflation pressure;
- ✓ Never attempt to reach an object if you have to lean forward, backward or sideways;
- ✓ Do not attempt to approach an obstacle whose height could reduce the stability of the wheelchair;
- ✓ Never carry passengers;
- ✓ Do not fasten objects to the backrest, except those provided with the wheelchair;
- ✓ Do not attempt to tilt the wheelchair without any assistance;
- ✓ Never use the footplate as a platform. When getting in or out of the wheelchair, make sure that the footplates are in the upward position.
- ✓ Do not stand on the frame of the wheelchair.
- ✓ Do not expose the wheelchair directly to the sun, the hot surfaces could cause severe burns;
- ✓ Always keep fingers and hands clear of any moving parts;
- ✓ Use the provided anchorage rings for specialized transportation. These anchorage rings do not substitute for the restraining system certified by Transport Canada.

3.3 Storage

The Mediclean wheelchair is designed to minimize storage space. However, these conditions need to be respected:

- ✓ Store the wheelchair in a dry place. Even though the wheelchair is corrosion resistant, prolonged exposure to damp conditions can lower the wheelchair's lifetime and damage the cloth.
- ✓ It is possible to use the push handles to store the wheelchair standing.
WARNING, it is quite heavy. Ask help if needed as to not injure yourself by lifting the wheelchair's weight.

4. Usage

4.1 Unfolding / folding the wheelchair

4.1.1 Folding the wheelchair

Before folding the wheelchair, make these verifications:

- A. There is nothing hanging from the IV pole or the catheter hook
- B. There is no oxygen bottle in the bottle support (it causes a considerable interference problem with the IV pole)
- C. The safety belt is attached under the upholstery
- D. The calfrests and footrests are closed.
- E. The legrests should not be closed under the wheelchair

Figure 2, Wheelchair ready to be folded

Take the upholstery by the front and back ends and lift it (figure 3) until the wheelchair is completely folded.

Figure 3, Folding the wheelchair

Figure 4, Folded wheelchair

ENGLISH**4.1.2 Unfolding**

To unfold the wheelchair, move apart the armrests from each other and firmly press the upholstery sidebars until they are well pushed in the frame receptacle as shown in figure 5.

Figure 5, Unfolding the wheelchair

4.2 Wheel Locks

When the wheelchair is not moving, you can apply the wheel locks if needed. It is very important to apply the wheel locks when transferring a patient to avoid wheelchair knock back during transfer.

To activate the wheel locks, push handle **A** down;

To deactivate the wheel locks, pull handle **A** up

Figure 6, Brakes

4.3 Legrest adjustments

The Mediclean wheelchair Legrests are adjustable to aid in many situations.

4.3.1 Length adjustment

Depending on patient height, 3 adjustment levels are available.

Simply press the buttons **A** as shown in figure 7 and slide lower part **B** on desired position.

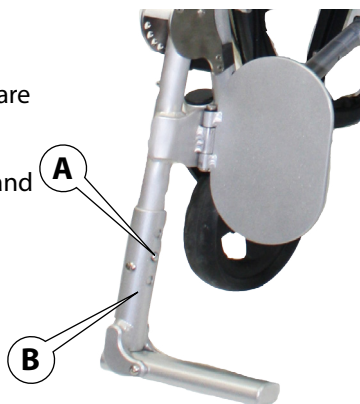
Figure 7, Legrest length adjustment

4.3.2 Elevation adjustment (angle)

It is possible to lift the legrest up according to the patients needs by pressing the push button **C** (that releases the button **D**) as shown in Figure 8.

Change the angle of the legrest beams as needed and release the push button **C**. Make sure the button **D** is engaged in one of the five holes before releasing the patient's leg.

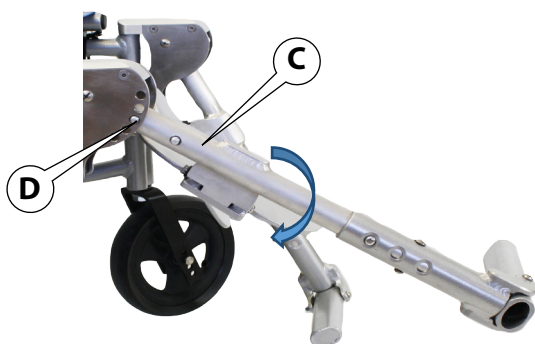
Figure 8, Angle adjustment

ENGLISH

4.3.3 Elevation adjustment (height)

Some postural needs require elevating the legrest. For example, if you need to have the calfrest almost at the same level as the upholstery as shown in Figure 9.

Figure 9, Raised legrests

To adjust height, pull up the block **E** as shown in Figure 10. Insert the pin **F** in one of the 2 adjustment holes available **G**. Holes are by in 1 inch increments. Push down on block **E** until the footrest is pressing on the pin.

Note : A permanent rod **H** is placed in the lowest position (reference position). The pin is just hanging from its wire on that position.

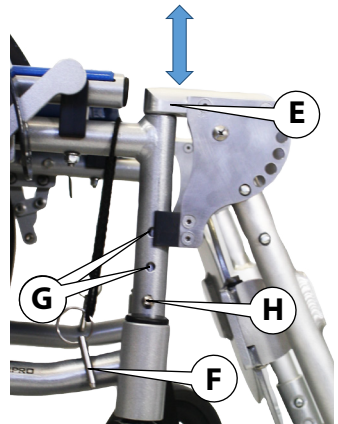
Figure 10, height adjustment

4.3.4 Folding the legrest

Before folding the legrest, it is important to fold the calfrest and the footrest rod to avoid any interference. Simply close them as shown in Figure 11.

Figure 11, Folding the calfrest and the footrest rod

ENGLISH

The legrests can be folded under the wheelchair upholstery to enable feet movement.

When the calfrests and the footrest beams are folded, slightly lift block **E** and close the legrest towards the inside of the wheelchair as shown in Figure 12.

Figure 12, Closing the legrests

It is also possible to pivot the legrests toward the outside to ease a patient's lateral transfer as shown in Figure 13.

Figure 13, Closing toward the outside

4.4 Safety belt

It is very important to correctly adjust the safety belt to maximize the patient's comfort and safety.

To adjust the belt length, slide strap **A** in plastic buckle **B** as to lengthen or shorten the belt.

To engage the buckle, push the pin in the box until a click is heard. Make sure the buckles are locked.

To unlock the buckles press the parts identified as **C** until it is released.

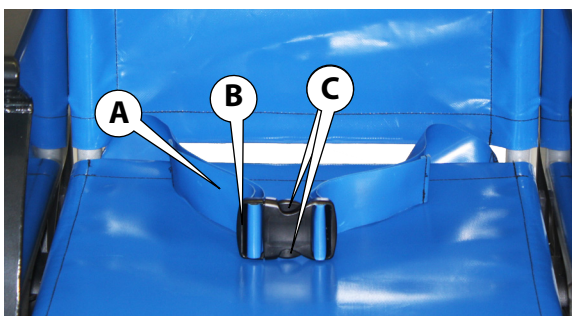
Figure 14, Safety Belt

Note: When the belt is not in use, buckle the two parts and leave the belt hanging under the upholstery. This will make folding the wheelchair easier.

ENGLISH

4.5 Accessories

4.5.1 IV pole

The height adjustable IV pole can be retracted to ease machine wash as shown in Figure 15.

To adjust height, unscrew handle **A** until pole **B** freely moves. Adjust the height and screw handle **A**.

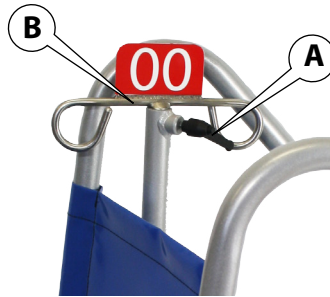
Figure 15, IV pole adjustment

Make sure the IV pole is securely fixed without overtightening the handle. This could damage the pole or make unscrewing impossible for another attendant.

4.5.2 Oxygen bottle

To insert an oxygen bottle in the support, simply slide the bottle until it is pressed on the bottom of the oxygen tank holder. **A** spring mechanism ensuring the oxygen bottle is fixed.

To take back the oxygen bottle, pull up on it.

Figure 16, Oxygen bottle support

4.5.3 Catheter loop

You can use the loop under the upholstery to hang a catheter bag.

Figure 17, Catheter loop

Make sure you unhook the bag before transferring the patient to avoid wound risks.

5. Cleaning

Eliminate all stains (food and/or biological) as fast as possible.

5.1 Cleaning with a wheelchair washing machine

This wheelchair is designed to be cleaned in a wheelchair washing machine. Lower the IV pole by following the instructions in section 4.5.1 and make sure there is no bottle, bag or any other object that is not part of the wheelchair. Insert the wheelchair in the wheelchair washing machine. Let it dry.

5.2 Hand cleaning

It is possible to use a cleaning agent like Virox® or Chlorox® towelettes on every surface. A home made 1 for 10 bleach water solution, a soft soap or a disinfectant solution can be sprayed or applied with a cloth. Rinse if needed and let it dry.

ENGLISH

6. Maintenance

As a precaution, a monthly functionality problem inspection should be made. This inspection includes:

- ☐ Check the wheel wear.
- ☐ Make sure the wheels turn freely.
- ☐ Make sure the hand rims are secure.
- ☐ Make sure the wheel locks work.
- ☐ Make sure the armrests are secure.
- ☐ Check if the backrest and upholstery cloths are intact and cleaned.
- ☐ Check if the footrests work.
- ☐ Test the effectiveness of brakes and make sure the wheel lock is firmly in place.
- ☐ Verify that every screw is tightend and tighten screws that might have worked loose by adding threadlocker "LOCTITE" type.

Contact your Physipro representative if you need assistance. If a part needs to be replaced, please look at the parts catalogue to make sure you have the right part number.

6.1 Backrest upholstery replacement

To remove the upholstery, remove the bolts **A** on each side with a 5/16 (8 mm) wrench.

Slowly tip over the IV pole and bottle support toward the ground.

Figure 18, Backrest cloth replacement

Also remove the screws **B** and be careful not to lose the flat washer.

Slightly fold the wheelchair so it can slide easier. Slide up the upholstery to remove it by the push handles.

Figure 19, Rear View of the upholstery Backrest

Insert the new upholstery on the push handles, eyelets up. Align the eyelets with the holes in the handles. Place the wheelchair in an unfolded position. Reinsert the bolts **A** and screws **C** while not forgetting the flat and lock washers.

6.2 Upholstery replacement

To remove the upholstery, remove the caps **A** with a regular screwdriver.

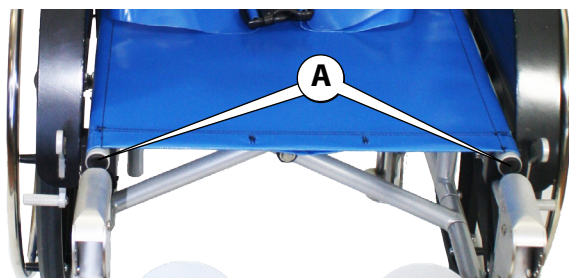
Figure 20, Upholstery replacement

Slightly fold the wheelchair so it can slide easier in the next steps.

Pull off aluminum rod **B** as in Figure 21. The upholstery can now be pulled from the crossbrace grooves.

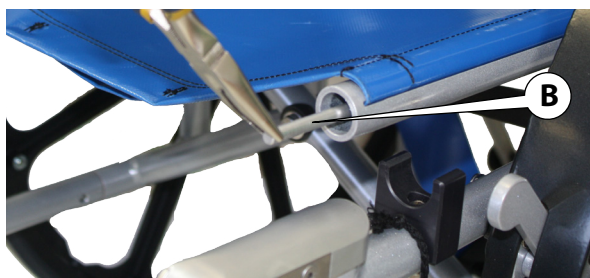
Figure 21, Upholstery replacement

Place the new upholstery, making sure the catheter loop is in front. Insert both sides in the crossbrace grooves. Insert the aluminum rods in the cloth eyelets, in the crossbrace tubes. Completely unfold the wheelchair to stretch the upholstery and set the rods in position. Put caps **A** back on.

ENGLISH

6.3 Wheel Locks Adjustments

It is possible to adjust the wheel locks if they are inefficient or too hard to engage. Start by measuring the distance between the wheel and the wheel locks rod **A** when the brake is off. This distance is a reference for the adjustment.

Use two 10mm wrenches to unscrew the bolt **B** and nuts **C** assemblies. Place the wheel locks closer to the wheel for a more effective stopping. Place the wheel locks farther from the wheel for ease of engagement of wheel locks. Tighten bolt **B** and nuts **C** assemblies at the desired position.

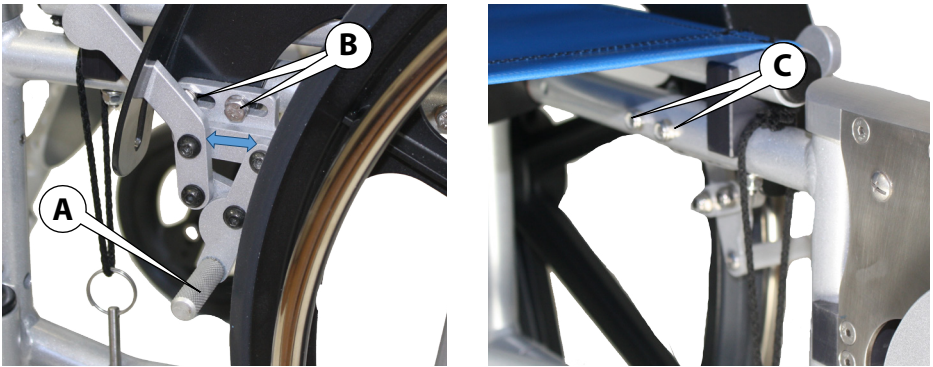
Figure 22, Wheel Locks Adjustments

6.4 Rear wheel change

6.4.1 Rear wheel removal

Loosen nut **C** with a $\frac{3}{4}$ inch wrench while holding bolt **A** with a $\frac{3}{4}$ inch ratchet wrench.

Slide the wheel out of the axle **B**, take bolt **A** off the wheel.

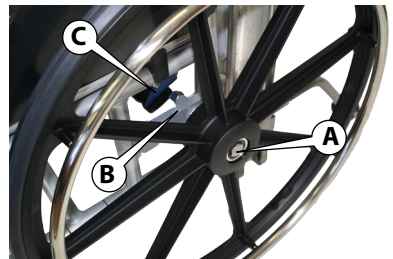
Figure 23, Rear Wheel Change

6.4.2 Rear wheel insertion

Place bolt **A** in the wheel axle. Slide the bolt in positioning axle **B**. Tighten nut **C** with a $\frac{3}{4}$ inch while holding bolt **A** with a $\frac{3}{4}$ inch ratchet wrench.

6.5 Front wheel change

6.5.1 Front wheel removal

Loosen bolt **A** and nut **C** with a 13 mm wrench.

Take bolt **A** out of the wheel axle. Take the wheel off while keeping spacers **B** in place.

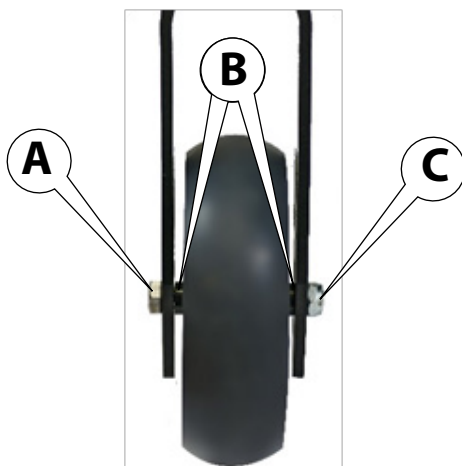
Figure 24,Front Wheel Change

6.5.2 Front wheel installation

Place the wheel axle and spacers **B** aligned with the steering stem. Insert bolt **A**. Secure bolt **A** and nut **C** with a 13 mm wrench.

Once secured, the wheel should turn normally. Do not over tighten.

ENGLISH**6.6 Calfrest and footrests**

The calfrest plates and footrest rods have friction control systems to avoid shaking. It is possible to adjust this friction as the user needs.

6.6.1 Calfrest

To adjust calfrest friction, use a 3 mm Allen key to loosen or tighten bolt **A** to the desired friction.

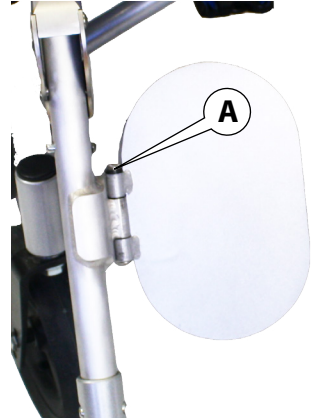
Figure 25, adjustment of the footrest rod friction

6.6.2 Footrest rod

To adjust footrest rod friction, use a 4 mm Allen key and a 10 mm wrench to loosen or tighten the bolt/nut assembly **A/B** to the desired friction.

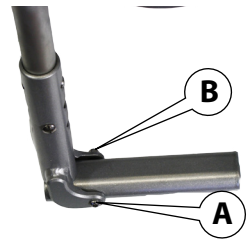
Figure 26, adjustment of the footrest rod friction

7. Warranty

3 years

Frame and cross-brace against defects in materials and workmanship.

1 year

Parts and labor for a period of 1 year, covers original defects in material and workmanship of the wheelchair components, optional components and new parts that have been replaced by the supplier or by an authorized dealer, with effect from the delivery date or, if applicable, from the date of replacement, with a copy of the seller's invoice required for coverage under this warranty.

30 days

Tires, inner tubes, comfort upholsteries, armrests, seat covers, backrest covers and ball bearings.

Physipro inc. agrees to repair or replace any inoperative or defective part for the duration of the warranty period. To enjoy this warranty service, please contact Physipro Inc. or an authorized dealer. Do not return this product without prior approval. If you are dissatisfied with this product, send us your comments at the address provided therein, and include the name and address of the supplier, the delivery date, and the serial number of the product.

Limitations and Exclusions

This warranty shall not apply to products that have been altered or abused, whether intentionally or by accident, or been subject to improper use, negligence, lack of maintenance or improper storage, and products whose serial number has been erased or removed. Furthermore, this warranty shall not apply to products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized dealer. The same restrictions shall apply for damages resulting from other circumstances beyond the control of Physipro Inc. Finally, this warranty shall not apply to normal wear and tear or to any non-compliance with the guidelines provided therein.

Physipro Inc. shall not be responsible for any damage that occurs to the product during transportation.

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