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Owner's Manual

This booklet contains important information about this product. Please give to the owner upon delivery.

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INTRODUCTION - ENGLISH

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

ENGLISH

DESCRIPTION

Comfortable, lightweight, durable and easy to use, The Premium™ ensures complete bilateral control of the posterior and anterior curvatures of the spine, providing the ideal back support.

Two covers are available: Standard or with Velcro Surface

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Padding

1" Urethane

Covers

Standard: Oxford and Triptex U

Velcro Surface: Oxford and Velcro Loop Fabric

Mounting system

Nylon straps, buckles and screws

Backpost covers (optional)

OPTIONAL COMPONENTS

Incontinence cover: Transparent cover with velcro

Folding tension bar: Standard or with headrest mounting fixture

Lateral supports : Adjustable, Foldable and adjustable or Fixed to back post
Anti-hammock effect (available with Velcro Surface only)

Adjustable

Foldable and adjustable

Fixed to back post

Anti-Hammock Effect (Available with Velcro Surface only)

TECHNICAL SPECIFICATIONS

Nominal Widths: 12" to 26" (305 mm to 660 mm)
Nominal Heights: 12" to 26" (305 mm to 660 mm)
Thickness (Padding included): 1"

Sizes associated with backrest

Wheelchair Width	Nominal Width	All widths are available in the following Nominal Heights:
12" (305 mm)	12" (305mm)	12" (305 mm)
13" (330 mm)	13" (330mm)	13" (330 mm)
14" (355 mm)	14" (355mm)	14" (355 mm)
15" (380 mm)	15" (380 mm)	15" (380 mm)
16" (405 mm)	16" (405 mm)	16" (405 mm)
17" (430 mm)	17" (430 mm)	17" (430 mm)
18" (455 mm)	18" (455 mm)	18" (455 mm)
19" (480 mm)	19" (480 mm)	19" (480 mm)
20" (505 mm)	20" (505 mm)	20" (505 mm)
21" (530 mm)	21" (530 mm)	21" (530 mm)
22" (560 mm)	22" (560 mm)	22" (560 mm)
23" (585 mm)	23" (585 mm)	23" (585 mm)
24" (610 mm)	24" (610 mm)	24" (610 mm)
25" (635 mm)	25" (635 mm)	25" (635 mm)
26" (660 mm)	26" (660 mm)	26" (660 mm)

RECOMMENDATIONS

To ensure your backrest provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the backrest components, please refer to the Maintenance checklist, page 16.
- Physipro Inc. backrests are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always install backrest with the provided mounting hardware.
- Never push or lift wheelchair with backrest, doing so may cause the backrest to detach from the wheelchair and can result in injury or damage.
- Before installation, make sure the backrest is compatible with the wheelchair's dimensions.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the backrest, this will diminish the backrest efficiency and increase the risk of developing skin lesions.
- It is preferable to wear clothes made of cotton, that are not too tight and do not have seams on pressure points.
- Before use, let the backrest get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Remove any foreign object (crumbs, debris, etc.) from the backrest surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.

It is recommended to consult your therapist in the month following the installation of a new backrest to make sure it is efficient.

WARNINGS

Consult a healthcare professional: Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required seating system and make any necessary adjustments.

Center of gravity: Installing a backrest on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, possibly resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new backrest, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted to increase stability.

Backrest dimensions: the dimensions of the backrest must be adapted to the user and the wheelchair, if not, the backrest's benefits will be reduced or eliminated.

Backrest and cover compatibility: the backrest cover must be well-fitted and used as directed in this manual to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the backrest and can reduce stability.

Environment: Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the backrest to extreme cold or heat. Never place backrest near open flame.

Instability/ Fall risk: Never place other products or materials (except those specified in this manual) between the user and the backrest, doing so, will cause instability and increase the risk of injury from a fall.

Positioning the backrest: Proper positioning of the adjustable tension straps is essential to ensure maximum comfort and support.

Skin inspection: Backrest should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro Inc. recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a healthcare professional immediately. Be aware that no backrest can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

Transport: Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

ENGLISH


INSTALLATION

Please read all the following instructions before beginning installation. It is recommended to follow the steps in the order indicated.

Required tool: Square screwdriver #2 (North America) or "Phillips" screwdriver (Europe)

Installing the backrest (without backpost covers)

1. Wrap the two top adjustment straps **A** around the backposts and secure with screws.
2. Insert straps **A** through the backrest loops **B** and into buckles **C** of the central piece.
3. Wrap each adjustment strap **D** around the backpost and into buckles **E** of the central piece.
4. Adjust the tension of each strap individually to meet the positioning and postural needs of the wheelchair occupant.
5. Insert cable ties through the lower grommets **F** and fasten the backrest to the wheelchair frame.
6. Flip the back cover flap over the adjustment straps.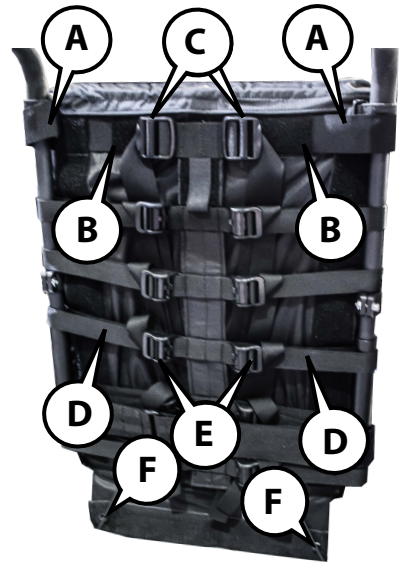


WARNING

Do not tighten the straps excessively, this may damage the backposts.

Installing the backrest (with backpost covers)

1. Wrap the backpost covers **A** around the backposts and secure with screws.
2. Insert straps **B** through the backpost cover buckles **C** and into buckles **D** of the central piece.
Note: The upper straps must be inserted in the backrest loops.
3. Adjust the tension of each strap individually to meet the positioning and postural needs of the wheelchair occupant.
4. Insert cable ties through the lower grommets **E** and fasten the backrest to the wheelchair frame.
5. Flip the back cover flap over the adjustment straps.

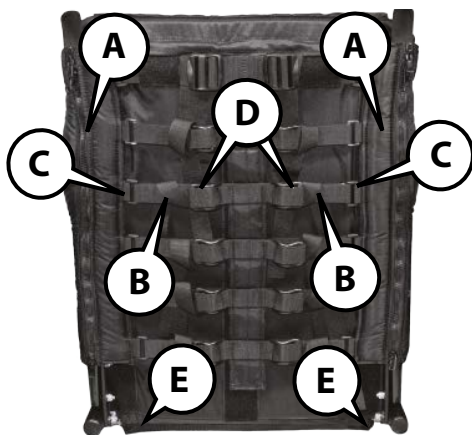WARNING

Do not tighten the straps excessively, this may damage the backposts.

Adjusting the tension of the backrest

1. Lift up the back cover flap to expose the adjustable straps **A**.
2. Adjust the tension of each strap individually to meet the positioning and postural needs of the wheelchair occupant.
3. If a lumbar strap is installed, adjust the tension of the lumbar strap **B**.
4. Once all adjustments are completed, flip the back cover flap over the adjustment straps.

WARNING

Do not tighten the straps excessively, this may damage the backposts.

ENGLISH


LATERAL SUPPORTS (OPTION)

Lateral supports are equipped with universal mounting clamps that permit installation on $\frac{3}{4}$ " to 1" backposts.

Installing the Adjustable or Foldable and adjustable lateral supports

Required tool: 4mm Allen key


1. Open the mounting clamp by unfastening screws **A** and loosening set screws **B** with a 4mm Allen key.
2. Open the zippers of the backpost covers.
3. Install the mounting clamp on the backpost with the screw heads facing outwards.
4. Position the mounting clamp at the desired height and tighten screws **A** and set screws **B**.
5. Repeat these steps for the second lateral support. Make sure that both mounting clamps are positioned at the same height.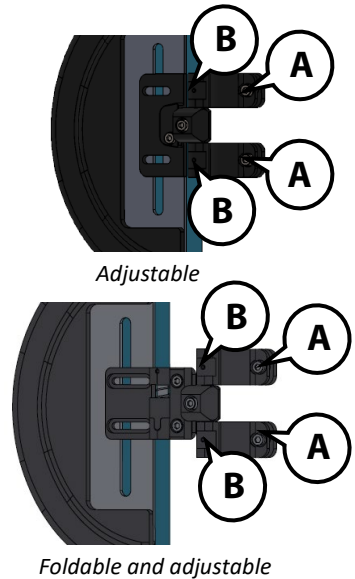

Installing the Fixed to backpost lateral supports

Required tools: 4mm and 5mm Allen keys

1. Open the mounting clamp by unfastening screws **A** with a 5mm Allen key and loosening set screws **B** with a 4mm Allen key.
2. Open the zippers of the backpost covers.
3. Install the mounting clamp on the backpost with the screw heads facing outwards.
4. Position the mounting clamp at the desired height and tighten screws **A** and set screws **B**.
5. Repeat these steps for the second lateral support. Make sure that both mounting clamps are positioned at the same height.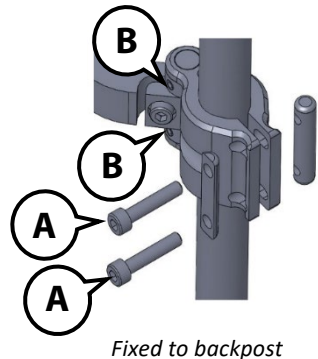

Adjusting the lateral supports


All of Physipro's lateral supports can be adjusted to improve trunk alignment and stability. Please refer to the adjustment section for your model of lateral support.

Adjustable lateral support

Required tool: 4mm Allen key


Adjusting the lateral support to the occupant's torso

1. Slightly loosen screw **C**.
2. Position the lateral support as near as possible to the occupant's torso.
3. Fasten screw **C** tightly.
4. Repeat these steps for the second lateral support.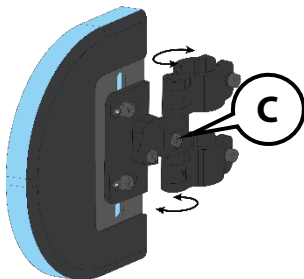


Adjusting the lateral support's vertical angle

1. Slightly loosen screw **D**.
2. Position the lateral support at the desired vertical angle.
3. Fasten screw **D** tightly.
4. Repeat these steps for the second lateral support.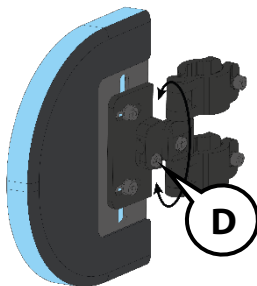

Adjusting the lateral support depth and height

1. Slightly loosen screws **E**.
2. Position the lateral support's depth and height by moving it in the horizontal and vertical adjustment slots.
3. Fasten screws **E** tightly.
4. Repeat these steps for the second lateral support.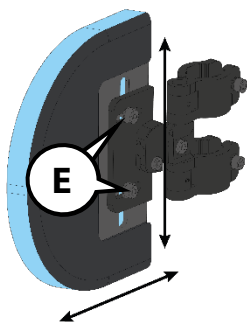


ENGLISH

Foldable and adjustable

Required tool: 4mm Allen key


Adjusting the lateral supports to the occupant's torso

1. Push mechanism **C** upwards and pivot inwards or outwards to position the lateral support as near as possible to the occupant's torso.
2. Repeat these steps for the second lateral support.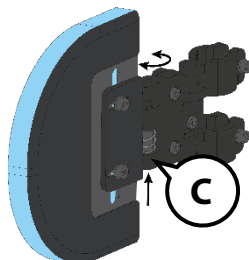


Adjusting the lateral support's vertical angle

1. Slightly loosen screw **D**.
2. Position the lateral support at the desired vertical angle.
3. Fasten screw **D** tightly.
4. Repeat these steps for the second lateral support.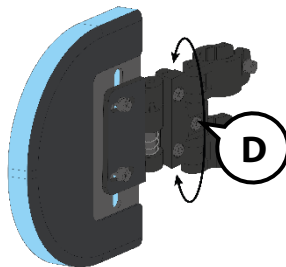

Adjusting the lateral support depth and height

1. Slightly loosen screws **E**.
2. Position the lateral support's depth and height by moving it in the horizontal and vertical adjustment slots.
3. Fasten screws **E** tightly.
4. Repeat these steps for the second lateral support.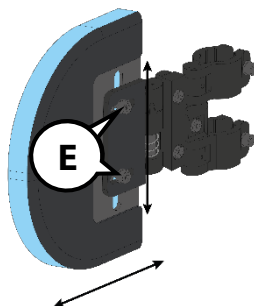


Fixed to back post

Required tool: 4mm Allen key


Adjusting the lateral supports to the occupant's torso

1. Slightly loosen screws **C**.
2. Position the lateral supports as near as possible to the occupant's torso.
3. Fasten screws **C** tightly.
4. Repeat these steps for the second lateral support.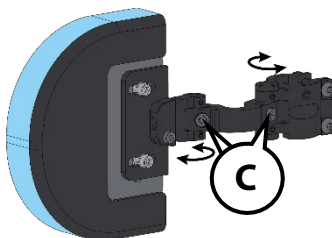


Adjusting the lateral support's vertical angle

1. Slightly loosen screws **D**.
2. Position the lateral support at the desired vertical angle.
3. Fasten screws **D** tightly.
4. Repeat these steps for the second lateral support.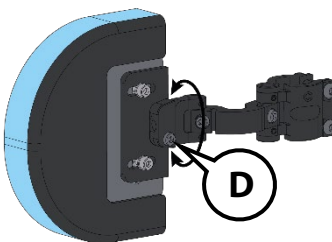

Adjusting the lateral support depth and height

1. Slightly loosen screws **E**.
2. Position the lateral support's depth and height by moving it in the horizontal and vertical adjustment slots.
3. Fasten screws **E** tightly.
4. Repeat these steps for the second lateral support.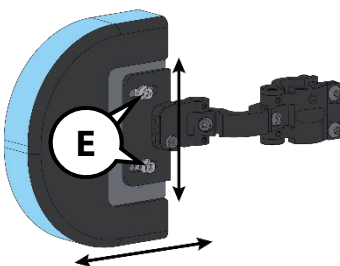

ENGLISH

Installing the lateral supports covers

1. Completely unfasten the cover's zipper.
2. Insert cover, starting at the top of the lateral support.
3. With a downward movement, push cover around the lateral support.
4. Fasten the zipper.


Anti-hammock effect lateral supports (Velcro Surface only)

Intalling the lateral supports

Once the installation of the Velcro Surface backrest is completed, you can install the lateral supports.

1. Position the lateral supports **A** on the backrest **B**.
Note: Make sure the height and distance between laterals is correctly adapted to the wheelchair occupant.
2. Once the position of the lateral supports is correctly determined, press the lateral supports **A** firmly on the backrest **B** to secure the velcro fastening system.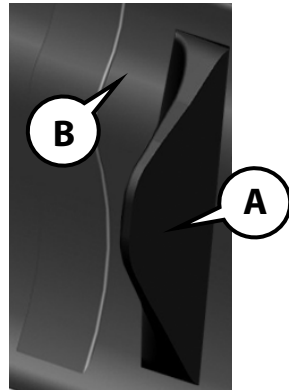

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your backrest. It is recommended to visually inspect the components of the backrest during the cleaning process.

Cleaning the backrest and lateral support covers

1. Remove cover from backrest and/or lateral supports and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam padding.

**WARNING**

Do not iron.

Cleaning the backrest and lateral supports padding

1. Remove backrest cover and the incontinence cover, if necessary.
2. Wipe foam padding lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on padding to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s) on the backrest.

**WARNINGS**

- Do not use soap.
- Do not submerge in water.
- Never machine wash or dry the foam padding.

Cleaning the straps and buckles

Wipe straps and buckles lightly with a damp cloth, using a mild detergent and warm water.

**WARNINGS**

- Do not machine wash the straps.
- Do not iron.

Cleaning the hardware

The aluminum hardware can be cleaned with warm water and a mild liquid detergent. No other maintenance is required.

ENGLISH**Maintenance checklist**

Upon receiving your backrest, it is important to complete all the inspections listed in the maintenance checklist to ensure the proper function and condition of the backrest and mounting hardware.

Maintenance checklist	Weekly	Monthly	Every 6 Months
Backrest - Check backrest and straps for any signs of wear		✓	
Lateral Supports - Verify the solidity of mounting clamps - Check lateral supports for any signs of wear	✓	✓	

Storage

Backrest must be stored in a dry and secure area.

**WARNINGS**

- Never store backrest in direct sunlight or leave it exposed to excessive humidity or heat.
- Never place backrest near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your backrest is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty periods for the parts and components of the Premium backrest are as stated:

Backrest: Twenty-four (24) months

Lateral supports : Twelve (12) months

Outer cover: Six (6) months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period.

Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at order@physipro.com, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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