Owners manual

This manual contains important informations about this product.
Please pass it to the final user upon delivery.

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ENGLISH

WARNING!

Physipro recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a Fusion® Cushion is appropriate.

Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

You should regularly check for skin redness especially around bony prominences. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist. The Fusion® Cushion is used to help reduce pressure. However, no cushion can completely eliminate sitting pressure or prevent pressure sores.

Do not place any obstructions between the user and the cushion because it will reduce product effectiveness. This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

Installation

Stick the rear part of the cushion (look for indications on the cover) in the back of the seat. If there is an abductor stop on the cushion, make sure the external rotator is positioned towards the front.

Usage precautions

The cushion must be installed on a clean and flat surface of good quality. During the installation, check the dimensional compatibility between the cushion and the seat and adjust the height of the armrests. It is preferable to wear clothes made of cotton, that are not tight and, if possible, without seams on the resting zones. Do not insert any towel, folded bed sheets or any other surface between the user and the cushion. It is recommended to consult your therapist in the month following the installation of this cushion to make sure it is efficient.

The Fusion® Cushion must be used with the cushion cover facing up. If the cover is not used correctly it may reduce or eliminate the cushion's benefits.

Life Span

The estimated life span of the Fusion® Cushion is three (3) years.

Storage

Physipro's cushions must be stored preferably on a flat surface, out of direct sunlight and in a dry area.

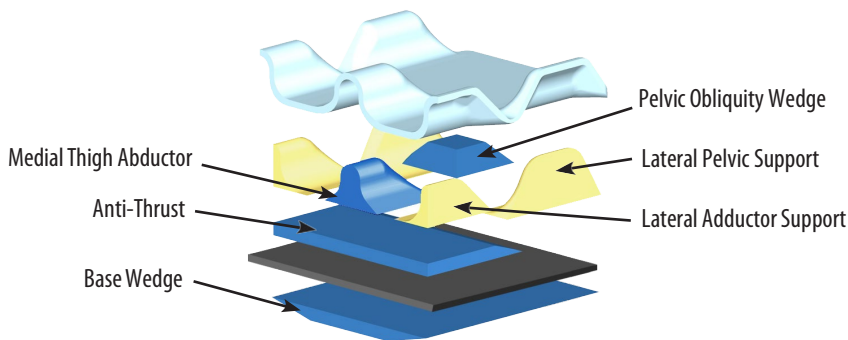
Terms and Conditions

The seat cushions and backrests are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment.

The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

Cushion composition

The Fusion cushion's layers of multi-density foam allow deeper immersion of the ischial tuberosities, enhanced envelopment, and excellent pressure redistribution. Positioning contours at the lateral hip and lateral thighs help promote a stable seated posture. The Physipro Fusion cushion is intended for wheelchair users who currently have or may be at risk of pressure sores and need positioning assistance for wheelchair seating.

Choice of postural components

Anti-thrust : Reduces sliding, controls pelvic rotation and posterior pelvic tilt.

Pelvic obliquity wedge : Decreases or compensates for pelvic obliquities.

Lateral pelvic support : Facilitates postural installation and controls pelvic rotation.

Medial thigh abductor : Controls lower limb adduction.

Lateral adductor support : Prevents thigh abduction.

Base wedge : For adding posterior/anterior or lateral wedging underneath the seat cushion.

Maximum user weight

14" to 20" width : 300 lb.

21" to 24" width : 500 lb.

Precautions regarding the materials

- Avoid long damp contacts with the foam.
- Never leave the cushion exposed to extreme temperatures during long periods of time.
- Let the cushion get to room temperature before using it.
- Sitting on too hot or too cold surfaces can cause cutaneous wounds.
- Inspect visually and regularly the state of the cushion.
- Avoid sharp objects or exposure to excessive heat or open flame.

ENGLISH

Maintenance & cleaning

Regular cleaning and maintenance may help extend the life of your cushion.

To clean cover:

- 1- Remove the cover from the foam base;
- 2- Machine wash in cold water.

No ironing required

Ensure the backs of the covers match up to the back of the foam base.

To clean incontinence cover: hand wash with soapy water.

To clean foam base:

- 1- Remove the cover and incontinence cover;
- 2- Wipe lightly with a damp cloth;
- 3- Wipe off with a clean cloth;
- 4- Let it completely air-dry before reassembly.

Do not use soap

Do not submerge in water

Ensure the back of the cover match up to the back of the foam base.

Disposal of Medical Devices

Do not burn and do not throw away the cushion and its protective cover outside of designated areas.

Warranty

The Fusion is a custom-designed cushion defined by the customer. Physipro can not guarantee the result and durability of the different combinations of foams and postural components that have been chosen. Please consult a healthcare professional to determine the most appropriate Fusion design for your needs.

The foam base is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase. This warranty covers original defects in material and workmanship of the cushion components, optional components and new parts that have been replaced by the supplier or by an authorized retailer, with effect from the delivery date or, if applicable, from the date of replacement. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover. The cover and incontinence covers are guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use.

Physipro inc. agrees to repair or replace any inoperative or defective part for the duration of the warranty period. To enjoy this warranty service, please contact Physipro inc. or an authorized retailer. Do not return this product without prior approval. If you are dissatisfied with this product, send us your comments at the address provided therein, and include the name and address of the supplier and the delivery date of the product. Furthermore, this warranty shall not apply to products that have been damaged through repairs or modifications made without the prior written consent of Physipro inc. or of an authorized retailer. The same restrictions shall apply for damages resulting from other circumstances beyond the control of Physipro inc.

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