LATCH ABDUCTOR MECHANISM S2795

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner’s manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION..... 3

COMPOSITION..... 3

RECOMMENDATIONS..... 3

WARNINGS 4

INSTALLATION 5

MAINTENANCE..... 8

WARRANTY..... 9

FRANÇAIS 10

DESCRIPTION

The latch abductor mechanism pivots under the seat with a simple push of its central lever to facilitate transfers. It is also possible to adjust the depth of the pommel.

This product has been assessed by the manufacturer and deemed to meet EU safety, health and environmental protection requirements and has obtained the CE marking.

COMPOSITION

Steel

Aluminium

RECOMMENDATIONS

To ensure the longevity of your lower limb support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the lower limb support's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required lower limb support and make any necessary adjustments.
- Clean regularly and inspect the lower limb support's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the lower limb support's surface.

It is recommended to consult your doctor or therapist in the month following the installation of a lower limb support to make sure it is efficient.

WARNINGS


- Physipro Inc. lower limb supports are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your lower limb support.
- Never push or lift wheelchair with a lower limb support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- New lower limb supports should be tested before prolonged use. To avoid redness due to pressure points, it is necessary to regularly check for redness in the various critical pressure areas, especially around bony prominences.
- Before use, let the lower limb support get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your lower limb support to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION

Required tools: 4 mm, 5 mm and 6 mm Allen keys


Installing the latch abductor mechanism to the seat plate

1. Center the slots of the abductor mechanism **A** with those of the seat plate **B**.
2. Assemble mechanism and seat plate with the provided screws **C**, washers **D** and nuts **E**, tighten firmly.
3. Position cushion on seat plate.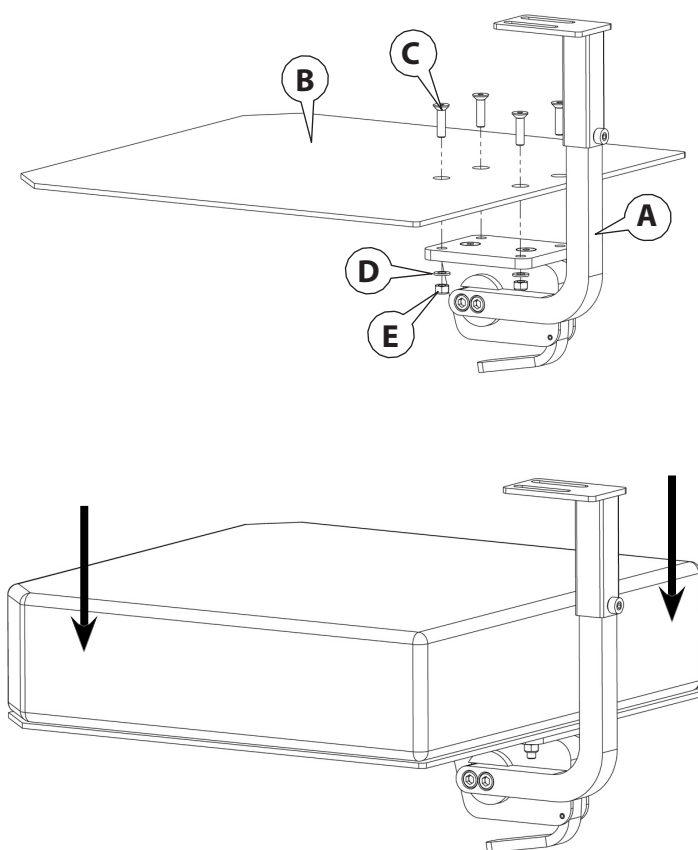


ENGLISH**Installing the abduction pommel**

1. Center the slots of the pommel **G** with those of the abductor mechanism **A**.
2. Install the abductor pommel **G** with the provided screws **H** and tighten firmly.

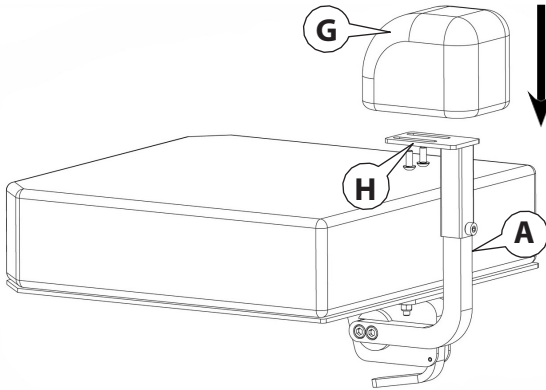
**Adjusting the depth of the abduction pommel**

1. Slightly loosen screws **H** to enable the abduction pommel **G** to be easily positioned.
2. Place the abduction pommel **G** at the desired depth and tighten screws **H** firmly.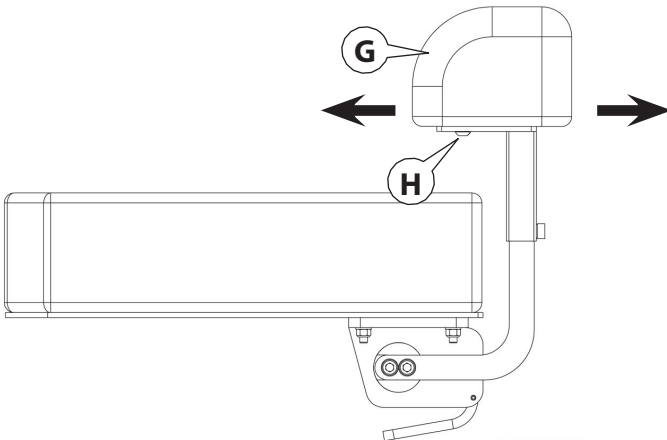


Adjusting the height of the abduction pommel

1. Slightly loosen screw **I** to enable the pommel **G** to be easily positioned.
2. Adjust the height of the pommel **G** and tighten screw **I** firmly.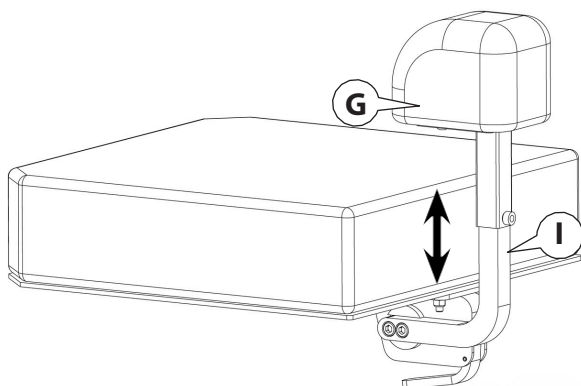

Pivoting the latch abductor mechanism under the seat

1. Press the central lever **J**, located under the seat, this will allow mechanism to pivot downwards.
2. Push mechanism down until a *click* is heard.

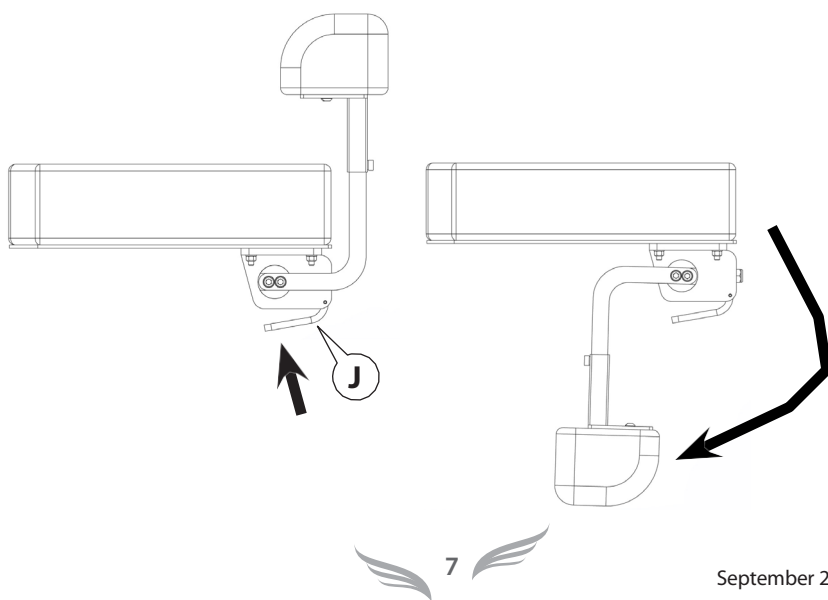ENGLISH**MAINTENANCE**

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your lower limb support. It is recommended to visually inspect all the components of the lower limb support during the cleaning process. A weekly cleaning is recommended.

Cleaning the abductor mechanism

1. Clean the abductor mechanism with a cloth slightly dampened with warm water and a mild detergent.
2. Wipe with a clean dry cloth.

**WARNING**

- Do not submerge mechanism in water.

Storage

Lower limb supports must be stored in a dry and secure area.

**WARNINGS**

- Never store your lower limb support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your lower limb support near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your lower limb support is two (2) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty period for the parts and components of the Latch Abductor Mechanism is **12 months**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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