2-Point padded hip belt

Owners manual

This manual contains important information about this product.
Please pass it to the final user upon delivery.

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ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our company by choosing one of our products. This user manual has been designed to allow the client to use the **2-point padded hip belt**, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your **2-point padded hip belt** and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the company.

For Physipro Inc., your satisfaction remains a priority.

Safety Rules

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user.

Physipro recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a cushioned belt is appropriate.

This product should be installed and fitted by a qualified rehab technician.

Ensure that the 2-point padded hip belt is worn tightly fitted at all times. A loose belt may cause the user to slip down and create risk of strangulation.

Accidental release of the 2-point padded hip belt can occur and cause the user to fall forward. If the user's movements or cognitive abilities could lead to accidental release, a healthcare professional should be present during the utilisation of these products.

The healthcare professional should regularly check for skin redness especially around bony prominences. If the user's skin develops redness, discontinue the use of this product immediately and see a doctor or therapist.

Ensure that the healthcare professional knows how to fasten and unfasten. Failure to do so may delay release in an emergency.

This product is NOT intended for transit transportation or as a personal restraint device. The 2-point padded hip belt should only have positioning purpose in the wheelchair. Misuse of this product is unauthorized and may occur in damage or injury.


2-point padded hip belt adult and pediatric sizes SC7540 and SC7545 (plastic buckle) SC7541 and SC7546 (metal buckle)

This 2-point padded hip belt allows pelvic alignment.


As it is fitted with Neoprene panels, it distributes pressure while protecting bony parts.

2-point padded hip belt installation principle

At each end, the belt is fitted with anchor plates. These can be installed screwed on a hard plate or simply drilled on tubes. (drill a 6 mm M5 screw).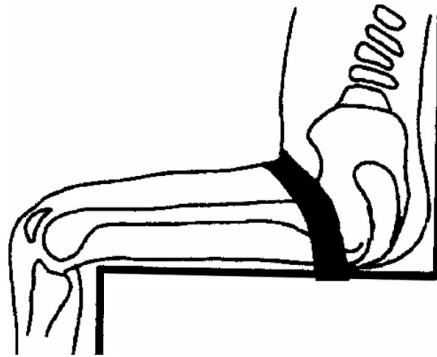

A pre-adjustment can be done with the belt clips and final adjustments are made with the velcro. Velcro is much appreciated when frequently changing more cumbersome clothes like thick outerwear.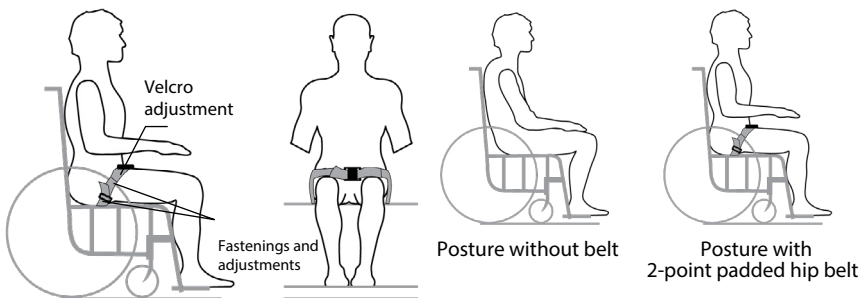


ENGLISH


Belt SC7696 clip

The belt clips allow firmly fitting strap ends without needing to drill the frame.

The clips fit on 1 inch (2.5 cm) wheelchair tubes.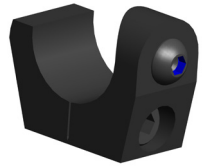

2-point padded hip belt selection guide

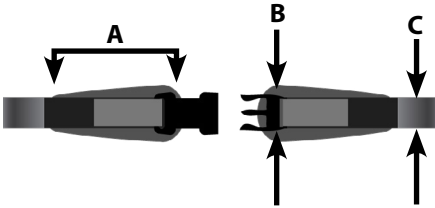	A	B	C	Maximal length
Child	7 in	3 in	1,5 in	29 in
Adult	9 in	3 in	2 in	50 in

Warranty

This product is covered with our twelve (12) month manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro Inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro Inc. or an authorized dealer to obtain your warranty services. Do not return your belt without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

Exclusions and Limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibility for damages incurred during shipping.

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