## Owners manual

This manual contains important informations about this product. Please hand to the user upon delivery.

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## Physipro Inc. Seating and mobility solutions

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This user manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

***For Physipro Inc., your satisfaction is our highest priority.***

### PRODUCT INFORMATION

**Date of purchase:** \_\_\_\_\_

**Supplier:** \_\_\_\_\_

**Address:** \_\_\_\_\_

**Telephone:** \_\_\_\_\_

## TABLE OF CONTENT

<b>Contour Insert™</b> .....	5
Product introduction .....	5
Cushion composition .....	6
Insert options available .....	7
Recommendations.....	8
Warnings .....	9
Installation .....	10
Maintenance .....	11
<b>Insert options</b> .....	12
Waffle Gel & Urethane.....	12
Pebble Gel & Urethane.....	13
Soft memory foam & Urethane .....	14
« Temper 36 & 38 » Memory foam .....	15
<b>ImmAirsiion™</b> .....	16
Product introduction .....	17
Components .....	18
Installation .....	19
Recommendations .....	20
Maintenance .....	21
Troubleshooting.....	23
<b>Warranty-Contour Insert™ and options</b> .....	25
<b>Français</b> .....	27

## CONTOUR INSERT™

The Contour Insert™ is composed of a firm base which features a recess in the pelvic loading area for the addition of the insert of your choice. The Contour Insert™ seat cushion can be purchased with different inserts and they can be used interchangeably to provide the pressure relief needed to continually ensure optimal comfort and skin protection.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

**ENGLISH****CUSHION COMPOSITION****Contour Insert™****Foam base**

Top layer: Urethane

Bottom layer: Plastazote

**Available Sizes**

Width: 14" - 24" (35 cm – 60 cm)

Depth: 14" - 24" (35 cm – 60 cm)

Thickness: 2", 3" or 4" (5 cm, 7,5 cm or 10 cm)

Foam base

**Insert(s) of your choice****Cover(s) of your choice**

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

**Physipro™ reserves the right to modify or change this manual without notice.**

## INSERT OPTIONS AVAILABLE

- **Waffle Gel & Urethane**
- **Pebble Gel & Urethane**
- **Soft Memory Foam & Urethane**
- **"Temper 36 & 38" Memory foams**
- **ImmAirSION™ Air Cell** (see page 16)

Waffle Gel & Urethane

Pebble Gel & Urethane

Soft Memory Foam & Urethane

"Temper 36 & 38" Memory foams

ImmAirSION™ Air Cell

**ENGLISH****RECOMMENDATIONS**

Before use, verify the quality and condition of the components of the cushion.

**To ensure your cushion provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.**

- Physipro™ seat cushions are reserved exclusively for use on mobility aids (manual or power wheelchairs), do not use cushion on other surfaces.
- Before installation, make sure the cushion is compatible with the wheelchair seat dimensions.
- Cushion must be installed on a clean and flat wheelchair surface of good quality and condition.
- Verify height of armrest. Adjustments should be made, if needed.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the cushion, this will diminish the cushion's efficiency and increase the risk of developing skin lesions.
- It is preferable to wear clothes made of cotton, that are not too tight and do not have seams on pressure points.
- Before use, let the cushion get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Never place cushion near sharp object.
- Remove any foreign object (crumbs, debris, tubing, etc.) from cushion surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.

**It is recommended to consult your therapist in the month following the installation of a new cushion to make sure it is efficient.**

## WARNINGS

**Consult a healthcare professional:** Physipro™ recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine and install the required seating system and make any necessary adjustments.

**Center of gravity:** Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new cushion, always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added or adjusted to increase stability.

**Cushion dimensions:** The dimensions of the cushion must be adapted to the user and the wheelchair, if not, the cushion's benefits will be reduced or eliminated.

**Cushion and cover compatibility:** The cushion cover must be well-fitted and used as directed in this manual to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the cushion and can create an unstable surface, which can lead to a fall.

**Environment:** Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the cushion to extreme cold or heat. Never place cushion near open flame.

**Instability/Fall risk:** Never place other products or materials (except those specified in this manual) between the user and the cushion or underneath the cushion, doing so, will cause instability and increase the risk of injury from a fall.

**Positioning of cushion:** Always refer to the orientation indicators of the cushion and the cover to properly position cushion on wheelchair. Proper positioning is essential to ensure maximum comfort and pressure relief.

**Skin inspection:** Cushion should be tested before prolonged use. Physipro™ recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a healthcare professional immediately. Be aware that no cushion can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

**Transport:** Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

## ENGLISH

### INSTALLATION

Your cushion is assembled according to your specifications before delivery.

Simply position the cushion on the wheelchair surface, make sure the insert is properly placed in the recess and that air cells are pointed upwards. The orientation indicator marked «Front» on the bottom of the outer cover must be placed towards the front and facing downwards. Make sure the stitches are aligned with the cushion's edges.

**Note:** Velcro fasteners are provided for installation on a flat wheelchair seat. Self-adhesive Velcro strips are attached to two fixed Velcro strips, located on the bottom side of the cushion's outer cover. To securely fix the cushion onto the rigid seat pan, remove the protective film of the self adhesive velcro, position cushion and then press down firmly on the cushion to ensure Velcro has properly adhered to the seat pan.

## MAINTENANCE & CARE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your cushion. It is recommended to visually inspect the components of the cushion during the cleaning process.

### Clean the outer cover:

1. Remove cover from the foam base and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam base.

*No ironing required.*

**Cleaning the incontinence cover:** Hand wash with soapy water and use a clean dry cloth to gently wipe off excess moisture, then let air dry completely.

### Cleaning the foam base:

1. Remove the outer cover and the incontinence cover and the insert.
2. Wipe lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on cushion to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s) on the foam base.

*Do not use soap.*

*Do not submerge in water.*

*Never place foam base in washer or dryer.*

### Storage

Physipro™ cushions should be stored on a flat surface, in a dry area and out of direct sunlight.

### Disposal of Medical Devices

Do not burn or throw away the cushion and its protective cover(s) outside of designated areas.

**ENGLISH**

## Insert Options

### WAFFLE GEL & URETHANE

#### COMPOSITION

Top layer: Waffle Gel

Bottom layer: Urethane

#### INSTALLATION

1. Install insert in the recess of the foam base, top layer (Waffle Gel) facing upwards.
2. Reinstall cover(s) on the foam base.

#### MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your insert.

1. Remove the insert from the foam base.
2. Clean the gel layer:
  - a. Wipe lightly with a damp cloth, with water and a mild detergent.
  - b. Use a clean dry cloth to gently wipe off excess moisture, then let air dry completely.
3. Clean the foam layer:
  - a. Wipe lightly with a damp cloth, without soap or detergent.
  - b. Use a clean dry cloth to gently press on foam to absorb excess moisture, then let air dry completely.
4. Reinstall the insert in foam base.

*Never place insert in washer or dryer.*

## Insert Options

### PEBBLE GEL & URETHANE

#### COMPOSITION

Top layer: Pebble Gel

Bottom layer: Urethane

#### INSTALLATION

1. Install insert in the recess of the foam base, top layer (Pebble Gel) facing upwards.
2. Reinstall cover(s) on the foam base.

#### MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your insert.

1. Remove the insert from the foam base.
2. Clean the gel layer:
  - a. Wipe lightly with a damp cloth, with water and a mild detergent.
  - b. Use a clean dry cloth to gently wipe off excess moisture, then let air dry completely.
3. Clean the foam layer:
  - a. Wipe lightly with a damp cloth, without soap or detergent.
  - b. Use a clean dry cloth to gently press on foam to absorb excess moisture, then let air dry completely.
4. Reinstall the insert in foam base.

*Never place insert in washer or dryer.*

**ENGLISH**

## Insert Options

### SOFT MEMORY FOAM & URETHANE

#### COMPOSITION

Top layer: Soft Memory Foam (blue)

Bottom layer: Urethane (yellow)

#### INSTALLATION

1. Install insert in the recess of the foam base, top layer (blue) facing upwards.
2. Reinstall cover(s) on the foam base.

#### MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your insert.

1. Remove the insert from the foam base.
2. Wipe lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on foam to absorb excess moisture, then let air dry completely.
4. Reinstall the insert in foam base.

*Never place insert in washer or dryer.*

## Insert Options

### "TEMPER 36 & 38" MEMORY FOAMS

#### COMPOSITION

Top layer: "Temper 36" Memory Foam (yellow)  
Bottom layer: "Temper 38" Memory Foam (pink)

#### INSTALLATION

1. Install insert in the recess of the foam base, top layer (yellow) facing upwards.
2. Reinstall cover(s) on the foam base.

#### MAINTENANCE & CLEANING

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your insert.

1. Remove the insert from the foam base.
2. Wipe lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on foam to absorb excess moisture, then let air dry completely.
4. Reinstall the insert in foam base.

*Never place insert in washer or dryer.*

## Owners manual

This manual contains important informations about this product. Please hand to the final user upon delivery.

## IMMAIRSION™

The ImmAirsion™ is an air cell insert with adjustable air pressure, designed for wheelchair cushions, that provides accurate pressure relief. With the use of pneumatic technology, the air cell insert allows a deeper immersion by adapting to the anatomy of the user and offers a higher level of skin and soft tissue protection, while also contributing to the healing process of existing skin lesions.

## ENGLISH

### COMPONENTS

The ImmAirson™ insert comes with the following components:

- Air cells insert with inflation valve(s)
- Hand pump with pressure gauge
- Repair kit
- User manual

#### Composition

Insert: Neoprene rubber

Inflation valves: Nickle plated brass

**Number of Air Cells (Width x Depth):** 7 x 5 cells, 8 x 5 cells and 9 x 6 cells

**Number of Inflation valves:** Single (left side only) or Dual (left and right side)

**Thickness:** 2", 3" or 4" (5 cm, 7,5 cm or 10 cm)

The product presented in this manual may not be an exact representation of your product. However, the instructions in this manual are still valid.

**Physipro<sup>MC</sup> reserves the right to modify or change this manual without notice.**

## RECOMMENDATIONS

The following recommendations are specific to the ImmAirion™ insert. However, it is important to read and comply with the **Recommendations** listed in the user manual of the accompanying cushion base to ensure safe and optimal use of the seating system.

**Air Pressure:** It is important to readjust air pressure at least once every 10 days or as needed. Daily verification of air pressure is recommended. Certain changes in the environment can modify air pressure inside the insert, such as change in altitude. It is important to verify and adjust air pressure after a change of altitude of 1,000 feet (300 meters) or greater.

**Air cell puncture:** To avoid puncture damage, never place insert near sharp objects.

**Environment:** If insert is exposed to temperatures lower than 32 °F (0 °C) and presents an unusual stiffness, allow the insert to warm to room temperature. Lightly rub the insert between your hands (with the valve(s) open) until the material is suppled. Readjust the air-pressure before use. The insert should be kept away from heat, direct sunlight and moisture. Avoid exposing the insert to extreme heat or open flame. Exposure to OZONE, OIL-BASED LOTIONS or LANOLIN may damage the neoprene rubber.

**Handling:** Never use the insert valve(s) as a handle. To move or carry your cushion, always lift the base or use the cushion's outer cover handle.

**Insert orientation:** The air cells of the insert must always be facing upwards.

**Under/Over-inflation:** Never use an under-inflated or over-inflated insert. An inadequately inflated insert will diminish or eliminate the performance of your insert and increase the risk of developing skin lesions.

**Skin inspection:** Insert should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro<sup>MC</sup> recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the insert. If discoloration does not disappear within thirty (30) minutes consult a Healthcare professional immediately. Be aware that no cushion or insert can substitute the need for proper skin hygiene, a balanced diet and frequent position changes

**It is recommended to consult your therapist in the month following the installation of the ImmAirion™ insert to make sure it is efficient.**

## ENGLISH

### INSTALLATION

#### Installing insert on foam base:

1. Install insert, with air cells facing upwards, in the recess of the foam base, while gently pushing the inflation tube(s) and valve(s) of the insert through the slot(s).
2. Reinstall cover(s) on the foam base and place the inflation valve(s) in the storage pocket(s).
3. Position the cushion on the wheelchair, make sure the orientation indicator marked "Front" on the bottom of the outer cover is placed towards the front and facing downwards. Make sure the stitches are aligned with the cushion's edges.

#### Inflating the ImmAirSION™ insert:

1. Before attempting to inflate the ImmAirSION™ insert, always verify that the safety valve of the hand pump is closed.
2. Gently remove the inflation valve from the storage pocket of the cover.  
*Note: with the dual valve model only one valve can be used to inflate insert, the other should remain closed.*
3. Unscrew the inflation valve cap of the insert.
4. Screw the metal adapter of the hand pump inflation tube onto the inflation valve of the insert.
5. Inflate the air cells with the hand pump until pressure gauge needle is situated in the green zone of the dial.
6. With the user well seated in his wheelchair, lift their leg slightly to enable you to place your hand between the user's bottom and the insert to detect their lowest bony prominence. Once determined lower their leg to a sitting position while keeping your hand in place.
7. Loosen the safety valve of the hand pump to release air pressure until you have difficulty moving your fingers. Close the safety valve.
8. Unscrew the metal adapter of the hand pump inflation tube and screw the inflation valve cap back on.
9. Place the inflation valve of the insert in the storage pocket of the insert cover.

**WARNING! The red zone of the pressure gauge dial indicates the insert is over-inflated, this may damage your insert.**

## MAINTENANCE

Cleaning, deodorization and disinfection are three separate processes. Cleaning should always precede deodorisation and disinfection.

### **Cleaning the ImmAirSION™ insert:**

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your insert. We recommend that you inspect your ImmAirSION™ insert during the cleaning process.

1. Remove the air cells insert from the foam base by carefully extracting the inflation tube(s) and valve(s).
2. Tightly close the inflation valve(s).
3. Wash by hand in warm water with a mild liquid soap or detergent. Use a sponge or a soft brush to thoroughly clean between each air cell.
4. Rinse with clean water and let air dry for 15 minutes. To accelerate the drying process, use a clean dry cloth to gently wipe off excess moisture.
5. Reinstall insert in the recess of the foam base and the cover(s).

*Never place insert in a washer or dryer.*

### **Deodorizing the ImmAirSION™ insert:**

1. Remove the air cells insert from the foam base by carefully extracting the inflation tube(s) and valve(s).
2. Tightly close the inflation valve(s).
3. To eliminate odor, prepare a solution of one (1) tablespoon of baking soda for one (1) liter of water.
4. Soak the air cell insert in solution for several hours or overnight.
5. Rinse with clean water and let air dry for 15 minutes. To accelerate the drying process, use a clean dry cloth to gently wipe off excess moisture.
6. Repeat procedure if necessary.
7. Place insert in the recess of the foam base and reinstall cover(s).

*Regular cleaning and the use of a waterproof cover can prevent odor formation and reduce the appearance of stains.*

**ENGLISH****Disinfecting the ImmAirSION™ insert:**

Disinfection is the process of destroying infectious agents and eliminating pathogenic microorganism. In most case regular cleaning is all that is required unless exposure to a known infection or an open wound has occurred. The insert must be cleaned before disinfection.

1. Remove the air cells insert from the foam base by carefully extracting the inflation tube(s) and valve(s).
2. Tightly close the inflation valve(s).
3. Prepare a solution of water and a neutral pH virucidal disinfectant cleaner, following the instructions on the disinfectant product label.
4. Soak the air cells insert in solution for 10 minutes.
5. Rinse with clean water and let air dry for 15 minutes. To accelerate the drying process, use a clean dry cloth to gently wipe off excess moisture.
6. Place insert in the recess of the foam base and reinstall cover(s).

**Disposal of Medical Devices**

Do not burn or throw away medical devices outside of designated areas.

## TROUBLESHOOTING

### Air Pressure loss

- ✓ Check the inflation tube(s) and valve(s) of the insert for damage.
- ✓ Inspect the insert to detect the presences of holes. If no holes are visible, immerse the inflated insert (with the valve(s) tightly closed) in water. The apparition of tiny air bubbles will allow you to locate the air leak.
- ✓ If a small hole is found, use the provided repair kit and follow the instructions written below (see **Repairing air leaks**).
- ✓ If a large hole is found, consult the “**Warranty**” section.

### Repairing air leaks

Included in the repair kit are the following items :

- 2 alcohol wipes
  - 2 patches
1. Once the location of the leak is determined, use the hand pump security valve to deflate the ImmAirsion<sup>MC</sup> insert:
    - a. Unscrew the inflation valve cap of the insert.  
*Note : with the dual valve model only one valve can be used to deflate insert, the other should remain closed.*
    - b. Screw the metal adapter of the hand pump inflation tube onto the inflation valve of the insert.
    - c. Unscrew the safety valve of the hand pump to release air.
    - d. The area around the hole should be clean and dry, use the provided alcohol wipe to cleanse the area, wait 10 seconds.
  2. Center and apply the adhesive side of the patch on the hole, press firmly and flatten out until the patch becomes transparent. Wait 10 minutes.
  3. Verify repair was effective by immersing the inflated insert (with the valve(s) tightly closed) in water. Make sure no air bubbles are visible.

## ENGLISH

### **Discomfort/Instability**

User must be seated at least one (1) hour to adapt to the insert, if discomfort persist.

Verify that :

- ✓ The insert is not under-inflated or over-inflated (see **Recommendations**).
- ✓ The air cells of the insert are facing upwards and the insert cover is properly installed.
- ✓ The size of the insert is well adapted to the user and the wheelchair.
- ✓ The insert is soft and pliable, exposure to cold temperature increases stiffness (see **Recommendations**).
- ✓ No foreign object (food, creased fabric, etc.) is placed between the user and the insert.

### **Trouble inflating the insert**

- ✓ Verify that the metal adapter of the pressure gauge is firmly screwed on the inflation valve of the insert.
- ✓ Ensure the safety valve of the hand pump is tightly closed.
- ✓ Check the inflation tube(s) and valve(s) of the insert for damage.
- ✓ Check the inflation tube and the inflation and safety valves of the hand pump for damage.

## WARRANTY-CONTOUR INSERT™ AND OPTIONS

The warranty period for the following parts and components of your cushion are as stated below:

- **Foam base:** Twenty-four (24) months
- **Covers:** Six (6) months
- **Waffle Gel & Urethane insert:** Twenty-four (24) months
- **Pebble Gel & Urethane insert:** Twenty-four (24) months
- **Soft memory foam & Urethane insert:** Twenty-four (24) months
- **“Temper 36 & 38” Memory foam insert:** Twenty-four (24) months
- **ImmAirion™ insert and hand pump with pressure gauge:** Twelve (12) months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period.

Physipro inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

### **This warranty does not apply to:**

- a. Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b. Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro inc. or of an authorized retailer.
- c. Normal wear and tear or damages caused by the user such as punctures, tears or burns.

## ENGLISH

### Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro<sup>MC</sup> product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro<sup>MC</sup> at the address listed below or send us an e-mail at **info@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro<sup>MC</sup>.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro inc.  
370 10<sup>th</sup> Avenue South, Sherbrooke, Quebec J1G 2R7 Canada

### Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro<sup>MC</sup> will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

**The foregoing warranty is exclusive and in lieu of all other expressed warranties.**

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Physipro Inc. s'engage à réparer ou remplacer les pièces défectueuses sans frais durant toute la période de garantie.