## Swing Away Half Tray

### Owners manual

This manual contains important information about this product.  
Please pass it to the final user upon delivery.

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
## ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the swing away half tray, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your swing away half tray, and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.


*For Physipro Inc., your satisfaction remains a priority.*

### 1 Swing away half tray installation


1. Loosen screws **A** to free the half tray's blocking bar.
2. Pull down the mechanism's blocking bar to have more space for the half tray installation.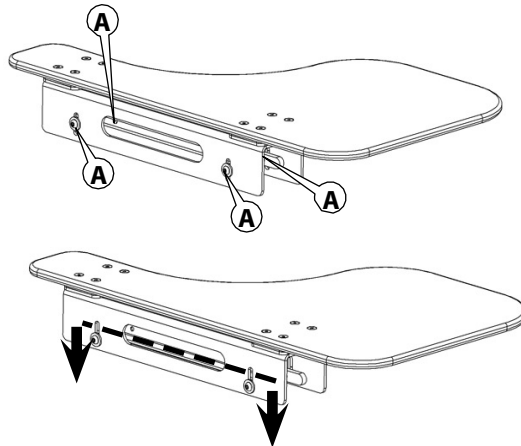

3. Insert the half tray **B** on the armrest by sliding it on the armrest.

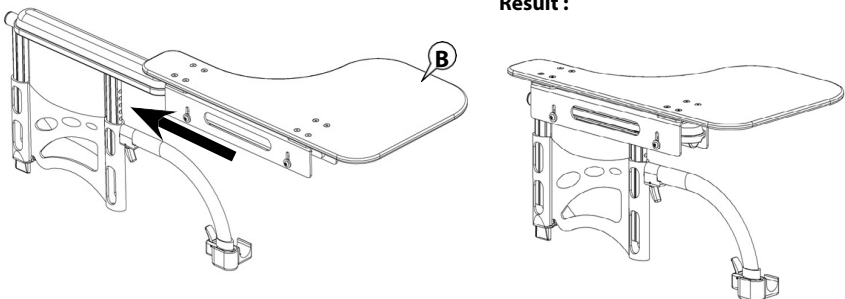
4. Raise the half tray mechanism's blocking bar so that the half tray firmly holds on the armrest cushion.
5. Firmly tighten screws **A**.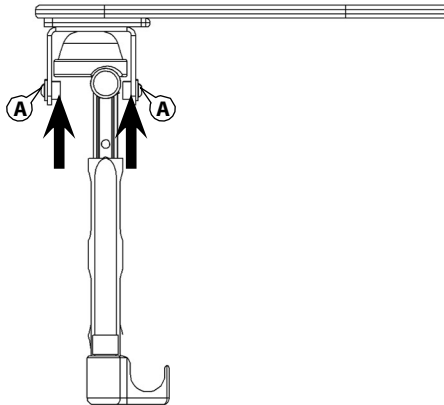

## 2. Folding the half tray

1. Lift the half tray **B** and fold it toward the outside.

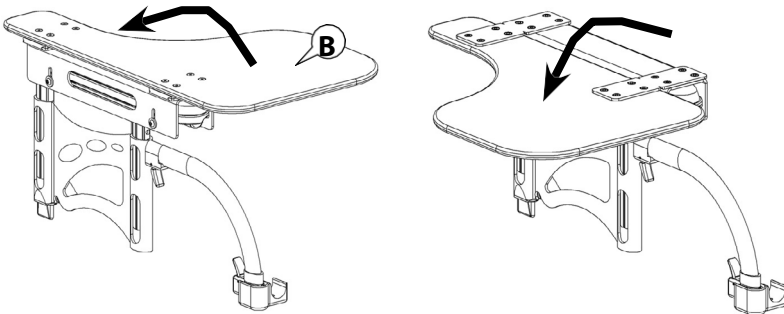2. Fold the half tray during transfer to free space when not in use or for storage.

**ENGLISH****Warranty**

This product is covered with our twelve (12) month manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro Inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro Inc. or an authorized dealer to obtain your warranty services. Do not return your tray without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

**Exclusions and Limitations**

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibility for damages incurred during shipping.

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