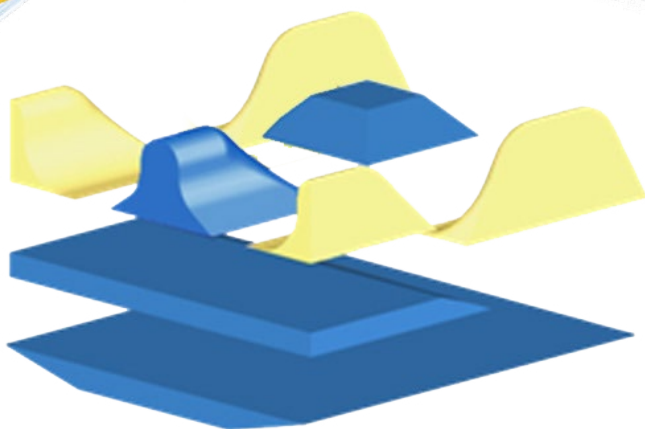
POSTURAL COMPONENTS FOR SEAT CUSHIONS

Owner's Manual

This booklet contains important information about this product. Please give to the owner upon delivery.

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ENGLISH**INTRODUCTION**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
RECOMMENDATIONS	3
ANTI-SLING WEDGES.....	3
POINTED ANTI-THRUST WEDGE.....	4
FLAT ANTI-THRUST WEDGE.....	4
PELVIC OBLIQUITY WEDGE	5
MEDIAL THIGH ABDUCTOR.....	6
LATERAL THIGH SUPPORT.....	7
REMOVABLE LATERAL PELVIC SUPPORT.....	8
MAINTENANCE.....	9
WARRANTY.....	10
FRANÇAIS	11

DESCRIPTION

Physipro inc. postural components can be added to a seat cushion to correct or improve sitting posture.

RECOMMENDATION

Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine and install the required seating system and make any necessary adjustments.

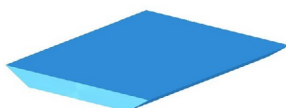
It is also recommended to consult your doctor or therapist in the month following the installation of a postural component to make sure it is efficient.

ANTI-SLING WEDGES

POINTED

FLAT


The anti-sling wedge is useful for limiting the hammock effect of a sling seat. By creating a flat surface, it helps re-establish pelvic balance. The anti-sling wedge can be glued under the cushion or simply inserted between the cushion and the sling seat. Two different shapes, pointed or flat, are available.

Composition: Constructa foam


Installation : Glued or fixed with velcro

Possibility of installation

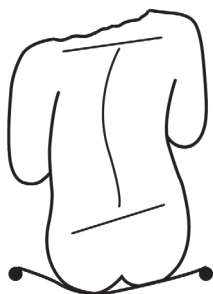
It is possible to order the flat anti-sling wedge with a cover in order to use it on a sling seat or a rigid seat.

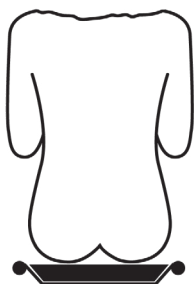Installation without the Anti-Sling Wedge


Installation with an Anti-Sling Wedge

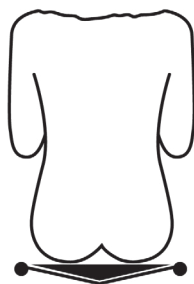Installation with a Pointed Anti-Sling Wedge

ENGLISH

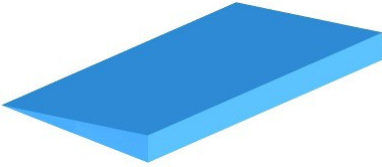POINTED ANTI-THRUST WEDGE


The pointed anti-thrust wedge reduces sliding, control pelvic rotation and posterior pelvic tilt. It is usually used with seat well cushions that procure a blocking effect. The wedge increases the pressure in front of the cushion and thus provides a femoral alignment.

Composition: Constructa foam

Installation : Glued or fixed with velcro

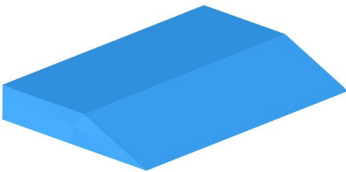
FLAT ANTI-THRUST WEDGE

This component is installed on the front part of the seat cushion. The flat anti- thrust wedge reduces sliding, controls certain patterns of spasticity, and helps distribute pressure over the entire seat.


Composition: Constructa foam

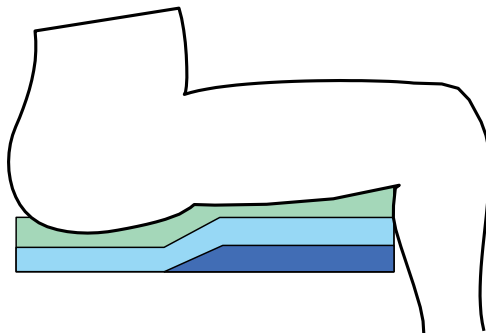Installation : Glued or fixed with velcro


PELVIC OBLIQUITY WEDGE

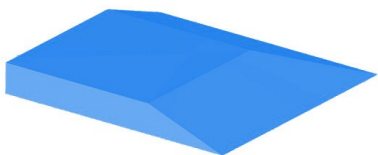CONSTRUCTA FOAM



URETHANE


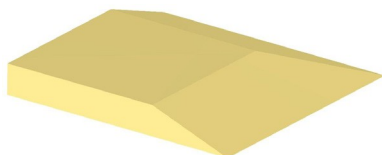
The pelvic obliquity wedge can compensate for loss of gluteal muscle tone or reduced pelvic tilt.

Composition: Constructa foam or urethane

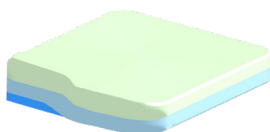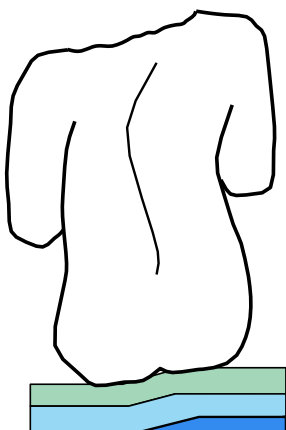
Installation : Glued or fixed with velcro

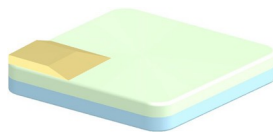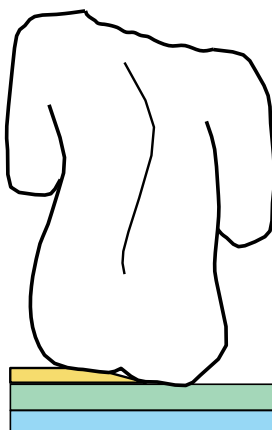Wedge under the cushion
(Constructa foam)



Wedge on top of the cushion
(Urethane)

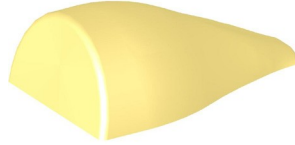ENGLISH**MEDIAL THIGH ABDUCTOR****CONSTRUCTA FOAM****URETHANE**

This medial thigh abductor is used for controlling lower-limb adduction. In some cases of posterior pelvic tilt, the medial thigh abductor will maintain a neutral pelvic alignment if the posture is not fixed.


Composition: Constructa foam or urethane

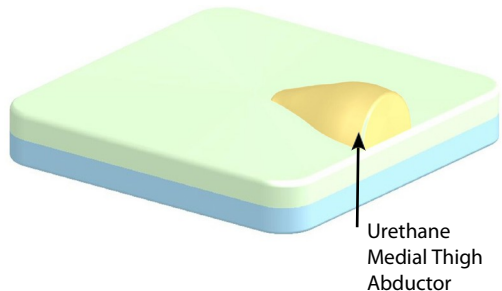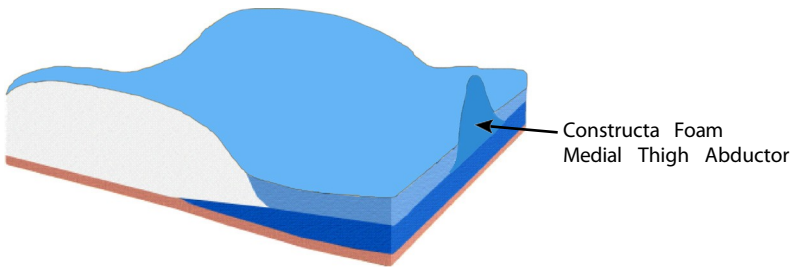Installation : Glued or fixed with velcro


LATERAL THIGH SUPPORT

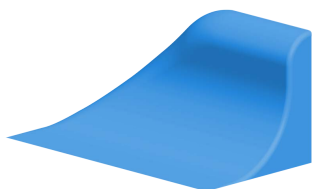CONSTRUCTA FOAM


URETHANE

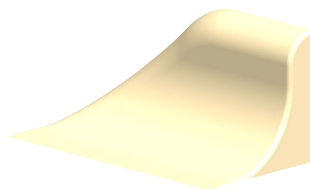The lateral thigh support ensures a better alignment of the lower limbs by limiting the risk of sliding towards the outside of the cushion.


Composition: Constructa foam or urethane

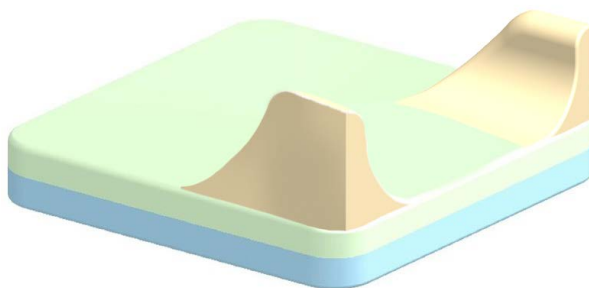Installation : Glued or fixed with velcro

ENGLISH

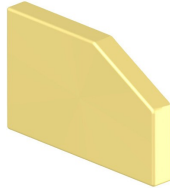REMOVABLE LATERAL PELVIC SUPPORT


The lateral pelvic support limits the abduction of the lower limbs while stabilizing the pelvis.

Standard composition: Firm urethane


Installation : Three installation methods are offered.

- Fixed with velcro to a clothes-guard
- Fixed with adjustable brackets
- Fixed with an adjustable bar


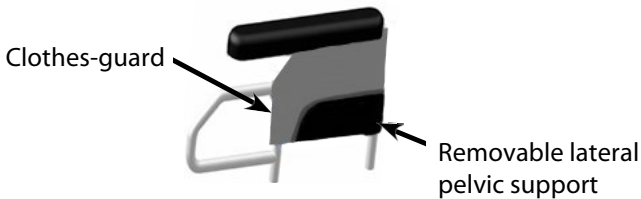
Fixed with velcro to a clothes-guard

Fixed with adjustable brackets

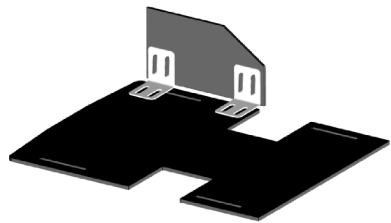Fixed with an adjustable bar

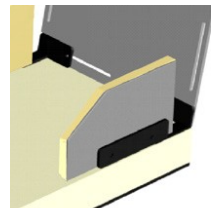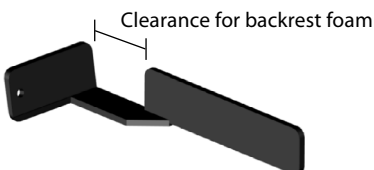MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your postural component. A visual inspection is recommended during the cleaning process. A weekly cleaning is recommended.

Cleaning the cover

1. Unfasten zippers and remove the cover.
2. Hand wash with a mild detergent and rinse.
3. Let air dry completely, then reinstall.

WARNING

- Never machine wash or dry.

Cleaning the padding

1. Wipe padding lightly with a damp cloth, without soap or detergent.
2. Use a clean dry cloth to gently press on padding to absorb excess moisture, then let air dry completely.

WARNING

- Do not submerge padding in water.

Storage

Physipro Inc. postural components for seat cushions must be stored in a dry and secure area.

Life span

The estimated life span of a postural component is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH

WARRANTY

The warranty period for a seat cushion postural component is 12 months.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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