Owners manual

This manual contains important informations about this product. Please hand to the user upon delivery.

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INTRODUCTION - ENGLISH

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This user manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

HP2™ BACKREST

The HP2™ backrest line was designed with high performance and precision in mind. Characterised by a rigid shell that encompasses the user's back to provide overall lateral and central support, this backrest was created for persons with significant postural asymmetries.

Three models are available:

A **contour model** designed to provide overall back support.

A **profiled model** designed for active users, which permits upper body freedom of movement and provides greater ease of propulsion.

A **sport model** designed for rigid structure chairs. It provides enhanced lumbar support and greater freedom of movement.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

SPECIFICATIONS

Hard Shell

Aluminium

Hardware

Two-point anchoring system with quick-release

Foam Padding

¼" Plastazote, ½" Soft memory foam and 1" Viscose

Available Covers

Standard cover with cutout for headrest support


Cover with cutouts for headrest support and harness

Technical specifications	
Nominal widths*	12", 14", 16", 18" or 20" (30, 35, 40, 45 or 50 cm)
Nominal heights**	12", 14", 16", 18" or 20" (30, 35, 40, 45 or 50 cm)
Thickness	1 ½" (3.8 cm)
Weight	7,4 lb (16" x 18" - Without lateral supports) 8,8 lb (16" x 18" - With lateral supports)
Lateral support widths	Adjustable and Foldable and adjustable: 4", 4 ½" or 5 ½" (10, 11.5 or 14 cm) Contour : 6" or 7" (15 or 18 cm)
Lateral support heights	Adjustable and Foldable and adjustable: 4", 6", 7", 8" or 10" (10, 15, 18, 20 or 25 cm) Contour : 7", 8" or 9" (18, 20 or 23 cm)

*The actual width of the HP2 back plate is one (1) inch smaller (2.5 cm) than the nominal value.

**The back height may be customized.

Each HP2 back size can be installed on different wheelchair widths. To choose correctly the size of your back, it is important to consider the widths of the wheelchair and of the user's thorax. See the dimensions tables below: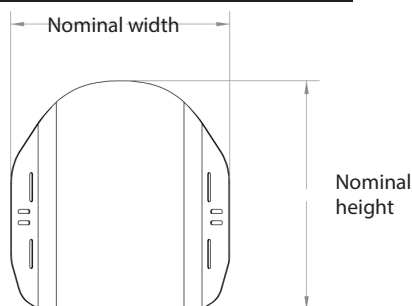

Dimensions of the HP2 back, Profiled and Contour models

Actual Plate Width	Nominal Width	Nominal Height Available for the Back	Thorax Width HP2 back *	Associated Wheelchair Width	Associated Wheelchair Width With Extension Option
11" (28 cm)	12" (30 cm)	12", 14" (30, 35 cm)	7" to 10" (18 to 25 cm)	12" to 13" (30 to 32.5 cm)	14" (35.5 cm)
13" (33 cm)	14" (35 cm)	14", 16" (35, 40 cm)	9" to 11 ½" (23 to 29 cm)	14" to 15" (35 to 37.5 cm)	16" (40.5 cm)
15" (38 cm)	16" (40 cm)	16", 18" and 20" (40, 45, 50 cm)	11" to 13 ½" (28 to 34.5 cm)	16" to 17" (40 to 42.5 cm)	18" (46 cm)
17" (43 cm)	18" (45 cm)	16", 18" and 20" (40, 45, 50 cm)	13" to 15 ½" (33 to 39 cm)	18" to 19" (45 to 47.5 cm)	20" (51 cm)
19" (48 cm)	20" (50 cm)	16", 18" and 20" (40, 45, 50 cm)	15" to 17 ½" (38 to 44.5 cm)	20" to 21" (50 to 52.5 cm)	22" (56 cm)

*With Pediatric Thoracic Supports : add 1" (2.5 cm) to the thorax width.

Sizes associated to the HP2 back Sport model

Actual Plate Width	Nominal Width	Nominal Height	Associated Wheelchair Width	Associated Wheelchair Width With Extension Option
11" (27.5 cm)	12" (30 cm)	9" (23 cm)	12 to 13" (30 to 32.5 cm)	14" (35.5 cm)
13" (33 cm)	14" (35 cm)	9" (23 cm)	14" to 15" (35 to 38 cm)	16" (40.5 cm)
15" (38 cm)	16" (40 cm)	9" (23 cm)	16" to 17" (40 to 43 cm)	18" (46 cm)
17" (43 cm)	18" (45 cm)	9" (23 cm)	18" to 19" (45 to 48 cm)	20" (51 cm)

RECOMMENDATIONS

To ensure your backrest provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the backrest components, please refer to Maintenance guide in section Maintenance, page 19.
- Physipro Inc. backrest are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always install backrest with the provided mounting hardware.
- Never push or lift wheelchair with backrest, doing so may cause the backrest to detach from the wheelchair and can result in injury or damage.
- Before installation, make sure the backrest is compatible with the wheelchair's dimensions.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the backrest, this will diminish the backrest efficiency and increase the risk of developing skin lesions.
- It is preferable to wear clothes made of cotton, that are not too tight and do not have seams on pressure points.
- Before use, let the backrest get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Remove any foreign object (crumbs, debris, tubing, etc.) from backrest surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.

It is recommended to consult your therapist in the month following the installation of a new cushion to make sure it is efficient.

WARNINGS

Consult a healthcare professional: Physipro Inc. recommends that a healthcare professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required seating system and make any necessary adjustments.

Center of gravity: Installing a backrest on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new backrest, always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added or adjusted to increase stability.

Backrest dimensions: The dimensions of the backrest must be adapted to the user and the wheelchair, if not, the backrest's benefits will be reduced or eliminated.

Backrest and cover compatibility: The backrest cover must be well-fitted and used as directed in this manual to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the backrest and can reduce stability.

Environment: Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the backrest to extreme cold or heat. Never place backrest near open flame.

Instability/Fall risk: Never place other products or materials (except those specified in this manual) between the user and the backrest or underneath the backrest, doing so, will cause instability and increase the risk of injury from a fall.

Positioning of backrest: Always refer to the orientation indicators, located on the backrest and the cover, to properly position backrest on wheelchair. Proper positioning is essential to ensure maximum comfort and pressure relief.

Skin inspection: Backrest should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro Inc. recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a Healthcare professional immediately. Be aware that no backrest can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

Transport: Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION


Please read all the following instructions before beginning installation. The steps required to properly install the HP2 backrest must be followed in the order that they are listed.

Required tool: 4 mm Allen Key

1 - Installing the mounting brackets on the backrest

When you receive your backrest, this step will have already been done by one of our technicians. The procedure explains the steps that must be taken if the backrest has been completely removed.



1. Insert bolt **A** into the vertical adjustment slot of the backrest with washer **B** inserted in between the bolt and the backrest.
2. Screw nut **C** onto bolt **A**, without tightening.
3. Repeat steps 1 and 2 for the opposite side.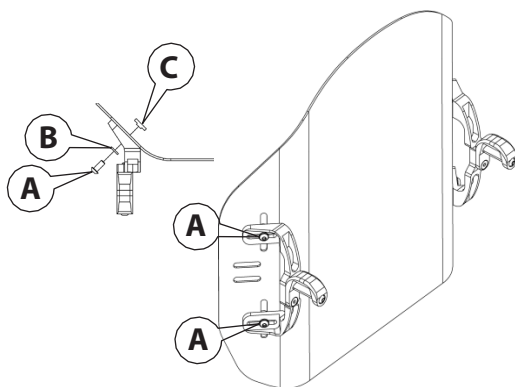

Note: At this stage of assembly, the mounting bracket is not tightly fastened to the backrest. It can move upwards and downwards, as well as sideways. It will be securely fastened in a subsequent step.


2 - Installing the mounting clamps on the backpost

1. Loosen screws **A** with the 4 mm Allen key.
2. Open the mounting clamp.
3. Place the mounting clamp on the backpost.
4. Position the mounting clamp.
5. Once the mounting clamp is correctly positioned, fasten screws **A**. (Do not fasten screws tightly at this stage, they will be fastened tightly in a subsequent step.)
6. Repeat the process for the second mounting clamp.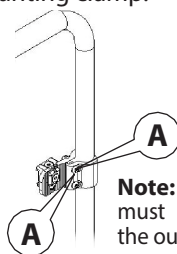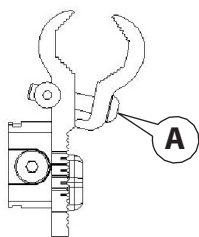


Note: The head of screws **A** must always point towards the outside of the wheelchair.

Note: Make sure the mounting clamps are positioned at the same height on each backpost and that they are placed parallel to the seat frame.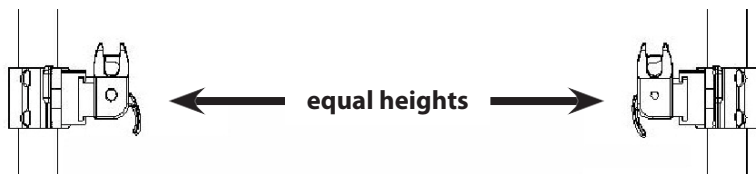

3 - Assembling the mounting hardware

1. Loosen screw **B** so that part **C** slides easily into the part **D**.
2. Insert part **C** into part **D**.
3. Tighten screw **B** firmly.
4. Repeat the process for the opposite side.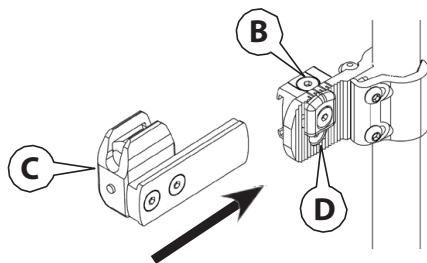


ENGLISH


4 - Inserting the HP2 backrest

The HP2 backrest is quick and easy to install with its two-point anchoring system.


1. Press latch **E** to open the anchor mechanism on both sides simultaneously.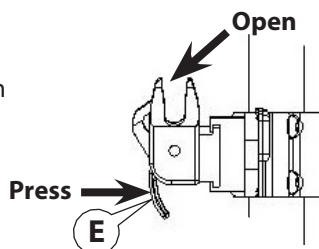

2. Slide the backrest downwards until the mounting brackets **F** are inserted into the anchor mechanisms **G**.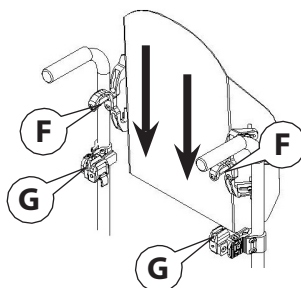

3. Pull on both latches **E** until you hear a "click". This means the backrest is locked in place.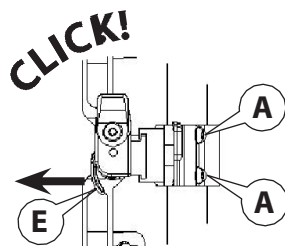


4. Tighten screws **A** firmly.

Note: After backrest installation, make sure the backrest is correctly fastened to the wheelchair. The latches must have made an audible "click" to confirm the backrest is locked in place. Make sure backrest is securely attached by pulling the backrest upwards.

5 - Removing the HP2 backrest

1. Press latch **E** to open the anchor mechanism on both sides simultaneously.
2. Pull the backrest upwards to remove.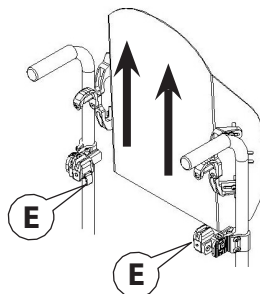


EXTENSION PLATE (OPTION)

The extension plate allows a smaller backrest to be mounted to a larger wheelchair frame. The extension plate can be added to the original mounting hardware.

Required tools: 4 mm Allen key and a 10 mm wrench.

1. Detach **Part A** from **Part B** of the mounting bracket by removing bolts **C** and **D**.
2. Insert the extension plate between **Part A** and **Part B** as illustrated.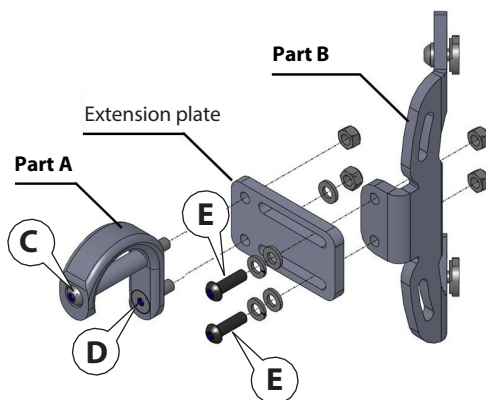

3. Fasten bolts **C**, **D** and **E** loosely with washer and nuts, as illustrated.
4. Fasten the backrest to the mounting hardware.
(Refer to section: Installation, 4 - Attaching the HP2 backrest)
5. Once all adjustments are made, tighten bolts **C**, **D** and **E**.

ADJUSTMENTS


Required tool: 4 mm Allen Key

Height Adjustment

1 - Adjusting the height of the mounting clamps

1. Loosen screws **A** and adjust the height of the back shell to required position.
2. Securely tighten screws **A**.
3. Repeat the process for the second mounting clamp.


Note: After height adjustment is complete, make sure the mounting clamps are positioned at the same height on each backpost and that the width was not inadvertently changed.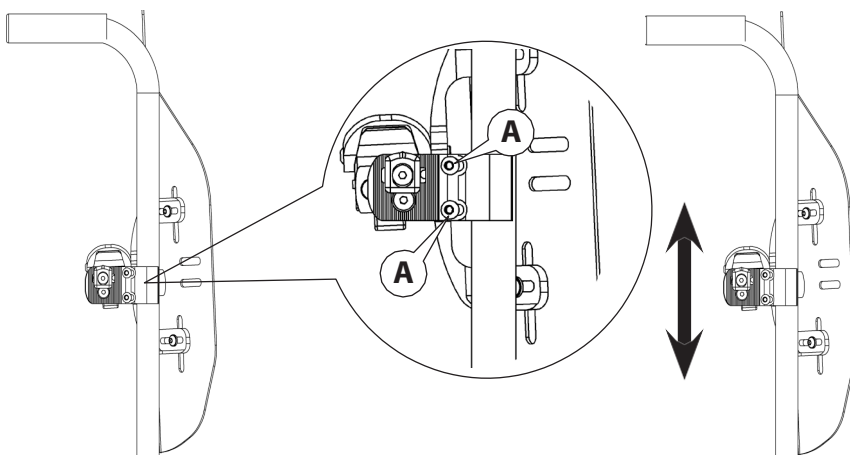

2 - Adjusting the height of the mounting brackets

1. Slide the mounting brackets upwards or downwards in the backrest's vertical adjustment slots until the correct height has been obtained.

Note: Make sure both sides of the backrest are positioned at the same height and that the backrest does not touch the backposts (If it does, slightly move the backrest forward by sliding the backrest in the mounting bracket's horizontal adjustment slots).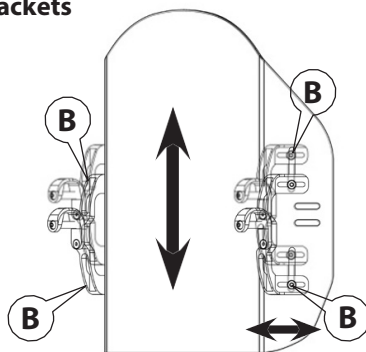

2. Securely tighten all 4 screws **B**.

Depth Adjustment

Depending on the width of the backrest, the width of the chair and the different ways of positioning the mounting hardware, you can obtain a depth adapted to the customer's needs.


Depth Chart

* 12" and 20" are unavailable for the sport version. These dimensions may vary of $\pm 1/16$ in (0.16 cm). The measures include foam padding. The depth measure is taken from the front of the backposts.

** OPTION, Extension plate.


		HP2 backrest width				
		12*	14	16	18	20*
External width of the wheelchair	12	-1" to +2 1/2"				
	13	-1 1/2" to +2"				
	14	** -2" to +1 1/2"	-1" to +2 1/2"			
	15		-1 1/2" to +2"			
	16		** -2" to +1 1/2"	-1" to +2 1/2"		
	17			-1 1/2" to +2"		
	18			** -2" to +1 1/2"	-1" to +2 1/2"	
	19				-1 1/2" to +2"	
	20				** -2" to +1 1/2"	-1" to +2 1/2"
	21					-1 1/2" to +2"

Technical diagram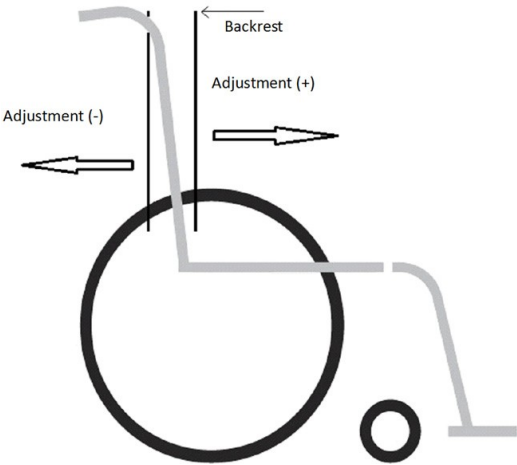

Adjusting backrest depth

1. Loosen screw **E** of the mounting hardware on both sides of the backrest.
2. Slide backrest forwards or backwards until the desired depth has been reached.
3. Securely tighten screws **E**.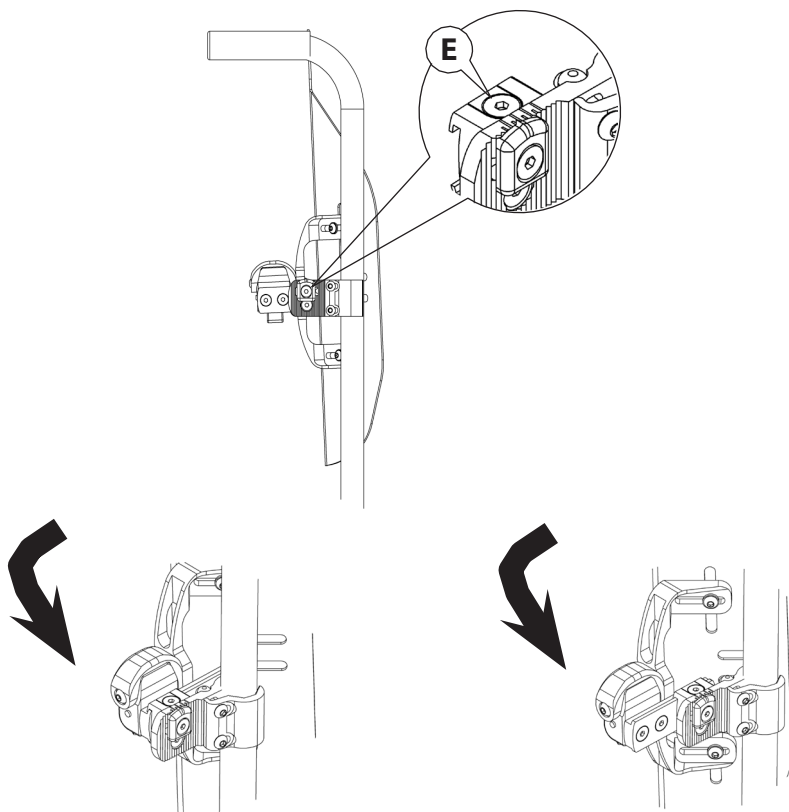


Note: If the depth of the HP2 backrest is not equal to zero, then the seat depth will be modified. If the depth of the backrest is negative, the seat will seem too far back, and if it is positive, it will seem too far forward.

Note: A security mechanism is installed on the hardware of the anchoring system to prevent the backrest from detaching, even in the event of a violent spasm.


Tilt Adjustment

Obtaining a positive or negative tilt angle is easy with the HP2 backrest.


1. Loosen screws **G** and **H** of the mounting hardware on both sides of the backrest.
2. Tilt the backrest to the desired angle.
3. Securely tighten screws **G** and **H**.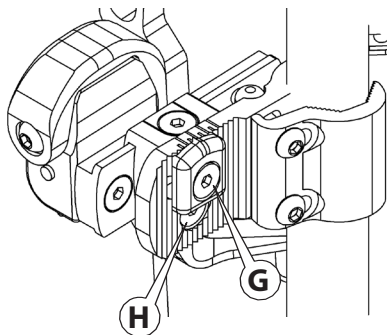

Note: It is important to note that, in most cases, the depth of the seat will be modified after adjusting the angle of the backrest. The depth will increase or decrease depending on the angle.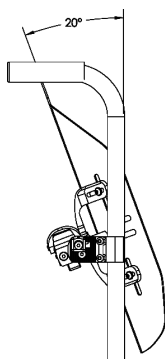

A 20° inclination represents a 110° seat-to-back angle.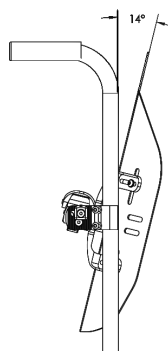

The maximum front tilt is -14° represents a 76° seat-to-back angle.

LATERAL SUPPORTS (OPTION)

Lateral supports can be installed on the HP2 Contour and Profiled models only.

Three models of lateral supports are available:

- Adjustable,
- Foldable and adjustable
- Contour

Please refer to the section pertaining to your lateral support model.

Adjustable & Foldable and adjustable

Installing the lateral support brackets on the backrest

Note: The installation procedure for both the Adjustable and the Foldable and adjustable lateral support brackets is the same.

1. Position the lateral support bracket on the front of the backrest.
2. Insert the bolts **A** with washers **B** through the backrest and lateral support as illustrated.




Note: The head of bolts **A** must be towards the back of the backrest.

3. Fasten nuts **C** to bolts **A**.
Note: Do not tighten the bolts right away, you need to be able to adjust the angle and the distance between the lateral supports.

4. Pivot the lateral support bracket until the desired angle is obtained.
5. Repeat steps 1 to 4 for the opposite lateral support.

6. Slide both lateral supports inwards or outwards until the distance between both lateral supports correctly to accommodate the individual's back.


7. Firmly tighten bolts **A** when the lateral support bracket adjustments are completed.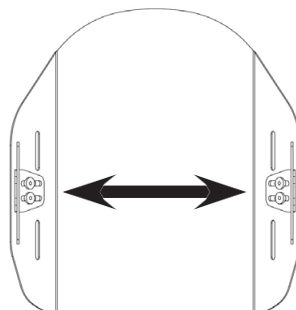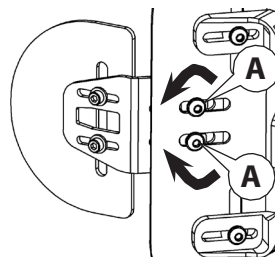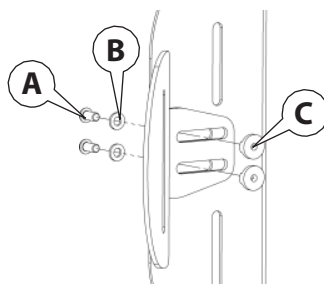

Adjusting the height and depth of the lateral support plates

Note: The height and depth adjustment procedure for both the Adjustable and the Foldable and adjustable lateral support brackets is the same.

1. Loosen bolts **B**.
2. Slide the lateral support bracket upwards or downwards in the vertical adjustment slot of the lateral support plate until the desired height has been obtained.
3. Slide the lateral support plate in the brackets horizontal adjustment slots until the desired depth is obtained.
4. Firmly tighten bolts **B**.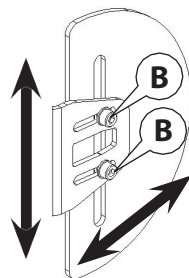

Installing the lateral supports covers



1. Completely unfasten the cover's zipper.
2. Insert cover, starting at the top of the lateral support.
3. With a downward movement, push cover around the lateral support.
4. Fasten the zipper.

Flipping away the Foldable and adjustable lateral supports

The foldable and adjustable lateral supports facilitate transfers by enabling the lateral supports to be moved aside with a pivoting mechanism.

1. Push the pivoting mechanism **C** upwards.
2. Move the lateral support plate towards the outside of the wheelchair.
3. To reposition the lateral support plate, push the pivoting mechanism **C** upwards.
4. Move the lateral support plate towards the inside of the wheelchair.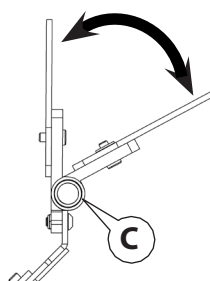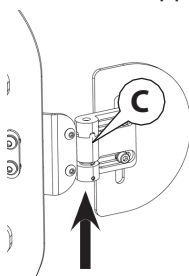

ENGLISH

Contour

Installing the lateral support brackets on the backrest




1. Position the lateral support on the front of the backrest.
2. Insert the bolts **A** with washers **B** through the backrest and lateral support as illustrated.

Note: The head of bolts **A** must be towards the back of the backrest.

3. Fasten nuts **C** to bolts **A**.

Note: Do not tighten the bolts right away, you need to be able to adjust the height and the distance between the lateral supports.

4. Slide the lateral support upwards or downwards in the backrest's vertical adjustment slot until the desired height has been obtained.
5. Repeat steps 1 to 4 for the opposite lateral support.
6. Slide both lateral supports inwards or outwards until the distance between both lateral supports correctly to accommodate the individual's back.
7. Firmly tighten bolts **A** when the lateral support adjustments are completed.

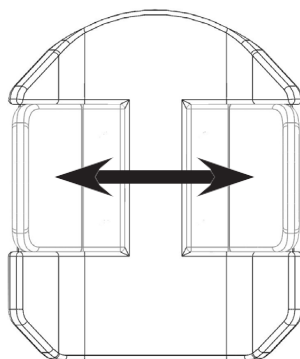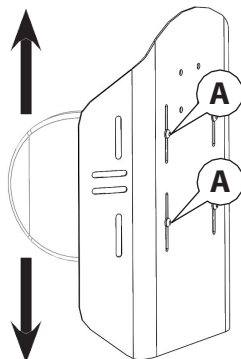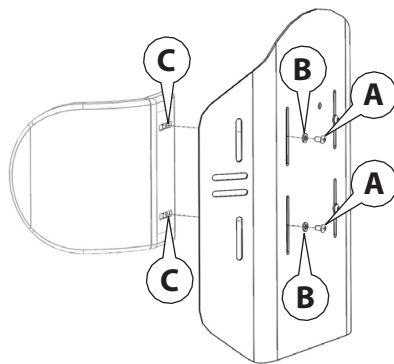Width Between the Laterals	
Backrest	Width
16"	12" to 13"
18"	14" to 15"
20"	16" to 17"

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your backrest. It is recommended to visually inspect the backrest during the cleaning process.

Maintenance Guide

Upon receiving your backrest, it is important to complete all the inspections listed in the Maintenance Guide to ensure the proper function and condition of the backrest and mounting hardware.

Checklist		Weekly	Montly
Backrest	- Inspect aluminum components for rust or damage. - Make sure the backrest is not deformed.		X X
Hardware	- Verify the solidity and condition of mounting plates and clamps. - Ensure latch lock mechanism operates properly.	X X	
Laterals	- Make sure that the laterals are fastened properly. - Inspect aluminum components for rust or damage.	X	 X

Cleaning the outer cover:

1. Remove cover from backrest and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam base.

WARNING

Do not iron.

ENGLISH

Cleaning the hard shell: The aluminum hard shell can be cleaned with warm water and a mild liquid detergent. No other maintenance is required.

Cleaning the backrest padding:

1. Remove outer cover and incontinence cover, if necessary.
2. Wipe foam padding lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on padding to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s) on the backrest.

WARNINGS

*Do not use soap.
Do not submerge in water.
Do not use washer or dryer to clean
the backrest padding.*

Storage

Backrest must be stored in a dry and secure area.

WARNINGS

*Never store backrest in direct sunlight or expose to excessive humidity or heat.
Never place backrest near sharp objects or in an area where it may be struck.*

Life span

The estimated life span of your backrest is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty periods for the following parts and components of your HP2 backrest are as stated:

Backrest: Twenty-four (24) months

Lateral supports : Twelve (12) months

Outer cover: Six (6) months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period.

Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a. Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b. Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c. Normal wear and tear or damages caused by the user such as punctures, tears or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon as possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date and the product's serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.
370 10th Avenue South, Sherbrooke, Quebec J1G 2R7 Canada

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Inc.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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