Chest Support Men, Women and Children

Owner's manual

This manual contains important information about this product.
Please pass it to the final user upon delivery.

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ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our company by choosing one of our products. This user manual has been designed to allow the client to use the **chest support**, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your **chest support** and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the company.

For Physipro Inc., your satisfaction remains a priority.

Safety Rules

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user.

Physipro recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a chest support is appropriate.

This product should be installed and fitted by a qualified rehab technician.

Ensure that the chest support is tightly fitted at all times. A loose belt may cause the user to slip down and create risk of strangulation.

Accidental release of the chest support can occur and cause the user to fall forward. If the user's movements or cognitive abilities could lead to accidental release, a healthcare professional should be present during the utilisation of these products.

The healthcare professional should regularly check for skin redness especially around bony prominences. If the user's skin develops redness, discontinue the use of this product immediately and see a doctor or therapist.

Ensure that the healthcare professional knows how to fasten and unfasten. Failure to do so may delay release in an emergency.

This product is NOT intended for transit transportation or as a personal restraint device. The chest support should only have positioning purpose in the wheelchair. Misuse of this product is unauthorized and may occur in damage or injury.

Chest Support


Men, Women and Children (4 or 5-point model)

These harnesses made of neoprene hold the trunk while allowing some freedom of movement.

Each side has a plastic buckle allowing for quick opening and closing during transfers. The 5-point child chest support is equipped with an additional crotch buckle to reduce the risk of slipping.

Chest support installation

- The chest support is equipped with 2 friction buckles on the upper straps. During installation, the buckles will be drilled and secured on the backrest rigid plate (screws and sleeved nuts). In the case that the backrest is too low, using raised anchor plates would be necessary.
- In the case that there is no rigid plate, buckles can be installed on an anchor bar fixed on the wheelchair handles.
- On its lower part, the harness is equipped with 2 anchor plates. So drilling in the backrest plate or in the wheelchair frame will be easy if it is necessary for installation.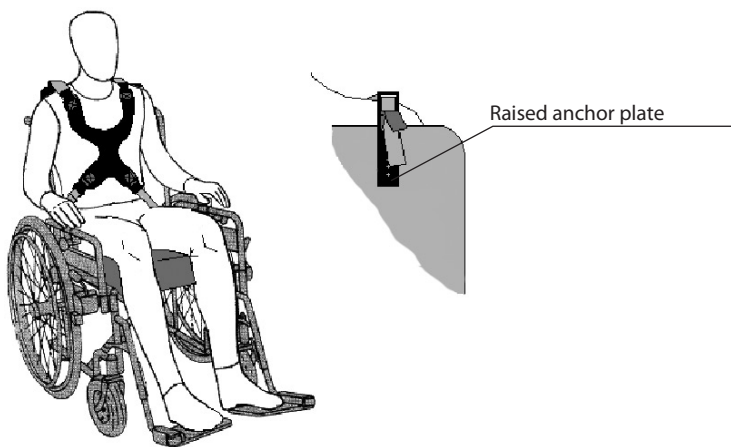


ENGLISH


Belt SC7696 clip

The belt clips enable firmly fitting strap ends without needing to pierce the frame.

The clips fit on 1 inch wheelchair tubes.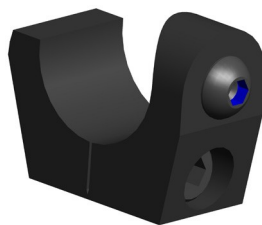


Installation of the chest support on a soft backrest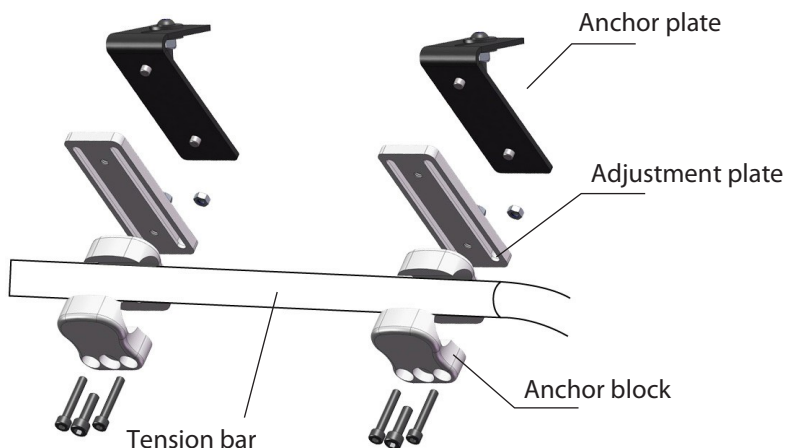

The anchor blocks can be fixed on a tension bar (block available in $\varnothing 25\text{mm}$ + reduction for $\varnothing 22\text{mm}$ and $\varnothing 19\text{mm}$).


The adjustment plate combined with an anchor plate allows for height adjustment.

Chest Support - Men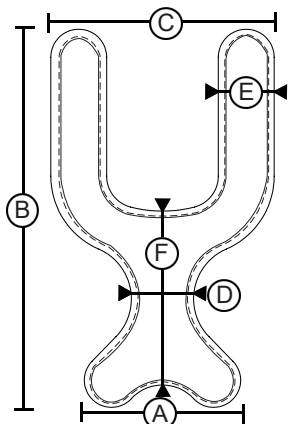


Size	Chest Support - Men					
	A	B	C	D	E	F
Small	156 mm	376 mm	222 mm	62 mm	57 mm	156 mm
SC7595	6"	14 ¾"	8 ¾"	2 ½"	2 ¼"	6"
Medium	191 mm	445 mm	273 mm	64 mm	57 mm	171 mm
SC7600	7 ½"	17 ½"	10 ¾"	2 ½"	2 ¼"	6 ¾"
Large	222 mm	470 mm	330 mm	64 mm	57 mm	171 mm
SC7605	8 ¾"	18 ½"	13"	2 ½"	2 ¼"	6 ¾"

Chest Support - Women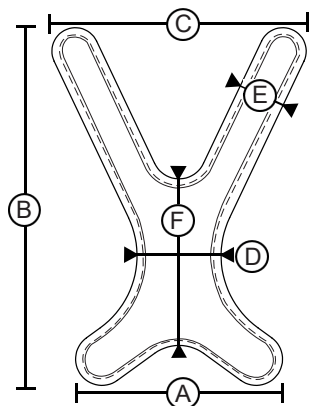

Size	Chest support - Women					
	A	B	C	D	E	F
Small	214 mm	370 mm	263 mm	87 mm	49 mm	172 mm
SC7610	8 ½"	14 ½"	10 ½"	3 ½"	2"	6 ¾"
Medium	203 mm	430 mm	330 mm	70 mm	57 mm	200 mm
SC7615	8"	16 ¼"	14 ½"	2 ¾"	2 ¼"	7 ¾"
Large	246 mm	471 mm	430 mm	70 mm	57 mm	200 mm
SC7620	9 ¾"	18 ½"	17"	2 ¾"	2 ¼"	7 ¾"

Chest Support - Children (4 or 5-point model)

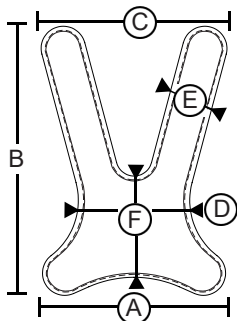Size	Chest Support - Children					
	A	B	C	D	E	F
One size	210 mm	308 mm	212 mm	127 mm	51 mm	116 mm
SC7625 SC7624	8 ¼"	12"	8 ¼"	5"	2"	4 ½"

ENGLISH**Warranty**

This product is covered with our twelve (12) month manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro Inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro Inc. or an authorized dealer to obtain your warranty services. Do not return your chest support without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

Exclusions and Limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibilities for damages incurred during shipping.

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