RIGID SEAT

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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ENGLISH**INTRODUCTION**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
RECOMMENDATIONS.....	3
WARNINGS	3
INSTALLATION	4
MAINTENANCE.....	5
WARRANTY.....	6
FRANÇAIS	7

DESCRIPTION

The rigid seat counteracts pelvic tilt by improving stability and enabling better control of lower limbs. Available in different shapes and sizes.

Installation on original
wheelchair canvas

Installation on rigid
plate

COMPOSITION

ABS or Aluminium

RECOMMEDATIONS

Upon receipt of your rigid seat, an inspection must be made to ensure the product safety.

- Before use, it is important to verify the quality and condition of the rigid seat and its components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine and install the required seating system and make any necessary adjustments.

It is also recommended to consult your doctor or therapist in the month following the installation of a new rigid seat to make sure it is efficient.

WARNINGS



- Always use the provided mounting hardware to install your rigid seat. All basic brackets and all four mounting hooks must be used to securely attach the rigid seat to the wheelchair.
- Never install the rigid seat on a wheelchair that was not specifically designed to be used with this type of seat.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

ENGLISH

INSTALLATION

Installation instructions for seat mounting system



1. Firmly press bracket **C** to the rigid seat plate.
2. Place bracket anchor **D** over bracket **C**.
3. Insert screw **A** through washer **B** and seat plate slot and then tighten nut **E** firmly.
4. Repeat process 1 to 3 for all mounting hooks.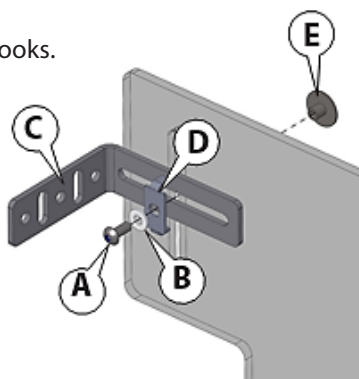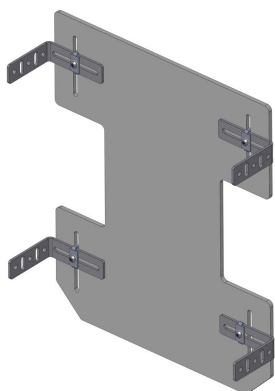

Note - It is important to align and level the brackets on left and right side, both horizontally and vertically.


Installation of mounting hooks on brackets

1. Align seat mounting hook **A** to bracket **B** and make the necessary adjustments.
2. Insert M5 screw **C** through hook and bracket slot and washer **D**, and then assemble with nut **E**.
3. Tighten firmly.
4. Repeat process 1 to 3 for all mounting hooks.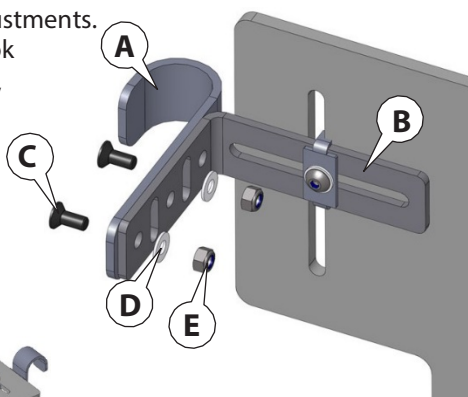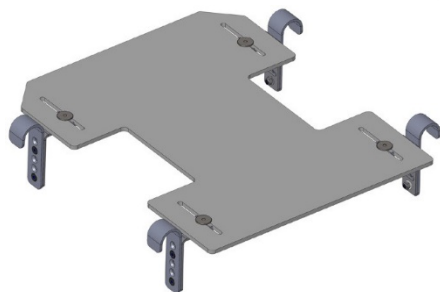


Installation of clicket seat latch to seat tube

1. Unscrew the clamping screw **A** of the clicket seat latch.
2. Place the clamps of the clicket seat latch **B** on both sides of seat tube.
3. Reinsert clamping screw **A** and tighten nut **C** firmly.
4. Repeat process 1 to 3 for the second clicket seat latch.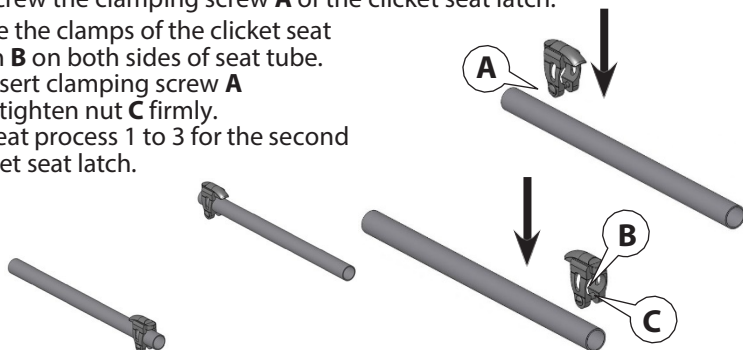

Installation of clicket seat latch to seat tube

1. Unscrew the clamping screw **A** of the clicket seat latch.
2. Place the clamps of the clicket seat latch **B** on both sides of seat tube.
3. Reinsert clamping screw **A** and tighten nut.

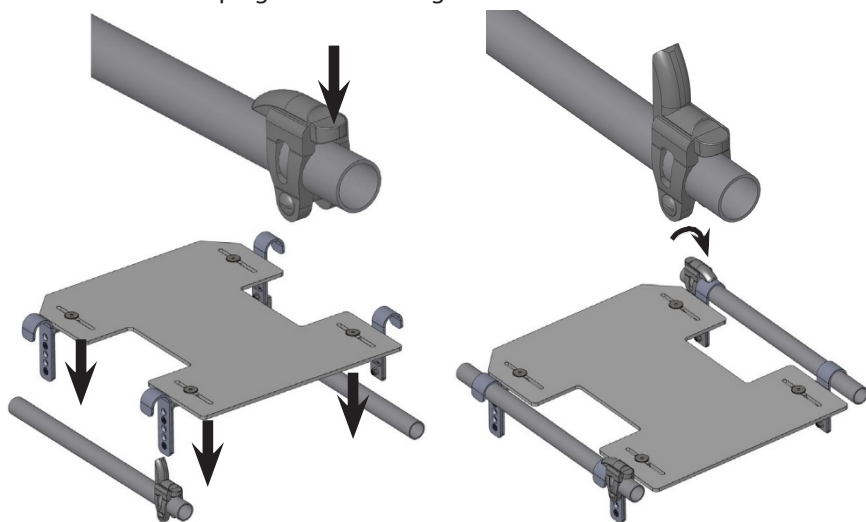MAINTENANCE

The rigid seat can be cleaned with warm water and a mild liquid cleaner and then wiped dry with a clean cloth.

Storage

The rigid seat must be stored in a dry and secure area.

Life span

The estimated life span of the Rigid Seat is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH

WARRANTY

The warranty period for the parts and components of the Rigid Seat is 12 months.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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