## Swing Away Lap Tray

### Owners manual

This manual contains important information about this product.  
Please pass it to the final user upon delivery.

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
## ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our company by choosing one of our products. This user manual has been designed to allow the client to use the swing away lap tray, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your swing away lap tray, and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the company.

*For Physipro Inc., your satisfaction remains a priority.*


### Swing Away Lap Tray Installation

1. Remove the cushion **A** from the armrest that will be used as an anchor for the tray.
2. Put the tray **B** directly on the armrest tube.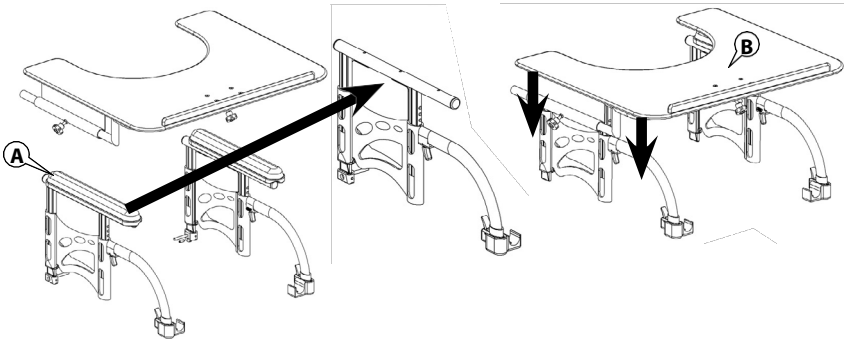


### Width Adjustment

1. Loosen handle **C** to have access to the positioning mechanism.
2. Slide the tray from left to right to have the desired position.
3. Make sure the tray rests on the armrest and secure the handle **C**.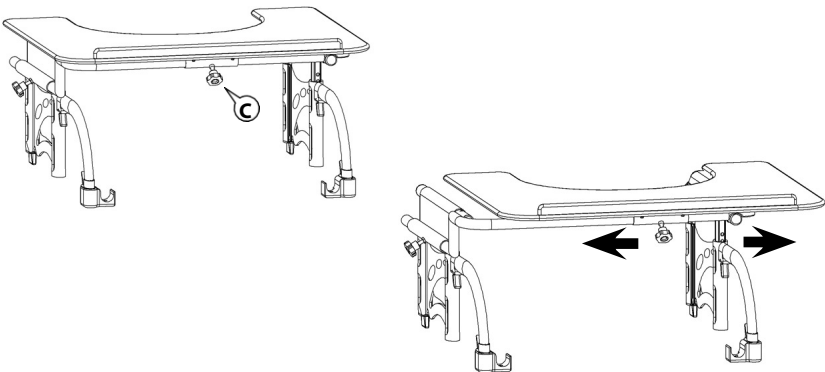

## Folding the tray

1. Slightly loosen handle **D**.
2. Lift the tray **B** and fold it toward the outside.

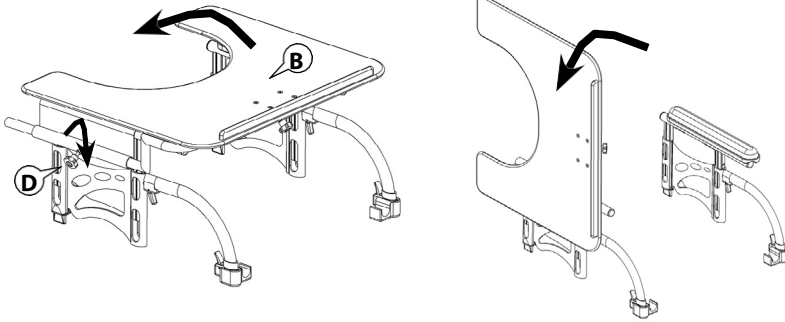## Warranty

This product is covered with our twelve (12) month manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro Inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro Inc. or an authorized dealer to obtain your warranty services. Do not return your tray without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

## Exclusions and Limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibility for damages incurred during shipping.

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