axis^{MC}

PHYSIPRO

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Owners Manual

Read this manual carefully before using the product for the first time.
Keep this manual for future reference.

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www.physipro.com

Thank you!

You have just acquired a high-quality product, proudly made in Canada by skilled technicians. We hope it meets your expectations. If you have any questions regarding its use, maintenance, or features, please do not hesitate to contact us at order@physipro.com.

For Physipro Inc., your satisfaction is our highest priority.

Notice

This user manual has been designed to allow the client to use the Axis Backrest, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your Axis Backrest and extend its operational life.

Liability

Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care, misuse or from any modification made to the product without prior written consent of the Company.

Product Information

Date of purchase:

Distributor:

Address:

Telephone:

Notes:

References

Reference #

Canada-D5224/DW5224/DF5224

Europe-D5229/ DW 5229/DF5229

DM Class : I

HIBC Basic UDI-DI : ++G094PHYBACKRESTNV

EMDN Code : Y180980

Condition : sold individually

About us

Physipro is a leader in designing and manufacturing complex rehabilitation technology products. We are committed to developing more effective aids, devices, and equipment for maximum comfort and greater freedom. Physipro's product line is the result of more than 35 years of experience perfecting this fusion of custom and off-the-shelf products.

"At Physipro, our focus has always been on the individuals for whom we create our products".

Mario Ouellette, CEO

Warning-Read This Manual

Do not use this backrest without reading and understanding this user manual. If you are unable to understand the warnings, cautions and instructions, please contact one of our sale representatives in your local region as detailed on our website at www.physipro.com.

Made in Canada with Canadian and imported parts.

Table of Contents

1. Safety Rules.....	p.2
2. Description.....	p.2
3. Composition.....	p.2
4. Technical specifications.....	p.3
5. Installation.....	p.4
6. Lateral support (option).....	p.4
7. Maintenance.....	p.6
8. Warranty.....	p.7
9. Français.....	p.9.

The product presented in this manual may not be an exact representation of your product.

However, the instructions provided in this manual are still valid. Physipro Inc. reserves the right to modify or change this manual without notice.

1. Safety Rules

- Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the wheelchair.
- Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Do not use the Axis Backrest to push or lift wheelchair. Improper use may cause the backrest to unexpectedly detach from the wheelchair.
- Do not use the Axis Backrest cover or straps to lift wheelchair

2. Description

The Axis adjustable tension backrest offers superior back support and comfort. Characterized by its unilateral strap system, the Axis backrest conveniently enables continuous and precise adjustments in order to ensure better support for individuals with progressive chronic diseases.

This class 1 medical device meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

3. Composition

Padding

Neocore 1", viscose 1/2"

Covers

Standard: Oxford and Triptex U

Velcro Surface: Oxford and Velcro Loop Fabric (optional)

StayClean Cover: 11" of Startex, Oxford, Triptex U (optional)

Mounting system

Nylon straps, buckles and screws

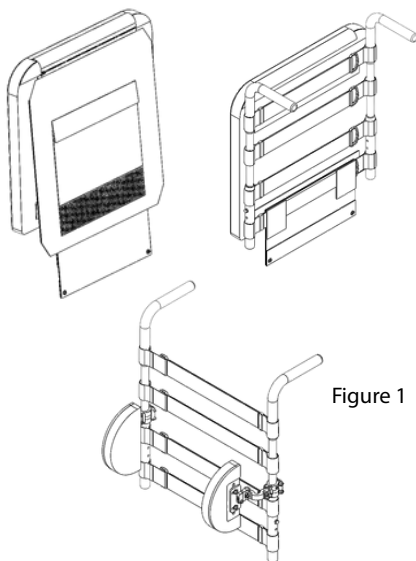
Figure 1

Features a unilateral strapping system fixed to the backpost.

It offers optional inhibition mechanism that favours customization true to a perfect back support.

A choice of laterals that can be set in three different directions in a straight or profiled way favours global trunk support.

Figure 1 (page 2).

4. Technical Specifications

Sizes associated with the backrest

Wheelchair width

14" to 24" (355,6 mm-609 mm)

Nominal width

14" to 24" (355,6 mm-609 mm)

Available in the following nominal heights

14" to 24" (355,6 mm-609 mm)

5. Recommendation and Installation

To ensure protection, optimal comfort, and to extend the product's lifespan, the following precautions should be observed:

- Loose-fitting cotton clothing, preferably without seams in pressure areas, is recommended;
- Before use, allow the backrest to reach room temperature to avoid discomfort due to heat or cold;
- Ensure there are no foreign objects (crumbs, debris, etc.) on the surface of the backrest;
- Clean regularly and periodically inspect the condition of the backrest components;
- It is important that the backrest is appropriately sized for the user and their wheelchair. Otherwise, the benefits of the backrest may be reduced or eliminated;
- The backrest and its cover must be dimensionally compatible and used as described in this manual. A lack of compatibility between the cover and the backrest may reduce or cancel the backrest's performance and increase the risk of skin injuries. The user may also end up in an unstable position and/or fall;
- Avoid prolonged moisture contact with the foam. Do not expose the backrest to extreme heat or cold. Never place it near an open flame.
- Do not place other products or materials (unless specified in this manual) under your backrest or between the user and the backrest. The user may end up in an unstable position and/or fall;
- Always check and position the backrest according to the orientation indicators marked on the backrest and the cover. Proper positioning is essential to ensure maximum comfort and effective pressure relief;
- Crash tests have not been conducted on this backrest. Even if the mobility aid it is installed on has passed ISO-7176-19, this ISO compliance will no longer apply;
- Before use, it is important to check the quality and condition of the backrest components. Refer to the Maintenance Checklist on page 6;
- Physipro Inc. backrests are intended exclusively for use on a mobility aid (manual or power wheelchair);
- Always use the hardware provided to install the backrest;
- Before installation, verify the dimensional compatibility between the backrest and the width of the wheelchair.
- Do not place towels, folded sheets, or any other material between the user and the backrest, as this may reduce or cancel the backrest's performance and increase the risk of skin injuries;

The backrest must be trialed before any prolonged use. To prevent the appearance of redness caused by pressure points, we recommend making small position shifts as often as possible. It is also necessary to regularly (at least once per day) check your skin for any redness, especially around bony prominences and the underarm area. If redness or skin discoloration is observed, discontinue use of the backrest immediately. If the discoloration does not disappear within thirty (30) minutes, consult a healthcare professional right away.

Be aware that no backrest can replace good skin hygiene, a proper diet, and frequent changes of position.

Note:

It is recommended to consult your therapist within one month of installing a new backrest to ensure its effectiveness.

Installation of the Axis Backrest

1. Evenly position the straps by wrapping their Velcro ends around the backrest posts on each side (see figures 2 and 3).

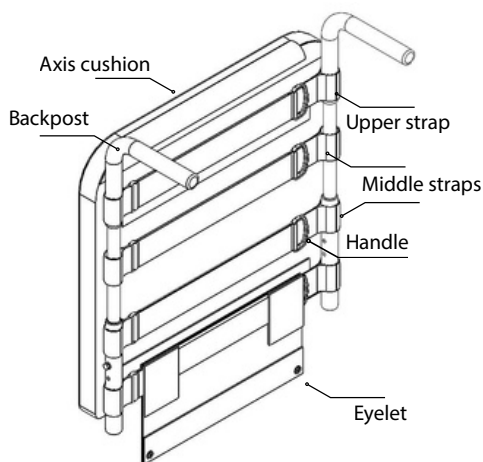
Figure 2

2. Pull the handles to adjust the strap tension as desired (see figure 2).

3. Install self-locking ties through the eyelets at the bottom of the backrest to ensure consistent tension. Position the cushion in its correct orientation. Pass the small Velcro flap over the upper strap and press firmly to secure it in place.

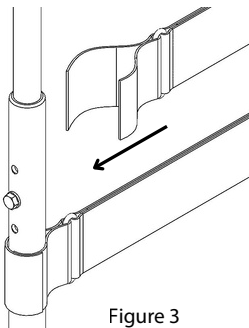
Figure 3

4. Cover the straps and backrest posts with the aesthetic flap. Press firmly along the edges of the flap to ensure they adhere to the Velcro on the posts and the section between the eyelets. Then, press the cushion against the straps.

6.Lateral Installation (optional)

Install the anchoring mechanisms at a predetermined height by tightening the clamp with a 4mm Allen key (Figures 4).

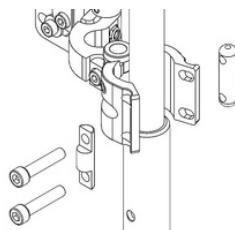
Figure 4

Screws A are used for the final height and depth adjustments.
 Screw B is used for the angle and thorax adjustments.
 Screws C are used for the width adjustment (figure 5).

Note: Use the right screw length that fit the backposts size.

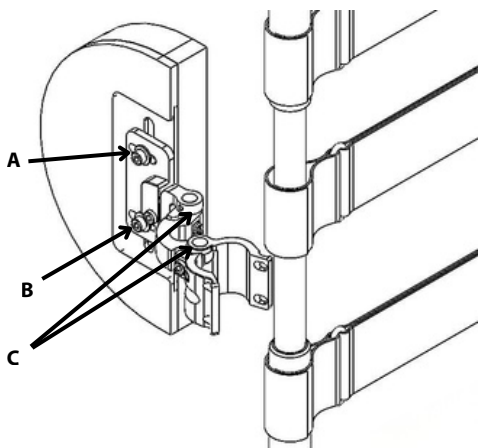
Figure 5

6.1 Installing the cover on the lateral thoracic support

Unfasten both zippers to fully expand the opening of the cover. (Figure 6.1)

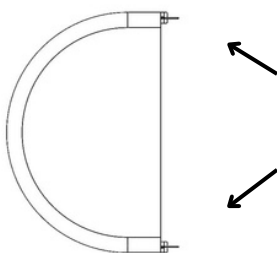
Figure 6.1

Start inserting the cushion by the top of the lateral. (Figure 6.2)

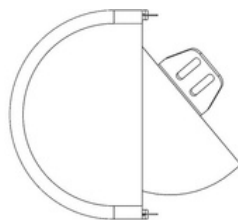
Figure 6.2

Push down while making sure the cover opening passes around the lateral by making a downward rotating movement. (Figure 6.3)

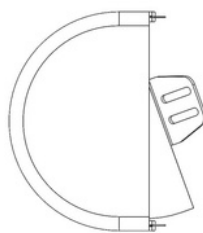
Figure 6.3

Fasten both zippers when cover is properly positioned. (Figure 6.4)

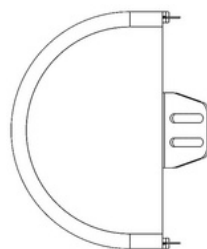
Figure 6.4

6.2 Axis Strapping System (optional)

1) Install the anchor mechanisms at a predetermined height by tightening the jaw with a 4mm Allen key. (Figures 7 and 8)

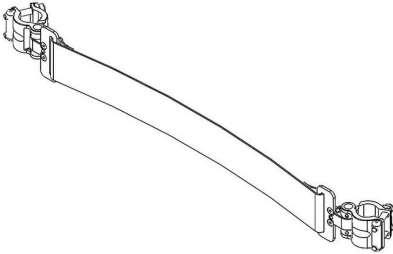
Figure 7

2) Change the fastener angles to adjust the strap depth with screw D. (Figure 8)

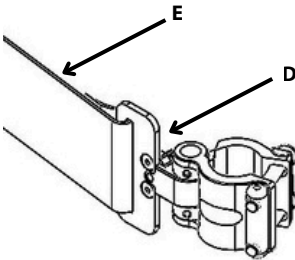
Figure 8

3) Tighten the strap by pulling part E.

Note: It is preferable to pass the strap under the loop behind the cushion.

7. Maintenance

The maintenance of the Axis backrest is essential; therefore you will extend its lifetime and ensure its safety. Physipro recommends that the Axis backrest be checked for wear and tear every 6 months. Make sure that the laterals are fastened properly when receiving and verify it each week. Do not continue to use the system after identifying loose or broken components.

Machine wash
max. 30 °C (122°F)

Do not iron

Do not use
chlorine-based bleach

Do not dry clean

Cleaning

Outer cover

Wipe down the outer cover with a soft cloth, warm soap and water. The outer cover can be machine washed in warm water on a gentle cycle. Air dry the outer cover.

Straps assembly

Wipe down strap assembly as needed using light soap and warm water. Do not machine wash.

Note

- Remove the foam from inside the cover prior to machine washing;
- The foam core may be washed by hand with light soap and water.

Maintenance Checklist

Upon receiving your backrest, it is important to complete all the inspections listed below to ensure the proper function and condition of the backrest and mounting hardware.

Maintenance Checklist	Weekly	Monthly
Backrest: Check backrest and straps for any signs of wear		✓
Lateral Supports: <ul style="list-style-type: none"> • Verify the solidity of mounting clamps • Check lateral supports for any signs of wear 	✓	✓

Life Span

The estimated life span of your backrest is three (3) years, depending on the user and how the product is used, under normal use conditions.

Storage

Backrest must be stored in a dry and secure area.

Disposal of Medical Device

Do not burn or throw away medical devices outside of designated areas.

WARNINGS

Never store backrest in direct sunlight or leave it exposed to excessive humidity or heat.

Never place backrest near sharp objects or in an area where it may be struck by an object.

8.Warranty

The Axis backrest, a product of Physipro inc., is covered with our twenty-four (24) months manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services.

Do not return your backrest without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro inc. will not take any responsibilities for damages incurred during shipping.

Terms and Conditions

The seat cushions and backrests are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained. The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment. The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

If you have any question, please contact us:
1 800 668-2252, order@physipro.com

Physipro Inc.
370, 10e Avenue South,
Sherbrooke, Quebec, J1G 2R7

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