PHYSIPRO ® MC

ANTI-FALL BED SYSTEM

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
RECOMMENDATIONS.....	4
WARNINGS.....	4
INSTALLATION.....	5
MAINTENANCE.....	6
WARRANTY.....	7
FRANÇAIS.....	9

ANTI-FALL BED SYSTEM

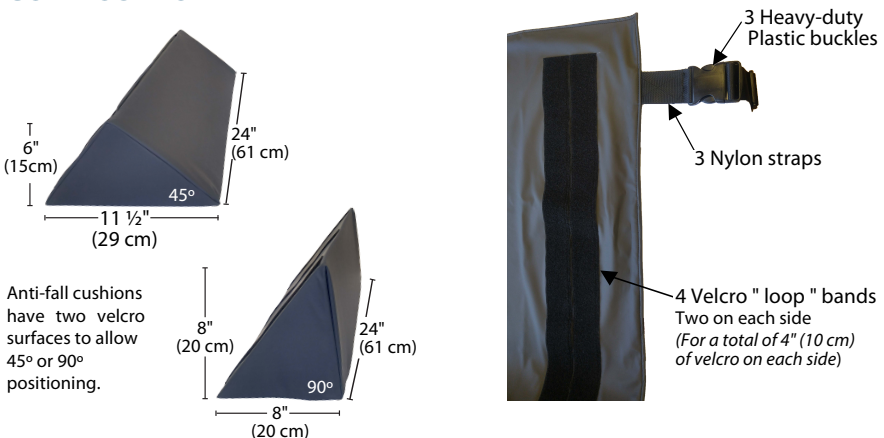
The Anti-Fall Bed System prevents the user from falling out of bed and reduces the risk of bedrail entrapment. Three anti-fall cushions placed on either side of the bed, create a comfortable barrier between the user and the edge of the mattress. Cushions can be positioned at a 45 or 90 degree angle depending on the postural needs of the user.

Made with hospital quality fabric that is fluid proof, antimicrobial, hypoallergenic and flame retardant.

This system is intended for use on mattress with a maximal width of 38" (96 cm).

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Cushion

- Firm urethane

Cushion cover

- Fabric: Startex
- 2" (5 cm) Velcro "hook" bands (3 bands on each velcro surface)
- Waterproof zipper

Velcro fabric

- Fabric: Startex
- 2" (5 cm) Velcro "loop"
- 36" x 80" (91 cm x 203 cm)

Straps

- 2" (5 cm) Nylon webbing
- 2" (5 cm) Plastic buckles

RECOMMENDATIONS

Before use, verify the quality and condition of the anti-fall bed system components.

To ensure your bed system provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- The anti-fall bed system must be installed on a mattress of no more than 38" (96 cm) in width. Mattress surface must be clean before installation.
- Inspect straps weekly to ensure they are well adjusted.
- Before use, let the cushions and Velcro fabric get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Remove any foreign object (crumbs, debris, etc.) from cushions and velcro fabric surfaces.
- Clean on a regular basis and visually and all components of the anti-fall bed system periodically.
- Never place cushions and velcro fabric near sharp object.
- The components of the anti-fall bed system should be stored on a flat surface, in a dry area and out of direct sunlight.

It is recommended to consult your therapist in the month following the installation of the bed system to make sure it is efficient.

WARNINGS

Consult a healthcare professional: Physipro™ recommends that a health care professional such as a doctor or therapist, experienced in positioning, be consulted to install the anti-fall bed system and make any necessary adjustments.

Positioning: The proper positioning of the anti-fall bed system is essential to ensure maximum comfort.

Environment: Avoid exposing component of the bed system to fluid or moisture for prolonged periods. Avoid exposing the bed system to extreme cold or heat. Never place near open flame.

INSTALLATION

The Anti-fall bed system is quick and easy to install, please follow the instructions below:

1. Position the velcro fabric on the mattress.
2. Slide straps under the mattress and fasten the plastic buckles.
3. Tighten straps firmly around the mattress.

WARNING

Loose straps may cause the bed system to slide around or off the mattress. Make sure the straps are tightly fitted to the mattress, this will keep the bed system in place and will prevent the velcro fabric from bunching up.

4. Determine at what angle the cushions should be positioned.
5. Position three cushions on one side of the bed on the velcro bands.
6. Firmly press down on cushions to ensure they are securely fastened to the velcro bands.
7. If you have a bilateral system, repeat steps 4 to 6 for the other side.
8. Once the anti-fall system is installed, cover the system with bed sheets.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your anti-fall bed system. It is recommended to visually inspect the components of the bed system during the cleaning process.

Cleaning the cushion covers and velcro fabric

Startex fabric is a wipe clean fabric, it can be cleaned with most disinfectants and cleaning compounds. We recommend using a damp cloth and mild soap to clean the bed system weekly or as necessary.

WARNINGS

Never use harsh solvents.

Do not submerge in water.

Never place foam cushion in washer or dryer.

Cleaning the foam cushion

1. Remove the cover.
2. Wipe foam lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on cushion to absorb excess moisture, then let air dry completely.
4. Reinstall cover.

WARNINGS

Do not use soap.

Do not submerge in water.

Never place foam cushion in washer or dryer.

Storage

Physipro Inc. bed systems should be stored on a flat surface, in a dry area and out of direct sunlight.

Life Span

The estimated life span of your bed system is one (1) year.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty period for the following part(s) and component(s) of your Anti-Fall Bed System is twelve (12) months.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at order@physipro.com, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc. The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person. Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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