## **PELVIC POSITIONING BELT WITH PLASTIC BUCKLE**

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This booklet contains important information about this product. Please give to the owner upon delivery.

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## INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This user manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way.

For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

**Physipro Inc., your satisfaction is our highest priority.**

## PRODUCT INFORMATION

Date of purchase : \_\_\_\_\_

Supplier: \_\_\_\_\_

Address : \_\_\_\_\_

Telephone : \_\_\_\_\_

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

**Physipro Inc. reserves the right to modify or change this manual without notice.**

OWNER’S MANUEL– PELVIC POSITIONING BELT WITH PLASTIC BUCKLE ..... 3

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    GARANTIE ..... 13

## DESCRIPTION

The Pelvic positioning belt with plastic buckle is intended to optimize safety and to improve the occupant's posture. Made with a durable nylon webbing and equipped with a heavy-duty plastic buckle, this lightweight pelvic positioning belt is offered in different widths to ensure it is well adapted to the occupant's body size. This positioning belt provides a single adjustment and has durable steel strap anchor plates for easy installation. Pediatric, Children and Adult models are available.

## COMPOSITION

Nylon strap (1-inch, 1½-inch or 2-inch)

Plastic buckle (1-inch, 1½-inch or 2-inch)

Plastic slide buckle

Steel strap anchor plates

Belt attachment block for round seat rails (option)

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

## RECOMMENDATIONS

**To ensure the longevity of your positioning belt, the following precautions should be applied.**

- Before use, it is important to verify the quality and condition of the positioning belt's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to install, and adjust the positioning belt.
- Clean regularly and inspect the positioning belt's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the positioning belt.

**It is recommended to consult your doctor or therapist in the month following the installation of a new postural support device to make sure it is efficient.**

## WARNINGS


- Physipro Inc. positioning belts are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your positioning belt.
- Never push or lift wheelchair with the positioning belt, doing so may cause the belt to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Before use, let the positioning belt get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your positioning belt to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

## INSTALLATION

Follow the installation procedure that applies to the type of seat rail installed on your wheelchair (round or square tubing).

Before beginning the installation of the pelvic positioning belt, determine the correct location for the belt strap anchor plates on the seat rails.


**Note** – The pelvic belt should be in direct contact with the occupant’s lower pelvis and hip bones. The pelvic belt should be positioned at an angle between 45 and 75 degrees to the horizontal.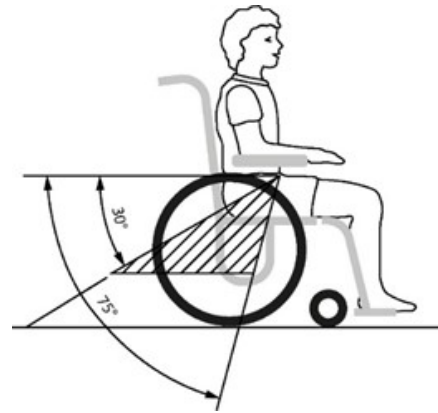

### Round seat rails

**Required tools:** 4mm and 5 mm Allen keys


1. Position both parts of the belt attachment block to the seat rail.
2. With a 5mm Allen key, tighten screw **B** firmly.
3. With a 4mm Allen key, secure the anchor plate to the attachment block by fastening screw **A**.
4. Repeat these steps for the other side.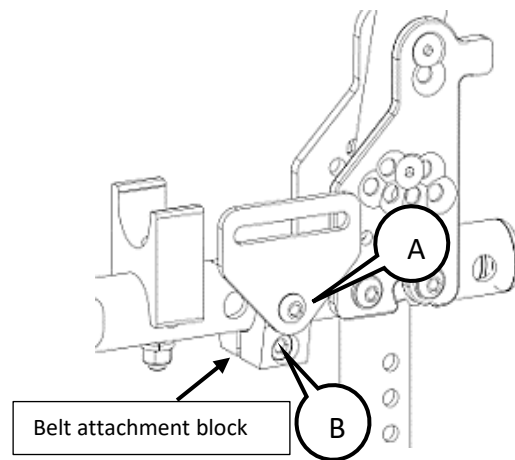

### Square seat rails

**Required tool:** 4mm Allen key

1. Tighten screw **A** with a 4mm Allen key.
2. Repeat these steps for the other side.

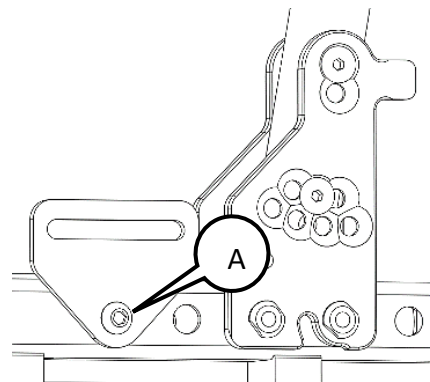**Note** – Make sure both anchor plates are placed in the corresponding adjustment holes of the opposite seat rail.

## Adjusting the pelvic positioning belt

To adjust the length of the positioning belt, slide the belt strap **C** into the plastic slide buckle **D** and tighten or loosen until the belt fits snugly without causing any discomfort.

### WARNINGS

- When using your wheelchair, it is important to always fasten your positioning belt for added protection.
- It is important to adjust your positioning belt to maximize comfort and safety.
- Improper use can result in serious injury or death.

## MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your positioning belt.

### Cleaning the pelvic positioning belt

A weekly cleaning is recommended.

1. With a mix of water and a soft soap or neutral detergent, gently wipe the belt.
2. Remove excess moisture by lightly patting the belt with a clean cloth and then let air dry completely.

### WARNING

- Do not submerge in water.
- Never machine wash or dry.
- Do not iron.

## Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

## WARRANTY

The warranty period for the parts and components of the pelvic positioning belt with plastic buckle is **12 months**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period.

Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

### **This warranty does not apply to:**

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

### Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **[order@physipro.com](mailto:order@physipro.com)**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

***Physipro Inc.***

*370, 10<sup>th</sup> Avenue South, Sherbrooke, Quebec, J1G 2R7*

### Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

**The foregoing warranty is exclusive and in lieu of all other expressed warranties.**

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