PHYSIPRO[®]

LATERAL CONTROL HEADRESTS

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
RECOMMENDATIONS.....	4
WARNINGS.....	4
INSTALLATION.....	5
ADJUSTMENTS.....	6
MAINTENANCE.....	7
WARRANTY.....	8
FRANÇAIS.....	9

DESCRIPTION

Physipro headrests are used to promote an upright and aligned head position for those with an asymmetrical cervical presentation, generalized weakness/fatigue, or who use a tilting wheelchair. The lateral control headrests offer additional support to control lateral flexion and rotation.

Physipro Inc. headrests feature a multi-axe mounting fixture that ensures precise head positioning.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

LATERAL CONTROL HEADREST WITH ADJUSTABLE EXTENSION(S)

Cushion

Plastazote
Soft memory foam

Rigid structure

Steel

LATERAL CONTROL HEADREST WITH INTEGRATED & ADJUSTABLE EXTENSIONS

Cushion

Plastazote
Soft memory foam

Rigid structure

Steel

RECOMMENDATIONS

To ensure the longevity of your headrest, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the headrest's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required headrest support system and make any necessary adjustments.
- Clean regularly and inspect the headrest's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the headrest's surface.

It is recommended to consult your doctor or therapist in the month following the installation of a new headrest to make sure it is efficient.

WARNINGS


- Physipro Inc. headrests are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your headrest.
- Never push or lift wheelchair with the headrest or its support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- Before use, let the headrest get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your headrest to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION


Required tool: 4 mm Allen key

Installing the headrest on the headrest support

1. Remove headrest cover.
2. Insert the ball joint **A** of the headrest support rod into the headrest's multi-axe mounting fixture **B**.
3. Position the ball socket clamp **C** on the headrest support rod.
4. Insert screws **D** into the ball socket clamp **C** and mounting fixture **B**.
5. Adjust headrest position and firmly tighten all three (3) screws **D**.
6. Adjust the lateral extensions before reinstalling the cover.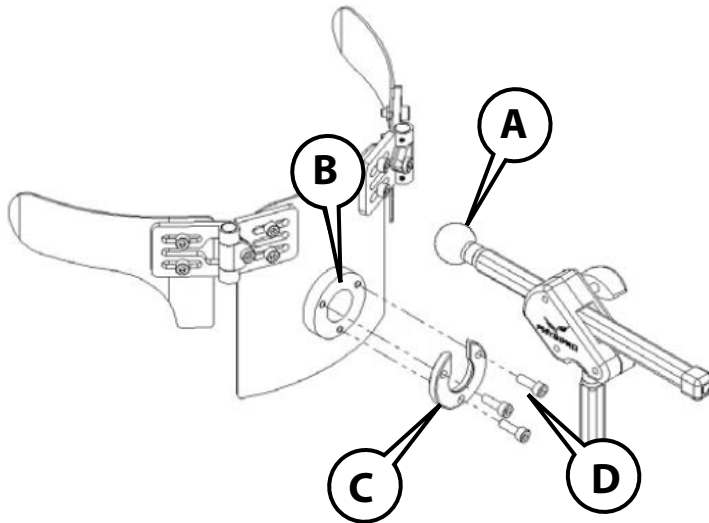

Product should be as shown

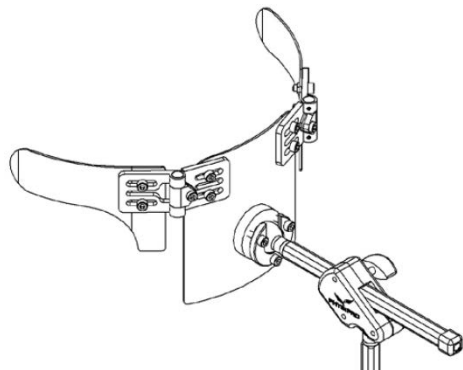
ADJUSTMENTS

Required tool: 4 mm Allen key


Adjusting the lateral extensions

1. Loosen screws **E** and **F**.
2. Position the lateral extensions **G** as desired.
3. Firmly tighten screws **E** and **F**.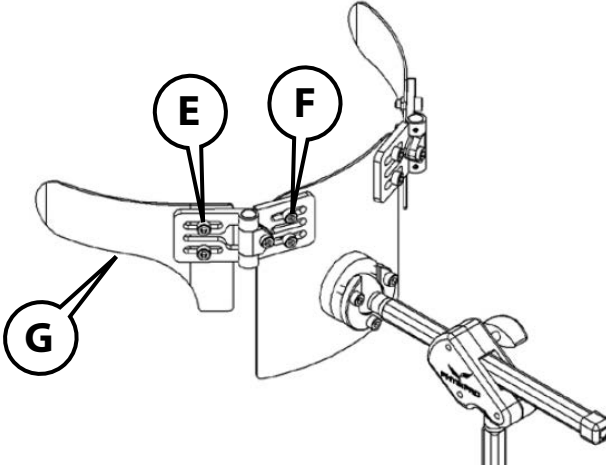

Adjusting the angle of the side panels

1. Loosen screws **H**.
2. Position the lateral extensions **G** at the desired angle.
3. Firmly tighten screws **H**.

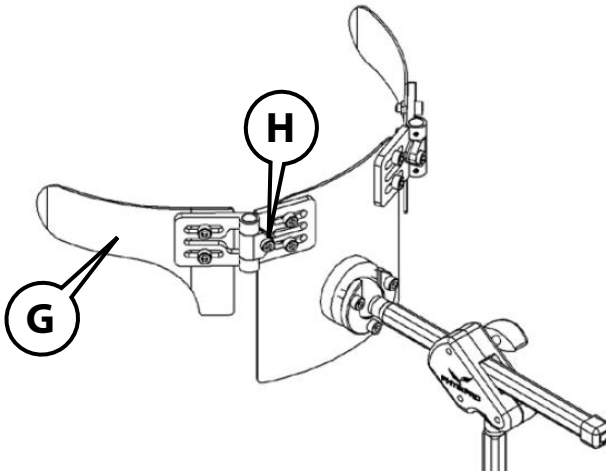MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your headrest. It is recommended to visually inspect all the components of the headrest during the cleaning process.

A weekly cleaning is recommended.

Cleaning the headrest

1. Remove the neoprene cover.
2. Wash the neoprene cover with a mild soap or neutral detergent and rinse thoroughly. Let the cover air dry completely.
3. With a dry cloth, dust off the rigid structure surface before cleaning.
4. Gently wipe the rigid structure with a cloth dampened with a mild soap or neutral detergent and dry-off with a clean cloth.
5. Remove all debris on the padding surface.
6. With a mix of water and a soft soap or neutral detergent, gently wipe clean the padding.
7. Remove excess moisture by lightly patting the surfaces with a clean cloth and then let air dry completely.
8. Once all components are completely dry, reinstall the neoprene cover.

WARNINGS

- Do not submerge the foam padding in water.
- Never machine wash or dry.

Storage

Headrest must be stored in a dry and secure area.

WARNINGS

- Never store your headrest in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your headrest near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your headrest is five (5) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH**WARRANTY**

The warranty period for the parts and components of Physipro Inc. headrests is **12 months**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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