# **Multi-adjustable Amputee Support**

## **Owners manual**

This manual contains important information about this product.  
Please pass it to the final user upon delivery.

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


## ENGLISH

Physipro Inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our company by choosing one of our products. This user manual has been designed to allow the client to use the multi-adjustable amputee support, a Physipro Inc. product, safely in an optimal way. For all adjustments and settings, Physipro Inc. strongly recommends you consult a qualified professional. Physipro Inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your multi-adjustable amputee support and extend its operational life. Physipro Inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the company.

*For Physipro Inc., your satisfaction remains a priority.*

### 1. Installing the amputee support on a legrest

1. Loosen screws **A** to free the mechanism anchorage.
2. Sink the anchorage mechanism in the legrest tubing.
3. Adjust the mechanism at the desired height and firmly tighten screws **A**.
4. Insert the amputee support anchorage **B** on legrest support **C**.
5. Firmly tighten screws **A**.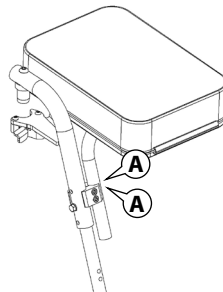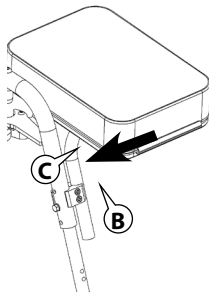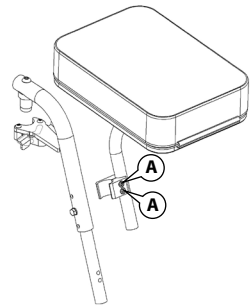


## 2. Amputee Support Adjustments

With its multiple adjustments, the multi-adjust amputee support enables an optimum alignment of the tibial segment.

### Angle adjustment

0° to 90° movement.


1. Loosen screw **D** to enable mechanism **E** to move easily.
2. Move the amputee support to the desired angle and firmly tighten screw **D**.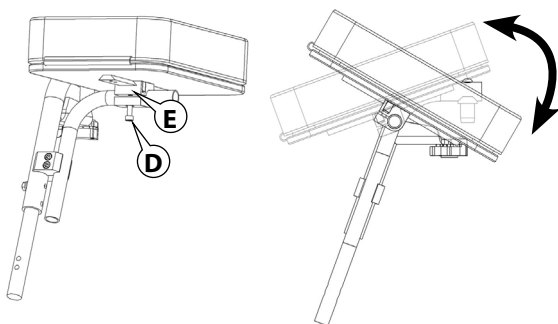

### Height adjustment

9 inch (23 cm) movement


1. Loosen screws **A** to free the mechanism anchorage.
2. Adjust the mechanism at the desired height and firmly tighten screws **A**.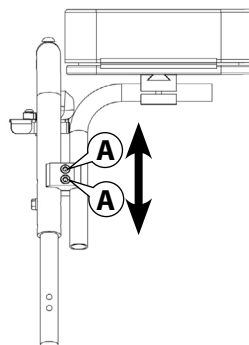

### Width adjustment

5 inch (13 cm) movement

1. Loosen screw **D** to enable mechanism **E** to move freely.
2. Adjust the mechanism at the desired width and firmly tighten screw **D**.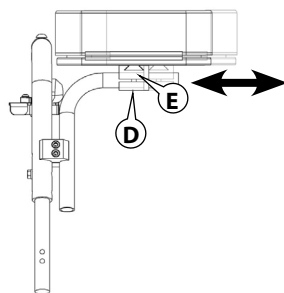


## ENGLISH

### Depth adjustment

1 inch (2.5 cm) movement

1. Loosen screw **D** to enable mechanism **E** to move freely.
2. Adjust the mechanism at the desired depth and firmly tighten screw **D**.

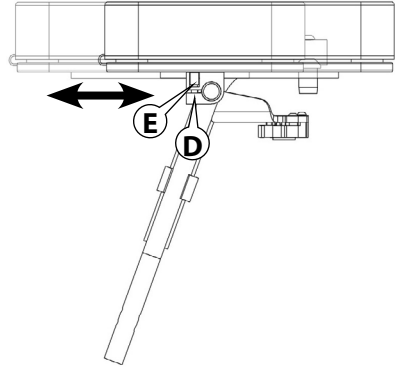### Warranty

This product is covered with our twelve (12) month manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro Inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro Inc. or an authorized dealer to obtain your warranty services. Do not return your multi-adjustable amputee support without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

### Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without written consent from Physipro Inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro Inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro Inc. will not take any responsibility for damages incurred during shipping.

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