Owners manual

This manual contains important information about this product.
Please pass it to the final user upon delivery.

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ENGLISH

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the Axis Back, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional. Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your Axis Back and extend its operational life. Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

For Physipro inc., your satisfaction remains a priority.

Table of contents

1. Safety Rules	2
2. Components	3
3. Axis Backrest Installation	4
4. Laterals installation (optional)	5
5. Axis strapping system (optional)	6
6. Care and Maintenance	7
7. Warranty	8

1. Safety Rules

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the wheelchair.

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess for the potential need for antitippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability

Do not use Axis Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

Do not use Axis Back cover or straps to lift wheelchair

2. Components

Features a unilateral strapping system fixed to the backpost.

It offers optional inhibition mechanism that favours customization true to a perfect back support.

A choice of laterals that can be set in three different directions in a straight or profiled way favours global trunk support.

The hyperkyphosis Axis version offers a protective and comfortable outline for the shoulders and pelvis.

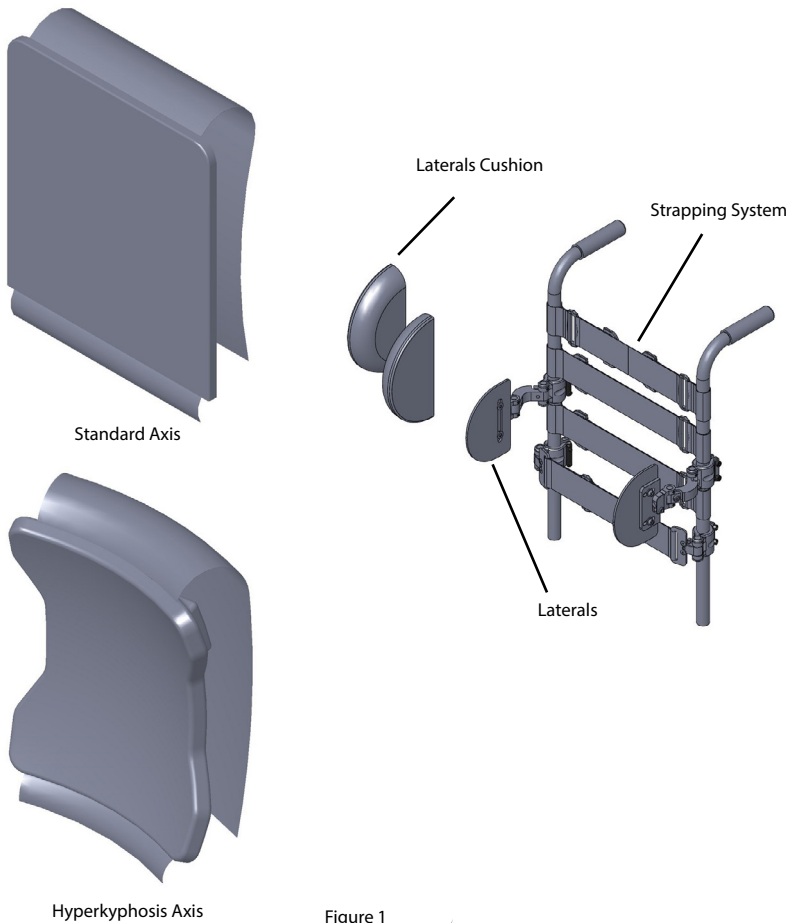
Figure 1

ENGLISH

3. Axis Backrest Installation

- 1) Insert the straps under the loops behind the standard or kyphosis cushion. (Figure 2) The top strap is built with a central elastic part.
- 2) Place the straps in a regular manner by winding their ends around the backposts beams on each side. (Figure 3)

Note : For more strap stability, fix the velcro strips on the backposts, under the strap fasteners.

- 3) Press the cushion on the backposts so that the velcro strips link up. Pull up the handles to adjust strap tension. (Figure 2)

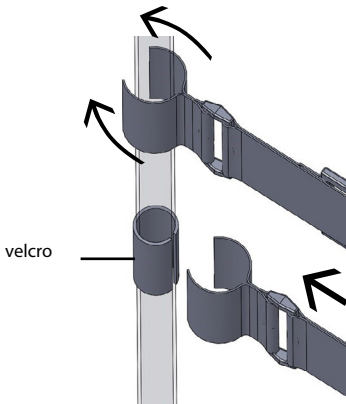
Figure 3

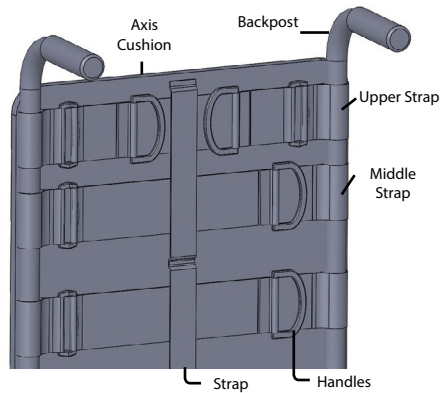
Figure 2

4. Laterals installation (optional)

- 1) Install the anchoring mechanisms at a predetermined height by tightening the clamp with a 4mm Allen key. (Figures 4, 5 and 6)

Screws **A** are used for the final height and depth adjustments.
 Screw **B** is used for the angle and thorax adjustments.
 Screws **C** are used for the width adjustment.

Note: Use the right screw length that fit the backposts size.

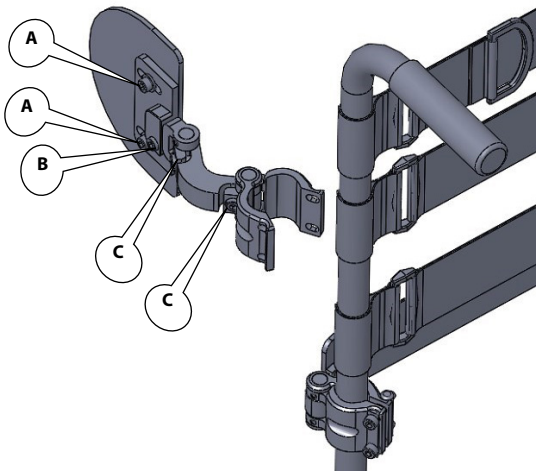
Figure 4

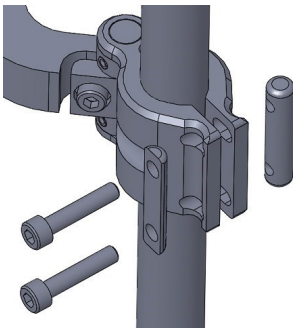
Figure 5

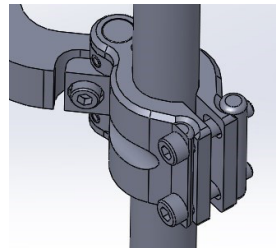
Figure 6

ENGLISH

4.1 Installing the contoured cushion

Make sure a Velcro strip is fixed on the thoracic support plate facing the inside. (Figure 6.1)

Insert your hand in the cushion through the cover opening to prevent the Velcro strips from touching each other while you place the cushion. (Figure 6.2)

Start inserting the cushion by the top of the lateral. (Figure 6.3)

Push down while making sure the cover opening passes around the plate by making a downward rotating movement. (Figure 6.4)

Place the cushion, remove your hand from the cover and push a few times on the front of the cushion to make sure the Velcro strips are hooked. (Figure 6.5)

Figure 6.1

Figure 6.2

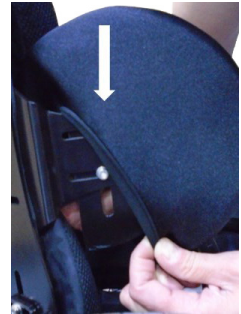
Figure 6.3

Figure 6.4

Figure 6.5

4.2 Removing the contoured cushion

Insert your hand inside the contoured cushion between the plate and the cushion to unhook the Velcro strips.

Move the cover opening down the lateral plate and remove the cushion with an upward rotating movement.

4.3 Removing the foam from the contoured cushion

Fold the cushion in the cover. (Figure 6.7)

Carefully and slowly take the cushion out of the cover opening. (Figure 6.8)

Completely remove the cushion. (Figure 6.9)

Figure 6.6

Figure 6.7

Figure 6.8

Figure 6.9

ENGLISH**5. Axis strapping system (optional)**

- 1) Install the anchor mechanisms at a predetermined height by tightening the jaw with a 4mm Allen key. (Figures 7 and 8)
- 2) Change the fastener angles to adjust the strap depth with screw **D**. (Figure 7 and 8)
- 3) Tighten the strap by pulling part **E**.

Note : It is preferable to pass the strap under the loop behind the cushion.

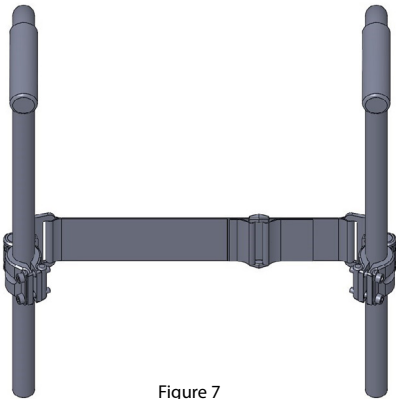
Figure 7

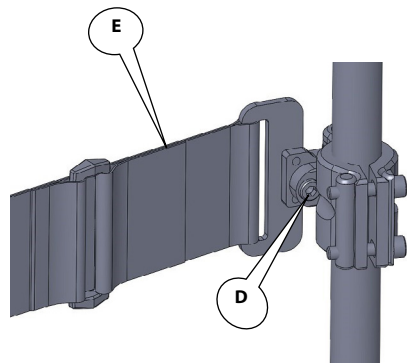
Figure 8

6. Care and maintenance

The maintenance of the Axis Backrest is essential; therefore you will extend its lifetime and ensure its safety. Physipro recommends that the Axis Backrest be checked for wear and tear every 6 months. Make sure that the laterals are fastened properly when receiving and verify it each week. Do not continue to use the system after identifying loose or broken components.

Cleaning

Outer cover

Wipe down the outer cover with a soft cloth, warm soap and water. The outer cover can be machine washed in warm water on a gentle cycle. Air dry the outer cover.

NOTE

- ✓ Remove the foam from inside the cover prior to machine washing
- ✓ The foam core may be washed by hand with light soap and water

Strap Assembly

Wipe down strap assembly as needed using light soap and warm water.

NOTE

- ✓ Do not machine wash the strap assembly.

machine wash
maximum temperature 50 C

do not iron

do not use chlorine-based bleach

do not dry clean

ENGLISH

7. Warranty

The Axis backrest, a product of Physipro inc., is covered with our twenty-four (24) months manufacturer's warranty. It includes parts and labor on manufacturer's defects, optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services. Do not return your backrest without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address and delivery date to the address on this manual cover.

Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected or did not have the proper maintenance. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro inc. will not take any responsibilities for damages incurred during shipping.

Terms and Conditions

The seat cushions and backrests are reserved exclusively for use on mobility aid (manual or power wheelchairs) Physipro will not be held responsible for any misuse of the product. Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

The respect of the safety instructions mentioned in this manual is essential to avoid injury. Improper or inadequate adjustment increases the risk of accident. Before making adjustments, it is important to read the entire contents of this manual. Physipro will not be held responsible for misuse or non professional adjustment.

The product presented in this manual may not be an exact representation of the product you have on hand. The instructions in this manual are still valid for your product. Physipro reserves the right to modify or change this manual without notice.

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