HEADBAND

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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ENGLISH**INTRODUCTION**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
TECHNICAL SPECIFICATIONS.....	3
RECOMMENDATIONS.....	3
WARNINGS	4
INSTALLATION	4
MAINTENANCE.....	5
WARRANTY.....	6
FRANÇAIS	7

DESCRIPTION

The headband helps control involuntary head movement by preventing the head from dropping forward. Padded with neoprene to comfortably stabilize the occupant's head, this headband is secured to the headrest with 2-inch wide by 4-inch-long Velcro fasteners at its extremities.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

- ¼-inch Neoprene
- Lycra
- 2-inch Velcro

TECHNICAL SPECIFICATIONS

The length of the headband is determined by the size of the headrest and the occupant's head circumference.

Available Headband Lenght	
cm	in
36	14
41	16
46	18
51	20
56	22

RECOMMENDATIONS

To ensure the longevity of your postural support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the headband.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to install, and adjust the headband.
- Clean regularly and inspect the headband daily to ensure it is properly fastened.
- Remove any foreign object (crumbs, debris, etc.) from the headband.

It is recommended to consult your doctor or therapist in the month following the installation of a new postural support device to make sure it is efficient.

ENGLISH

WARNINGS

- Physipro Inc. postural support devices are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Before use, let the headband get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your headband to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION

1. Fasten one side of the headband to the headrest with the Velcro fastener.
2. Properly position the occupant in their wheelchair with their head in an upright position.
3. Fasten the other side of the headband to the headrest.

Note - It is important that the headband be properly adjusted to the occupant's head. The headband must be tight enough to prevent the head from dropping but should not cause any discomfort to the occupant.

WARNING

Always make sure the headband is securely attached before using your wheelchair.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your headband.

Cleaning the headband

A weekly cleaning is recommended.

1. Remove all debris on the headband's surface and between seams and folds.
2. With a mix of water and a soft soap or neutral detergent, gently wipe clean.
3. Remove excess moisture by lightly patting the headband with a clean cloth and then let air dry completely.

WARNING

- Never machine wash or dry.

Storage

Postural supports must be stored in a dry and secure area.

WARNINGS

- Never store your postural support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your postural support near sharp objects or in an area where it may be struck by an object.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH**WARRANTY**

The warranty period for the parts and components of the headband is 90 days.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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