PHYSIPRO[®]

4-POINT PADDED "Y STYLE" BELT

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
RECOMMENDATIONS.....	3
WARNINGS	4
INSTALLATION	4
MAINTENANCE.....	5
WARRANTY.....	6
FRANÇAIS	7

DESCRIPTION

The 4-point padded "Y Style" positioning belt supports the user's upper thighs and hips to prevent forward sliding and stabilize the pelvis. Suitable for users with low muscle tone who are at risk of falling or seizing. This belt promotes proper pelvic alignment by limiting pelvic tilt.

To ensure a balanced pressure distribution and to optimize the user's comfort, velcro adjustments have been placed on each side of the buckle, this allows for proper centering of the buckle and positioning of the neoprene cushions. The length of the straps can be adjusted by sliding buckles, placed at the anchor points.

Made of lightweight and durable nylon webbing with steel anchor plates. This belt can be equipped with a metal auto buckle or a plastic buckle.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

Nylon strap

Neoprene pads or gel and neoprene pads (option)

Auto buckle or Plastic buckle

Plastic slide buckles

Steel anchor plates

Belt attachment block for round seat rails (option)

RECOMMENDATIONS

To ensure the longevity of your positioning belt, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the positioning belt's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to install, and adjust the positioning belt.
- Clean regularly and inspect the positioning belt's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the positioning belt.

It is recommended to consult your doctor or therapist in the month following the installation of a new postural support device to make sure it is efficient.


ENGLISH

WARNINGS

- Physipro Inc. positioning belts are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your positioning belt.
- Never push or lift wheelchair with the positioning belt, doing so may cause the belt to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Before use, let the positioning belt get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your positioning belt to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION

Follow the installation procedure that applies to the type of seat rail installed on your wheelchair (round or square tubing).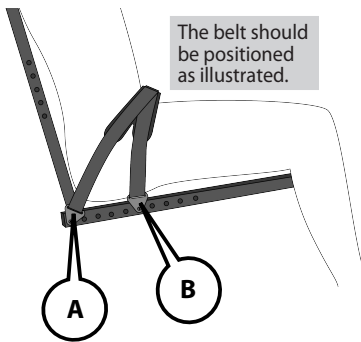


Square seat rails

1. Determine the correct location of the anchor plate **A** on the lowest part of the backpost.
2. Tighten screw with a 4mm Allen key.
3. Repeat these steps for the other side.
4. Determine the correct location of the anchor plate **B** on the seat rail.
Note : If installation requires the use of a belt attachment block, please refer to the following section.
5. Tighten screw with a 4mm Allen key.
6. Repeat these steps for the other side.

Round seat rails (with belt attachment block)


1. Determine the correct location of the anchor plate **B** on the seat rails.
2. Position both parts of the belt attachment block on the seat rail.
3. With a 5mm Allen key, tighten screw **C** firmly.
4. With a 4mm Allen key, secure the anchor plate to the attachment block by fastening screw.
5. Repeat these steps for the other side.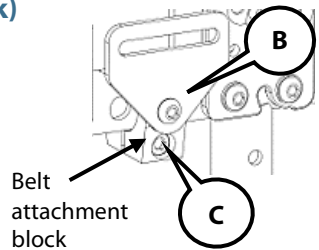

Note – Make sure anchor plates are placed in the corresponding adjustment holes of the opposite seat rail and backpost.

Adjusting the pelvic positioning belt

To adjust the length of the positioning belt slide belt straps **A** into the plastic slide buckles **B**, then tighten or loosen straps until the belt fits snugly without causing any discomfort. Center buckle **C** by tightening or loosening straps **D**.

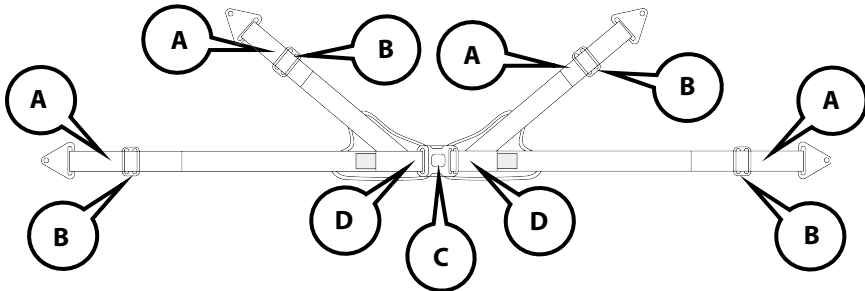WARNINGS

- When using your wheelchair, it is important to always fasten your positioning belt for added protection.
- It is important to adjust your positioning belt to maximize comfort and safety.
- Improper use can result in serious injury or death.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your positioning belt.

Cleaning the pelvic positioning belt

A weekly cleaning is recommended.

1. With a mix of water and a soft soap or neutral detergent, gently wipe the belt.
2. Remove excess moisture by lightly patting the belt with a clean cloth and then let air dry completely.

WARNINGS

- Do not submerge in water.
- Never machine wash or dry.
- Do not iron.

Storage

Postural supports must be stored in a dry and secure area.

WARNINGS

- Never store your postural support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your postural support near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your postural support is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH**WARRANTY**

The warranty period for the parts and components of the 4-point padded "Y Style" positioning belt is **12 months**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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