 **CONTOUR CINCH**TM
Dossier · Back

Owners manual

This manual contains important informations about this product. Please hand to the user upon delivery.

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Table of content / Table des matières

INTRODUCTION - ENGLISH	3
CONTOUR CINCH™ BACKREST	4
COMPOSITION	5
RECOMMENDATIONS.....	7
WARNINGS.....	8
INSTALLATION	9
ADJUSTMENT	12
MAINTENANCE	14
WARRANTY	16

INTRODUCTION – FRANÇAIS	18
DOSSIER CONTOUR CINCH ^{MC}	19
COMPOSITION	20
RECOMMANDATIONS.....	22
AVERTISSEMENTS	23
INSTALLATION	25
AJUSTEMENT	28
ENTRETIEN.....	30
GARANTIE.....	32

ENGLISH**INTRODUCTION - ENGLISH**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

CONTOUR CINCH™ BACKREST

The Contour Cinch™ Backrest is composed of an easily adjustable and removable aluminum shell and soft foam padding. With lateral supports built directly into the hard shell, the Contour Cinch™ Backrest curves around the side of the body and helps keep the user in an upright position for better postural control. The contoured rigid backrest improves lumbar and thoracic stability. Soft foam padding, made with urethane and viscose memory foam, is used for increased comfort, and completely covers the sides of the backrest to prevent skin lesions by eliminating all possible contact with the hard shell.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

ENGLISH

COMPOSITION

Technical specifications

Nominal Widths: 14" to 26" (36 cm to 66 cm)

Nominal Heights: 16" to 20" (41 cm to 51 cm)

Thickness (Padding included): 2" (5 cm)

Weight (Padding included): 7 lb (3,2 kg)

Angle adjustment (forward/backward): 20°

Lateral Support Width (Padding included): 2" (5 cm)

Contour Cinch™ Backrest

Lateral Support Height:

- Apex of 12.5" (32 cm) for 14" and 15" (35 cm and 38 cm) wide backrest
- Apex of 14" (36 cm) for 16" to 26" (41 cm to 66 cm) wide backrest

Hard Shell

Composition: Aluminium

Thickness :

- For 20" backrest and under: 0.094 (2,4 mm)
- For 21" backrest and over: 0.125 (3,17 mm)

Hardware

Mounting plate

Universal clamp

Foam Padding

Top layer: Viscose ½" (12 mm)

Bottom layer: Urethane 1" (24 mm)

Available Covers

Stretch

Waterproof

Backrest size

Each Contour Cinch™ back size can only be installed on a specific width of wheelchair. To correctly choose the size of your back, it is important to consider the widths of the wheelchair and of the user's thorax. See the dimensions tables below:

Table 1: Sizes associated with backrest

Actual Plate Width	Nominal Width	Nominal Height Available for Backrest (Padding included)	Thorax Width Contour Cinch™ Backrest	Associated Wheelchair Width
10" (25 cm)	14" (35 cm)	17" (43 cm)	10" (25 cm)	14" (35 cm)
11" (27 cm)	15" (38 cm)	18" (45 cm)	11" (27 cm)	15" (38 cm)
12" (30 cm)	16" (40 cm)	19" (48 cm)	12" (30 cm)	16" (40 cm)
13" (32 cm)	17" (43 cm)	20" (50 cm)	13" (32 cm)	17" (43 cm)
14" (35 cm)	18" (45 cm)	21" (53 cm)	14" (35 cm)	18" (45 cm)
15" (38 cm)	19" (48 cm)	22" (55 cm)	15" (38 cm)	19" (48 cm)
16" (40 cm)	20" (50 cm)	23" (58 cm)	16" (40 cm)	20" (50 cm)
17" (43 cm)	21" (53 cm)	24" (60 cm)	17" (43 cm)	21" (53 cm)
18" (45 cm)	22" (55 cm)	25" (63 cm)	18" (45 cm)	22" (55 cm)
19" (48 cm)	23" (58 cm)	26" (66 cm)	19" (48 cm)	23" (58 cm)
20" (50 cm)	24" (60 cm)	27" (68 cm)	20" (50 cm)	24" (60 cm)
21" (53 cm)	25" (63 cm)	28" (71 cm)	21" (53 cm)	25" (63 cm)
22" (55 cm)	26" (66 cm)	29" (74 cm)	22" (55 cm)	26" (66 cm)

ENGLISH**RECOMMENDATIONS**

To ensure your backrest provides optimal protection and comfort and to extend your product lifespan, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the backrest components, please refer to Maintenance guide in section Maintenance, page 14.
- Physipro Inc. backrest are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always install backrest with the provided mounting hardware.
- Never push or lift wheelchair with backrest, doing so may cause the backrest to detach from the wheelchair and can result in injury or damage.
- Before installation, make sure the backrest is compatible with the wheelchair's dimensions.
- Never place a towel, a blanket, a folded bed sheet or any other material between the user and the backrest, this will diminish the backrest efficiency and increase the risk of developing skin lesions.
- It is preferable to wear clothes made of cotton, that are not too tight and do not have seams on pressure points.
- Before use, let the backrest get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Remove any foreign object (crumbs, debris, tubing, etc.) from backrest surface.
- Clean on a regular basis and visually inspect all components of the cushion periodically.
- It is recommended to consult your therapist in the month following the installation of a new cushion to make sure it is efficient.

WARNINGS

Consult a healthcare professional: Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required seating system and make any necessary adjustments.

Center of gravity: Installing a backrest on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. To ensure optimal stability of your wheelchair after the addition of a new backrest, always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added or adjusted to increase stability.

Backrest dimensions: The dimensions of the backrest must be adapted to the user and the wheelchair, if not, the backrest's benefits will be reduced or eliminated.

Backrest and cover compatibility: The backrest cover must be well-fitted and used as directed in this manual to prevent the formation of skin lesions. An oversized or undersized cover will reduce or eliminate the effectiveness of the backrest and can reduce stability.

Environment: Avoid exposing foam to fluids or moisture for prolonged periods. Avoid exposing the backrest to extreme cold or heat. Never place backrest near open flame.

Instability/Fall risk: Never place other products or materials (except those specified in this manual) between the user and the backrest or underneath the backrest, doing so, will cause instability and increase the risk of injury from a fall.

Positioning of backrest: Always refer to the orientation indicators, located on the backrest and the cover, to properly position backrest on wheelchair. Proper positioning is essential to ensure maximum comfort and pressure relief.

Skin inspection: Backrest should be tested before prolonged use. To avoid the appearance of skin redness due to pressure points, Physipro Inc. recommends shifting your weight as frequently as possible and performing a pressure relief technique at least once every two (2) hours. You should regularly check (at least once a day) for skin redness especially around bony prominences. If skin redness or discoloration develops, discontinue use of the cushion. If discoloration does not disappear within thirty (30) minutes consult a Healthcare professional immediately. Be aware that no backrest can substitute the need for proper skin hygiene, a balanced diet and frequent position changes.

Transport: Crash tests have not been conducted on these products and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

ENGLISH

INSTALLATION

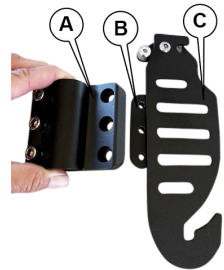
Please read all the following instructions before beginning installation. The steps required to properly install the Contour Cinch™ Backrest must be followed in the order that they are listed.

Required tool: 5mm Hex Key

ASSEMBLING THE MOUNTING PLATE AND UNIVERSAL CLAMP

The assembly of the mounting plates and the universal clamps will have been done by our technicians before delivery. However, if reassembly is necessary, please follow the instructions listed below:

1. Place section **B** of the universal clamp underneath mounting plate **C**.

2. Place section **A** of the universal clamp over mounting plate **C**.

3. Align mounting holes of sections **B** and **A** of the universal clamp with the central adjustment slots of mounting plate **C**.

4. Insert 3 M6 screws and tighten firmly with a 5 mm hex key. Repeat these steps for the second mounting plate.

INSTALLING UNIVERSAL CLAMPS ON BACK POST

Note – These steps must be completed before the wheelchair occupant is seated.

1. Loosen all 3 clamp screws **A** with a 5 mm hex key to open the universal clamp enough to fit around the back post.

2. Place the clamp on back post with the screw heads towards the outside of the wheelchair.

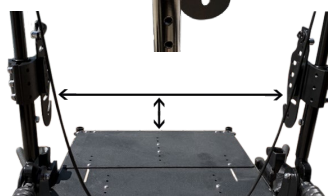
3. Position the universal clamp at the desired height and tighten the clamp screws **A**.

4. Repeat these steps for the second universal clamp.

5. Once installed on back post, verify that both clamps are vertically aligned.

ENGLISH**INSTALLING BACKREST ON MOUNTING PLATES**

The two-point mounting system of the Contour Cinch™ Backrest ensure a quick and easy installation. It takes only two simple movements to complete the installation of your Contour Cinch™ Backrest:

1. Simultaneously insert both inferior mounting pins of the backrest into the lower slots of the mounting plate.

2. Push latch on each mounting plate upwards and insert the upper mounting pins of the backrest into the upper slots of the mounting plates

**WARNING**

It is important to make sure that the latches are in a locked position and that the backrest is securely attached before use.

ADJUSTMENT

BACKREST HEIGHT ADJUSTMENT

1. Loosen the 3 clamp screws **A** on each universal clamp.
2. Lift or lower backrest to the desired height.
3. Securely tighten all clamp screws **A**.

Note – After height adjustment is completed, make sure the width was not inadvertently changed.

BACKREST DEPTH AND ANGLE ADJUSTMENT

1. Loosen screws **B** on each universal clamp.
2. Position backrest at the desired depth and inclination, by sliding backrest forward or backwards while simultaneously tilting the backrest at the desired angle.
3. Securely tighten all screws **B** of both universal clamps.

Note – The Contour Cinch™ Backrest offers a depth-adjustment range of $\frac{3}{4}$ ". The backrest can be positioned up to 1" behind the front of the back post, in its most backward position and $\frac{1}{4}$ " behind the front of the back post, in its most forward position. (The depth measurement is taken from the front of the backpost.)

ENGLISH**REMOVING BACKREST**

1. Simultaneously lift each latch **D** of mounting plates to unlock the backrest.
2. Gently push the backrest forward to slide mounting pins out of the upper slots of the mounting plates.
3. Slide the lower mounting pins out of the lower slots of the mounting plates to completely remove backrest.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and may help extend the life of your backrest. It is recommended to visually inspect the backrest during the cleaning process.

Maintenance Guide

Upon receiving your backrest, it is important to complete all the inspections listed in the Maintenance Guide to ensure the proper function and condition of the backrest and mounting hardware.

Checklist		Weekly	Montly
Backrest	- Inspect aluminum components for rust or damage. - Make sure the backrest is not deformed.		X
Hardware	- Verify the solidity and condition of mounting plates and universal clamps. - Ensure latch lock mechanism operates properly.	X	

Cleaning the outer cover:

1. Remove cover from backrest and fasten the zipper.
2. Machine wash in cold water on the gentle cycle or hand wash with a mild detergent.
3. Allow to air dry completely.
4. Reinstall cover on the foam base.

WARNING
Do not iron.

ENGLISH

Cleaning the hard shell: The aluminum hard shell can be cleaned with warm water and a mild liquid detergent. No other maintenance is required.

Cleaning the backrest padding:

1. Remove outer cover and incontinence cover, if necessary.
2. Wipe foam padding lightly with a damp cloth, without soap or detergent.
3. Use a clean dry cloth to gently press on padding to absorb excess moisture, then let air dry completely.
4. Reinstall cover(s) on the backrest.

**WARNINGS**

*Do not use soap.
Do not submerge in water.
Do not use washer or dryer to clean
the backrest padding.*

Storage

Backrest must be stored in a dry and secure area.

**WARNINGS**

*Never store backrest in direct sunlight or expose to excessive humidity or heat.
Never place backrest near sharp objects or in an area where it may be struck.*

Life span

The estimated life span of your backrest is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty periods for the following parts and components of your Contour Cinch backrest are as stated:

Backrest: Twenty-four (24) months

Outer cover: Six (6) months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period.

Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a. Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b. Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c. Normal wear and tear or damages caused by the user such as punctures, tears or burns.

ENGLISH**Warranty service**

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon as possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date and the product's serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.
370 10th Avenue South, Sherbrooke, Quebec J1G 2R7 Canada

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Inc.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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