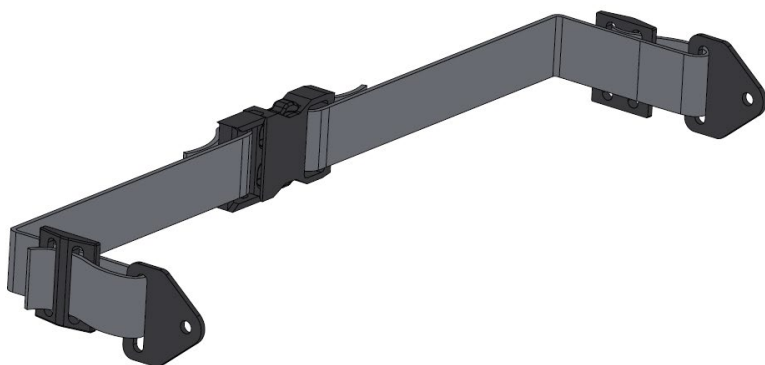
TRAY STRAP

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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ENGLISH**INTRODUCTION**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
RECOMMENDATIONS.....	3
WARNINGS	3
INSTALLATION	4
MAINTENANCE.....	4
WARRANTY.....	5
FRANÇAIS	6

DESCRIPTION

The tray strap prevents the wheelchair tray from being pushed forward and falling off. Equipped with two slide buckles for accurate length adjustment and a plastic buckle, this tray strap was designed for quick and easy installation.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

1-inch Nylon strap
Plastic buckle
Plastic slide buckles
Plastic anchor plates

RECOMMENDATIONS

To ensure the longevity of the tray strap, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the tray strap's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to install, and adjust the upper limb support.
- Clean regularly and inspect the tray strap's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the tray strap.

WARNINGS

- Physipro Inc. tray strap is reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Never push or lift wheelchair with the tray strap, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

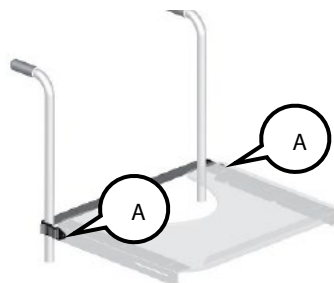
ENGLISH

INSTALLATION

Wheelchair tray slide rails are pre-drilled to facilitate the installation of the tray strap.

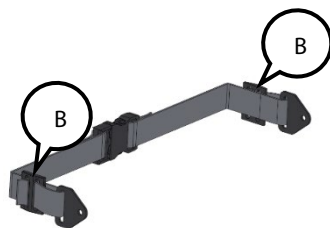
Installing the tray strap

1. If necessary, insert the brass inserts into the pre-drilled holes of the tray slide rails.
2. Align the plastic anchor plate with the brass inserts.
3. Place washer on screw **A** and then fasten screw **A** into the insert with a 3mm Allen key.
4. Repeat steps 2 and 3 for the opposite side.

Adjusting the tray strap

To adjust the length of the tray strap, slide the strap into the plastic slide buckles **B**.

WARNING

Do not tighten the strap excessively, this may deform the back post.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of the tray strap. A weekly cleaning is recommended.

Cleaning the tray strap

1. With a mix of water and a soft soap or neutral detergent, gently wipe the strap.
2. Remove excess moisture by lightly patting the strap with a clean cloth and then let air dry completely.

Storage

Tray strap must be stored in a dry and secure area. Never store tray strap in direct sunlight or near a heat source.

Life span

The estimated life span of your upper limb support is one (1) year.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty period for the parts and components of the tray strap is **90 days**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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