PHYSIPRO[®]

DOUBLE KNEE STABILIZER

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPOSITION.....	3
RECOMMENDATIONS.....	3
WARNINGS.....	4
INSTALLATION.....	4
MAINTENANCE.....	5
WARRANTY.....	6
FRANÇAIS.....	7

DESCRIPTION

The double knee stabilizer prevents the pelvis from sliding forward and stabilizes the knees. The velcro adjustments permit accurate positioning of the nylon straps and restrict forward movement of the thighs and pelvis to ensure a correct sitting position is maintained. The soft neoprene cushions with lycra trim gently surrounds the knees to improve comfort and support.

This system is attached directly under the rigid seat plate with brackets and is made according to the measurements provided.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

¼-inch Neoprene

Lycra

2-inch Nylon straps

2-inch Velcro fasteners

Glide buckles

Mounting hardware

Brackets

Screws and inserts

RECOMMENDATIONS

To ensure the longevity of your lower limb support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the lower limb support's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required lower limb support and make any necessary adjustments.
- Clean regularly and inspect the lower limb support's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the lower limb support's surface.

It is recommended to consult your doctor or therapist in the month following the installation of a lower limb support to make sure it is efficient.

ENGLISH

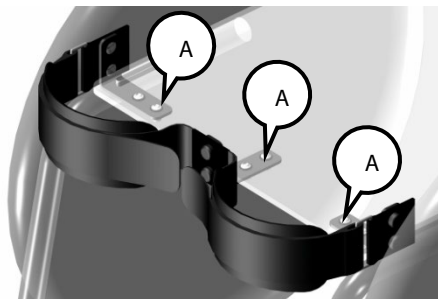
WARNINGS

- Physipro Inc. lower limb supports are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your lower limb support.
- Never push or lift wheelchair with a lower limb support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- New lower limb supports should be tested before prolonged use. To avoid redness due to pressure points, it is necessary to regularly check for redness in the various critical pressure areas, especially around bony prominences.
- Before use, let the lower limb support get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your lower limb support to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION

Required tool: 3mm Allen key

1. Determine the location of the brackets and drill six ¼" holes in the rigid seat plate.
2. Insert the six inserts into the holes.
3. Position the brackets under the seat plate and secure the six screws **A** with a 3mm Allen key.

MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your lower limb support. It is recommended to visually inspect all the components of the lower limb support during the cleaning process.

A weekly cleaning is recommended.

Cleaning the double knee stabilizer

1. With a mix of water and a soft soap or neutral detergent, gently wipe the stabilizer.
2. Remove excess moisture by lightly patting the stabilizer with a clean cloth.
3. Let air dry completely.

Cleaning the hardware

The hardware can be cleaned with warm water and a mild liquid cleaner.

Storage

Lower limb supports must be stored in a dry and secure area.

WARNINGS

- Never store your lower limb support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your lower limb support near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your lower limb support is two (2) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH

WARRANTY

The warranty period for the parts and components of the double knee stabilizer is 90 days.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at order@physipro.com, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. DO NOT RETURN product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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