SWING-AWAY HALF TRAY

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner’s manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION..... 3

COMPOSITION..... 3

RECOMMENDATIONS..... 3

WARNINGS 3

INSTALLATION 4

MAINTENANCE..... 5

WARRANTY..... 6

FRANÇAIS 7

DESCRIPTION

The swing-away half tray is particularly convenient for hemiplegics by providing adequate support for the paralyzed limb. The swing-away bracket allows the table to be flipped away when not in use and is mounted directly on the armrest.

This product has been assessed by the manufacturer and deemed to meet EU safety, health and environmental protection requirements and has obtained the CE marking.

COMPOSITION

Transparent polycarbonate (Lexan)

RECOMMENDATIONS

To ensure the longevity of your upper limb support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the upper limb support's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required upper limb support and make any necessary adjustments.
- Clean regularly and inspect the upper limb support's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the upper limb support's surface.

WARNINGS


- Physipro Inc. upper limb supports are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your upper limb support.
- Never push or lift wheelchair with an upper limb support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- Before use, let the upper limb support get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your upper limb support to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

ENGLISH


INSTALLATION

Installing the half tray

1. Loosen screws **A** to free the blocking bars **B** and **C**.

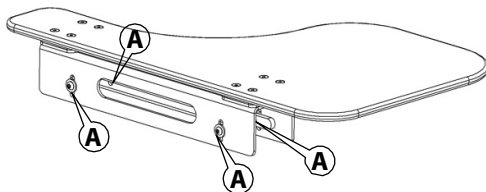
2. Pull blocking bars **B** and **C** downwards, this will facilitate the installation of the bracket on the armrest.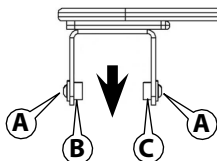


Note : Blocking bar **B** is thicker than **C** and must be placed on the opposite side of the table, as illustrated.

3. Slide bracket on the armrest.

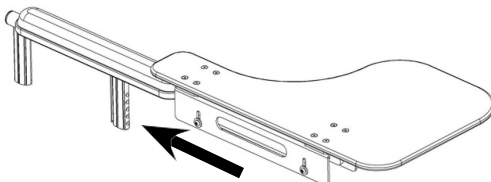
4. Raise blocking bars **B** and **C** until they are firmly pressed against the armrest.
5. Firmly tighten screws **A**.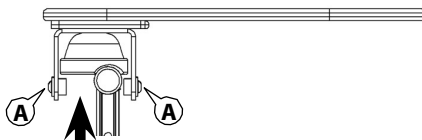

Flipping the half tray

1. Lift the half tray and flip it toward the outside.

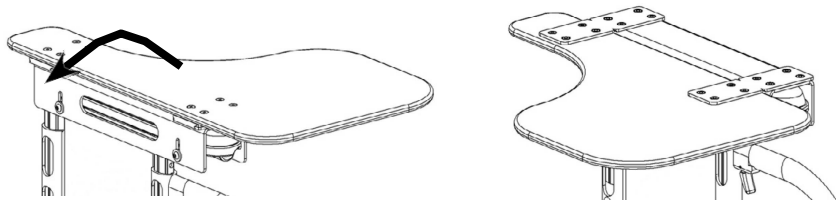MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your upper limb support. It is recommended to visually inspect all the components of the upper limb support during the cleaning process. A weekly cleaning is recommended.

Cleaning the tray

1. Clean the tray with a cloth slightly dampened with warm water and a mild detergent.
2. Wipe with a clean dry cloth.

WARNING

- Do not submerge tray in water.

Storage

Upper limb supports must be stored in a dry and secure area.

WARNINGS

- Never store your upper limb support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your upper limb support near sharp objects or in an area where it may be struck by an object.

Life span

The estimated life span of your upper limb support is three (3) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

WARRANTY

The warranty periods for the parts and components of the Swing-away Half Tray are as stated:

Tray (Lexan): 36 months
Standard components: 12 months
Optional components: 12 months

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc. The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.
 370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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