Owners manual

Wedge – Seat Cushions

This manual contains important informations about this product.
Please pass it to the final user upon delivery.

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ENGLISH

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products.

This user manual has been designed to allow the client to use the wedges, a Physipro product, safely in an optimal way. For all adjustments and settings, Physipro strongly recommends you consult a qualified professional.

We provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your wedges and extend its operational life.


Your satisfaction remains a priority.

Anti-hammock effect base wedge

POINTED


FLAT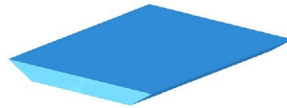


Glued under the seat cushion, the anti-sling wedge is useful for controlling hammock effect. By creating a flat surface, it helps re-establish pelvic balance. The anti-sling wedge can be laminated under the cushion or simply inserted between the cushion and the wheelchair fabric. Two different shapes are available.

Possibility of installation

It is possible to order the base wedge with a cover in order to use it on a canvas seat or a hard seat. The base wedge can then be fixed by Velcro to the cushion or be fully removed, if required.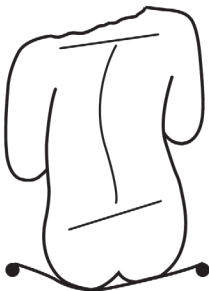

Installation without the
Anti-Sling Wedge

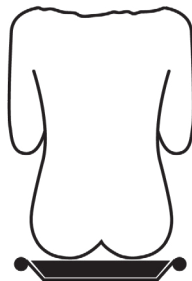
Installation with an
Anti-Sling Wedge

Installation with a
Pointed Anti-Sling
Wedge

Wedge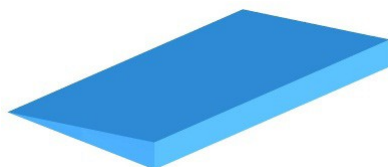


The wedge reduces sliding, control pelvic rotation and posterior pelvic tilt. Available in two different densities: soft urethane (Neocor) and constructor foam.

The constructor foam wedge is glued or fixed by Velcro depending on the type of seat cushion. If the wedge is made of a low density foam (urethane) the anti-thrust wedge will be placed on top of the cushion.



It is usually used with seat well cushions that procure a blocking effect. The wedge increases the pressure in front of the cushion and thus provides a femoral alignment.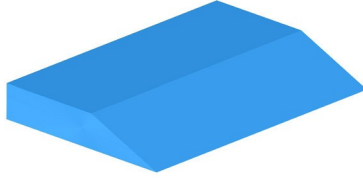

ENGLISH**Anti-thrust Wedge**

This component is installed on the front part of the seat cushion. The Anti-thrust Wedge reduces sliding, controls certain patterns of spasticity, and helps distribute pressure over the entire seat.

Made of constructor foam, this component is glued or fixed by Velcro under dispersion foam and comfort foam.

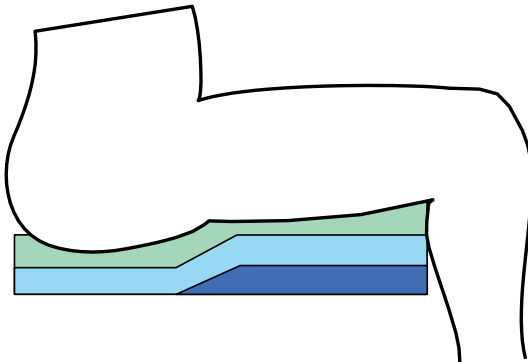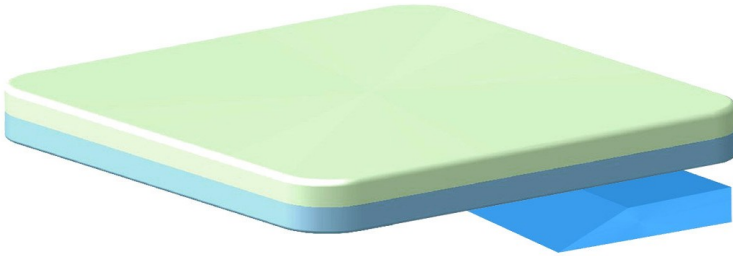
Medial Thigh Abductor

CONSTRUCTOR FOAM


NEOCOR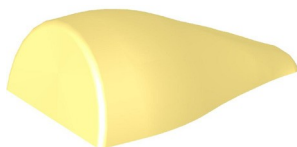


This medial thigh abductor is used for controlling lower-limb adduction. The urethane (Neocor) soft-foam Medial Thigh Abductor can be fix by velcro on top of the cushion. The constructor foam Medial Thigh Abductor is laminated to the cushion.



In some cases of posterior pelvic tilt, the Medial Thigh Abductor will maintain a neutral pelvic alignment if the posture is not fix.

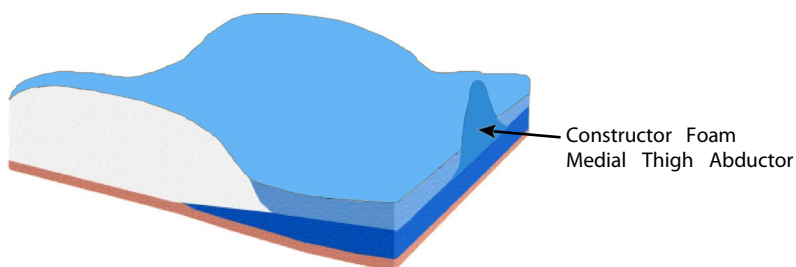

Constructor Foam
Medial Thigh Abductor


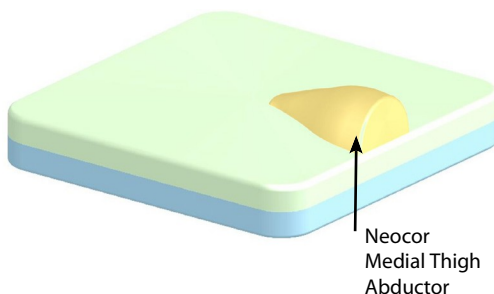
Neocor
Medial Thigh
Abductor

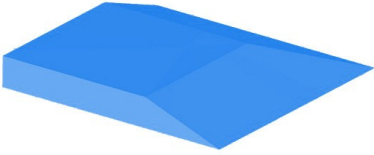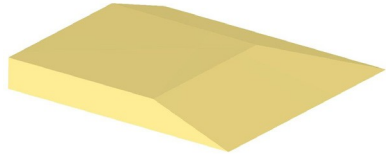ENGLISH**Pelvic Obliquity Wedge****CONSTRUCTOR FOAM****URETHANE**

The Pelvic Obliquity Wedge can compensate for loss of gluteal muscle tone or reduced pelvic tilt. The Urethane wedge uses Velcro to attach to the top of the cushion. The constructor foam wedge is glued under de dispersion foam and the constructor foam.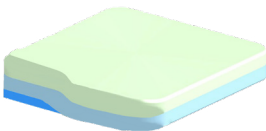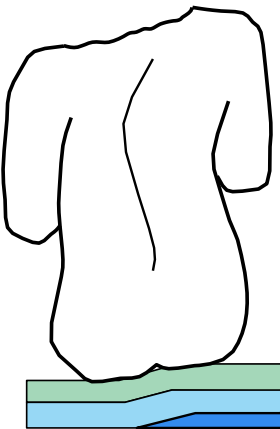

Wedge under the cushion
(Constructor foam)

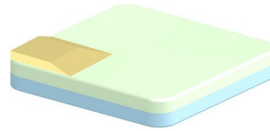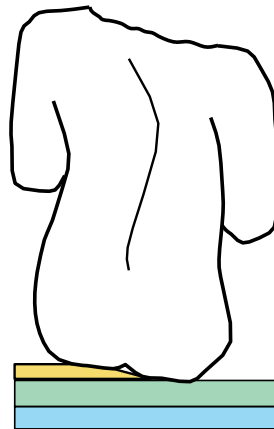
Wedge on top of the cushion
(Urethane)

Lateral Pelvic Support


This shape component is used for controlling abduction. It enables the users to maintain proper femoral alignment. The Lateral Pelvic Support is made of firm foam (Ethafoam) and is glued under the layers of dispersion foam of the seat cushion.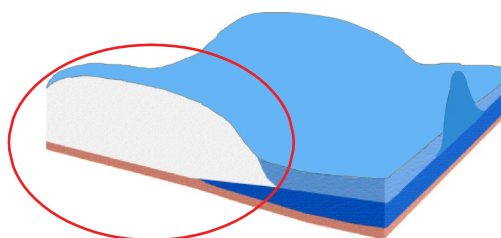


Lateral Adductor Support

Controls lower-limb abduction. This Ethafoam wedge is glued to the seat cushion.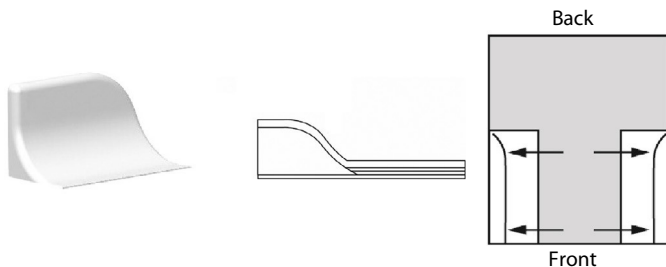



Lateral Pelvic Cushion

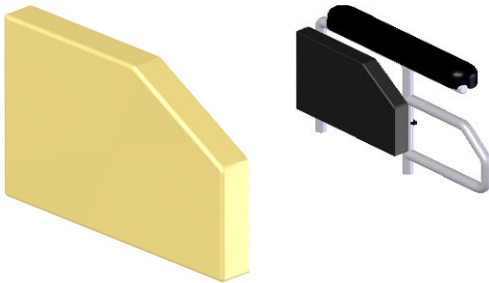The lateral pelvic support is covered with a Meshtex cover. The Velcro banners allow to easily attach the lateral pelvic support to the clothing guard.

- ✓ Fill the gap caused by pelvic rotation
- ✓ Allows the pelvis to be positioned directly in the cushion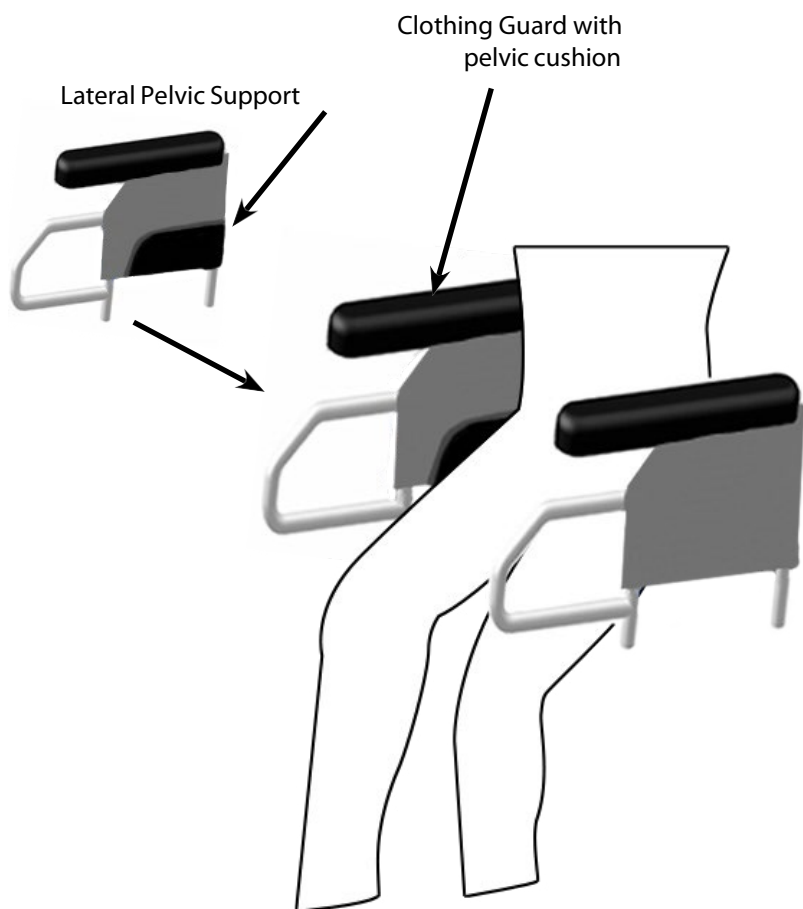

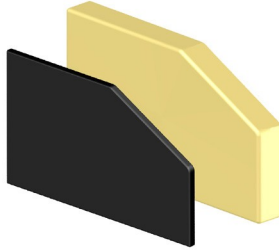ENGLISH**Lateral Pelvic Cushion**


These pelvic supports can be installed on the slotted mounting brackets or installed using a sliding bar in the backrest slots : (the system with the sliding bar allow more adjustment). This support is covered with a Meshtex cover.

2 type of mounting systems for rigid pelvic support


Adjustable brackets for rigid pelvic support


With this system, the pelvic supports can be attached and adjusted in width and height.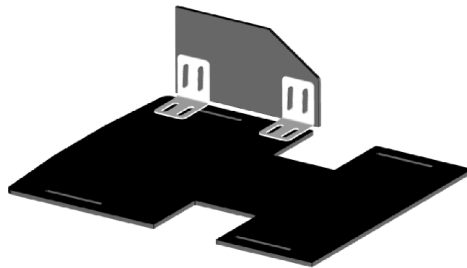

Adjustable bar for rigid pelvic support

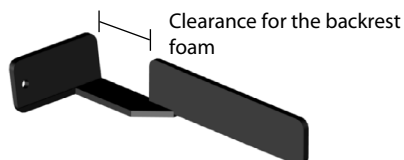Installation principle

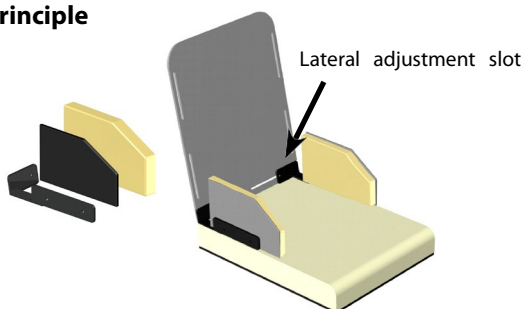MAINTENANCE & CLEANING

To clean the foam:

- 1- Wipe lightly with a damp cloth;
- 2- Wipe off with a clean cloth;
- 3- Let it completely air-dry before reassembly.

Do not use soap

Do not submerge in water

DISPOSAL OF MEDICAL DEVICES

Do not burn and do not throw away the wedges outside of designated areas.

WARRANTY

Those products are guaranteed to be free from defects in materials and workmanship for a period of 12 months from the date of purchase. This warranty covers original defects in material and workmanship of the wedges, optional components and new parts that have been replaced by the supplier or by an authorized retailer, with effect from the delivery date or, if applicable, from the date of replacement. Physipro inc. agrees to repair or replace any inoperative or defective part for the duration of the warranty period. To enjoy this warranty service, please contact Physipro inc. or an authorized retailer. Do not return this product without prior approval. If you are dissatisfied with this product, send us your comments at the address provided therein, and include the name and address of the supplier and the delivery date of the product. Furthermore, this warranty shall not apply to products that have been damaged through repairs or modifications made without the prior written consent of Physipro inc. or of an authorized retailer. The same restrictions shall apply for damages resulting from other circumstances beyond the control of Physipro inc.

Physipro relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

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