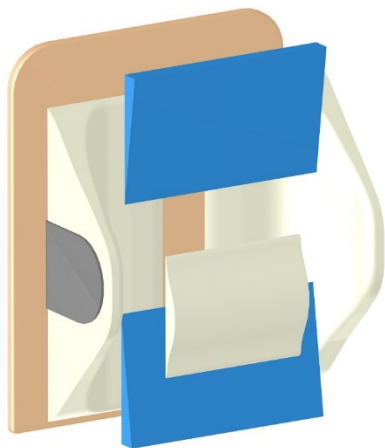
BACKREST WEDGES

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION - ENGLISH

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

For Physipro Inc., your satisfaction is our highest priority.

PRODUCT INFORMATION

Date of purchase: _____

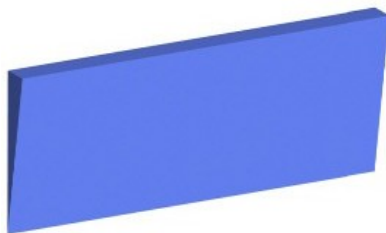
Supplier: _____

Address: _____

Telephone: _____


The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

ENGLISH**THORACIC WEDGE**

This wedge can be installed in different places on the backrest to improve posture. When installed at the top of the backrest, the thoracic wedge helps to stabilize and support the occupant's back by filling in the gap created by a kyphosis condition.

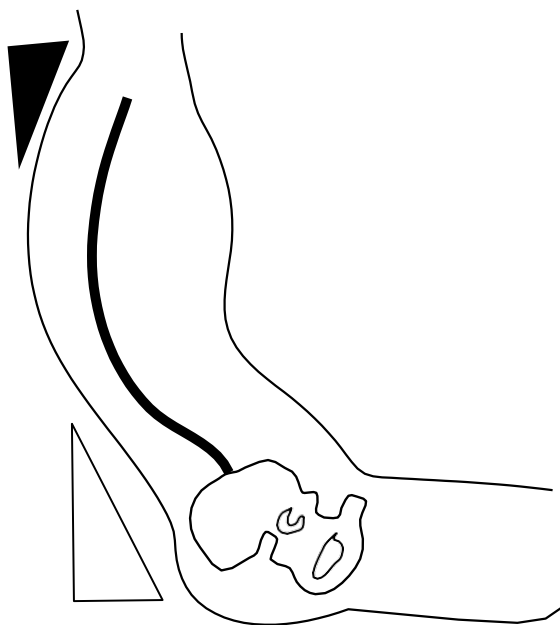Available in urethane or Constructor foam in thicknesses of 1", 1 ½" or 2". The urethane wedge is fixed with velcro, while the Constructor foam wedge will be glued to the backrest foam.


LUMBAR WEDGE

This wedge can be installed in different places on the backrest to improve posture. Installed across the lower section of the backrest, the lumbar wedge creates a greater support surface in the lumbar region.



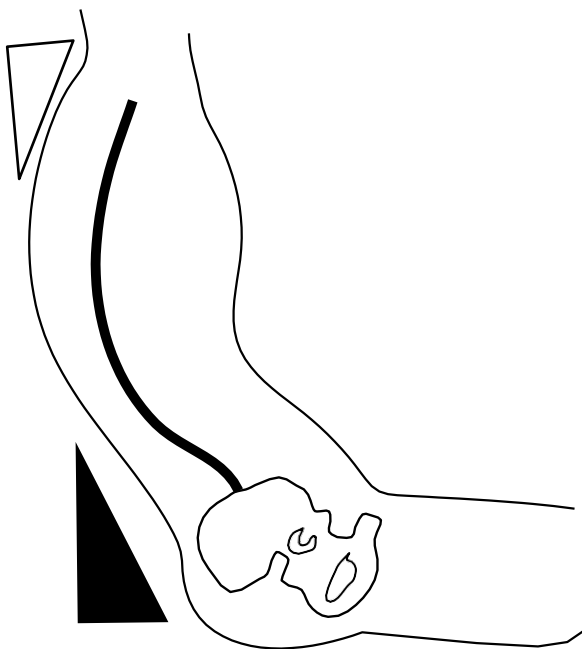
Available in urethane or Constructor foam in thicknesses of 1", 1 ½" or 2". The urethane wedge is fixed with velcro, while the Constructor foam wedge will be glued to the backrest foam.


ENGLISH**THORACOLUMBAR WEDGE**

Installed on one side of the backrest, the thoracolumbar wedge compensates for or helps correct poor posture caused by trunk rotation. Used on both sides of the backrest, the thoracolumbar wedge improves overall back posture and comfort by increasing the back's support surface.

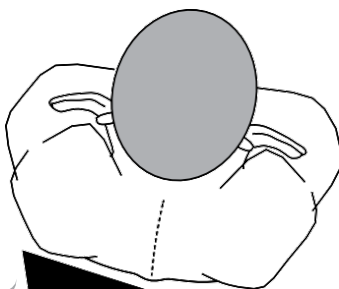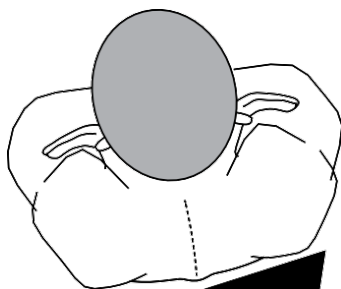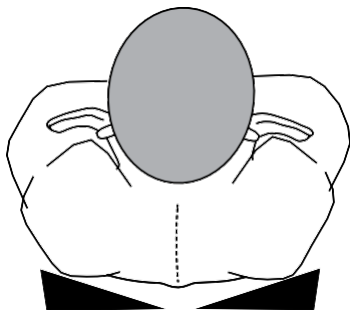
This wedge can be installed on a rigid or sling backrest. Available in urethane or Constructor foam in thicknesses of 1", 1 ½" or 2". The urethane wedge is fixed with velcro, while the Constructor foam wedge will be glued to the backrest foam.


SACROLUMBAR CORRECTOR

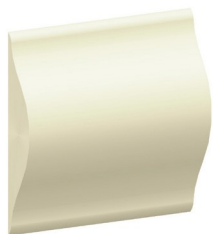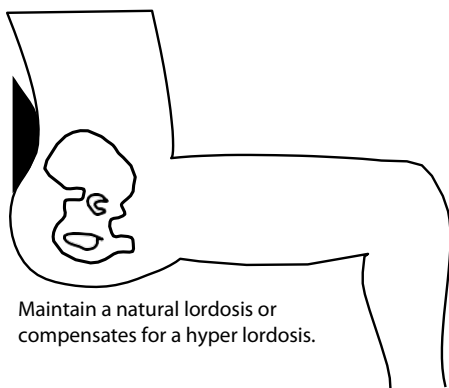The sacrolumbar wedge is used to correct a pelvic tilt. When placed in the lower section of the backrest, this wedge increases back support and helps reduce the effect of a lordosis condition, it can also be used to promote an anterior tilt of the pelvis. The urethane wedge is fixed with velcro, while the Contractor foam wedge will be glued to the backrest foam.

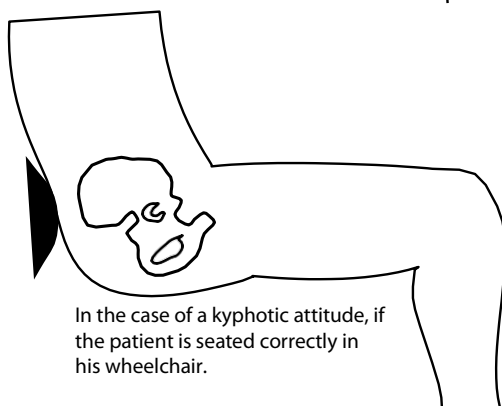Maintain a natural lordosis or compensates for a hyper lordosis.

In the case of a kyphotic attitude, if the patient is seated correctly in his wheelchair.

WARRANTY

Backrest wedges are covered by a warranty on parts and labor for a period of twelve (12) months against any manufacturing defect on optional components and new parts replaced by the supplier or by an authorized establishment, and this, from the date of delivery or, if applicable, the date of replacement. Physipro Inc. is committed to repair or replace defective parts during the entire warranty period. To benefit from the warranty service, contact Physipro Inc. or an authorized establishment. In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

Physipro Inc.
370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions et limitations

This warranty does not apply to products that have been accidentally or intentionally altered, or to products that have been subject to improper use, negligence, poor maintenance, or improper storage. In addition, this warranty does not apply to products damaged as a result of repair or modification without the written consent of Physipro Inc. or an authorized facility. The same restrictions apply to damage resulting from any other circumstance beyond the control of Physipro Inc. This warranty does not apply to normal wear and tear of parts or failure to comply with the indications mentioned in this document.

Physipro Inc. is not responsible for any damage that may occur during transport.

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Physipro Inc. se dégage de toute responsabilité liée aux dommages pouvant être subis au cours du transport.