## **PROTECTOR PAD**

### **Owner's Manual**

This booklet contains important information about this product.  
Please give to the owner upon delivery.

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**ENGLISH****INTRODUCTION**

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

**PRODUCT INFORMATION**

**Date of purchase:** \_\_\_\_\_

**Supplier:** \_\_\_\_\_

**Address:** \_\_\_\_\_

**Telephone:** \_\_\_\_\_

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

**TABLE OF CONTENT**

|                               |          |
|-------------------------------|----------|
| DESCRIPTION.....              | 3        |
| COMPOSITION.....              | 3        |
| TECHNICAL SPECIFICATIONS..... | 3        |
| RECOMMENDATIONS.....          | 3        |
| WARNINGS .....                | 4        |
| INSTALLATION .....            | 4        |
| MAINTENANCE.....              | 5        |
| WARRANTY.....                 | 6        |
| <b>FRANÇAIS .....</b>         | <b>7</b> |

DESCRIPTION

The protector pad can be added to a positioning belt over the buckle to provide greater comfort or can be positioned on either the left or right side to improve postural alignment.

This item meets the requirements of California 116 & 117, NFPA 701, CE marking and NF EN1021-1, EN1021-2 norms.

COMPOSITION

- ¼-inch Neoprene
- Lycra
- 1-inch Nylon straps
- Snap button

TECHNICAL SPECIFICATIONS

| Product No. | Size | Height    |                  | Width     |                 |
|-------------|------|-----------|------------------|-----------|-----------------|
|             |      | Inches    | Centimeters      | Inches    | Centimeters     |
| SC7550      | #1   | 1½" to 3" | 3.7 cm to 7.5 cm | 3" to 6"  | 7.5 cm to 15 cm |
| SC7555      | #2   | 1½" to 4" | 3.7 cm to 10 cm  | 6" to 16" | 15 cm to 40 cm  |

RECOMMENDATIONS

To ensure the longevity of the protector pad, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the protector pad.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to install, and adjust the protector pad.
- Clean regularly and inspect the protector pad periodically.
- Remove any foreign object (crumbs, debris, etc.) from the protector pad.

It is recommended to consult your doctor or therapist in the month following the installation of a new postural support device to make sure it is efficient.

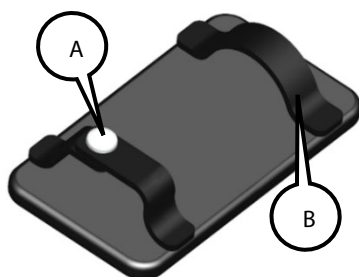
## ENGLISH

### WARNINGS

- Physipro Inc. protector pads are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Never push or lift wheelchair with the protector pad, doing so may cause the pad to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- The protector pad should be tested before prolonged use. To avoid redness due to pressure points, it is necessary to regularly check for redness in the various critical pressure areas, especially around bony prominences.
- Before use, let the protector pad get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing the protector pad to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

### INSTALLATION

1. Unfasten the snap button **A**.
2. Pull belt through the fixed loop **B**.
3. Position the protector pad as desired.
4. Fasten snap button **A**.

## MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of the protector pad.

A weekly cleaning is recommended.

### Cleaning the protector pad

1. Remove the protector pad from the belt.
2. With a mix of water and a soft soap or neutral detergent, gently wipe the neoprene padding and then rinse by wiping with water only.
3. Remove excess moisture by lightly patting the neoprene padding with a clean cloth and then let air dry completely.

### Storage

Postural supports must be stored in a dry and secure area.

#### WARNINGS

- Never store your postural support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your postural support near sharp objects or in an area where it may be struck by an object.

### Life span

The estimated life span of your protector pad is one (1) year.

### Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

## ENGLISH

### WARRANTY

The warranty period for the parts and components of the protector pad is **90 days**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

#### **This warranty does not apply to:**

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

#### **Warranty service**

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

#### **Physipro Inc.**

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

#### **Exclusions and limitations**

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

**The foregoing warranty is exclusive and in lieu of all other expressed warranties.**

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