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Multiadjustable headrest support

Owners manual

This manual contains important information about this product. Please pass it to the final user upon delivery.

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ENGLISH

Physipro inc. is delighted to have you among its clients. We sincerely wish to thank you for the trust you have shown in our Company by choosing one of our products. This user manual has been designed to allow the client to use the headrest support, a Physipro inc. product, safely in an optimal way. For all adjustments and settings, Physipro inc. strongly recommends you consult a qualified professional.

Physipro inc. provides you with specific verifications you should perform on a regular basis in order to optimize the performance of your headrest support and extend its operational life. Physipro inc. relieves itself from any liability should any bodily injury or property damage result from a lack of care or misuse or from any modification made to the product without prior written consent of the Company.

For Physipro inc., your satisfaction remains a priority.

Table of Contents

1. Recommendations..... 2
2. Components..... 3
3. Attach mounting bracket..... 3
4. Installation on tension bar..... 4
5. Headrest support adjustment 4
6. Pivot point anchorage depth adjustment..... 4
7. Multidirectional Adjustment 5
8. Warranty..... 5

1. Recommendations

Do not use this equipment without having read and understood this user manual in its entirety. This manual includes critical information intended to ensure the safety of the user and the individuals who handle the wheelchair.

1.1 Basic verification

At the time of delivery and receipt check all parts for shipping damage before using.

1.2 Safety rules

Several safety measures must be applied to ensure the safety of the user. Note that the following list is not comprehensive. Therefore, the people who use the headrest support have the responsibility to use it cautiously:

- ✓ Do not use the headrest support to push or lift wheelchair.
- ✓ Installation and adjustment of your headrest support should be performed only by an authorized dealer or qualified therapist.
- ✓ Make sure that all screws and adjustments are tightened properly, otherwise serious injury damage may result.

2. Components

1. Anchor band (AT5210)
2. Mounting Bracket (PH-02066B)
3. Fixing collar (PH-02066C)
4. Clamping handle
5. Vertical bar
6. Horizontal bar
7. End piece protection
8. Clamping handles
9. Support screw MS (20mm)
10. Left side support
11. Right side support

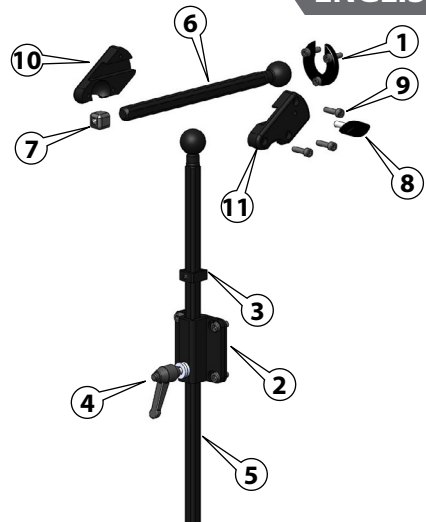
Figure 1

3. Attach mounting bracket

1. Position the mounting bracket towards the 4 holes on the top of the plate.
2. Attach the mounting bracket to the backrest with the four-button head screws and nuts.

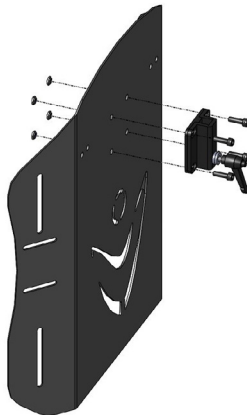
Figure 2

ENGLISH

4. Installation on tension bar

- 1 Place adaptor **A** inside the tension bar and align it with mounting bracket **B**.
- 2 Install the mounting bracket with screws as shown.

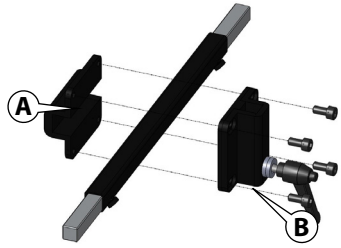
Figure 3

5. Headrest support adjustment

- 1 Align the vertical bar in the mounting bracket.
- 2 Adjust the headrest support in the desired position and tighten the clamping handle **A**.
- 3 To adjust the height, slide fixing collar **B** on mounting bracket and tighten.

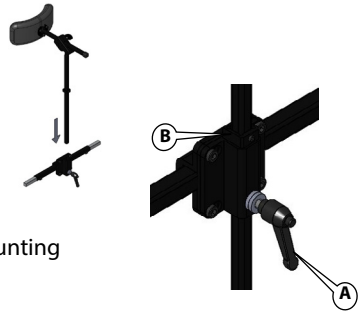
Figure 4

6. Headrest pad depth adjustment

- 1 Loosen clamping handle **A** from the horizontal bar.
- 2 Adjust the headrest pad in the desired position and tighten the clamping handle **A**.

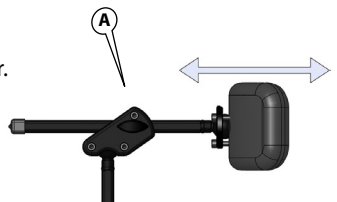
Figure 5

7. Multidirectional Adjustment

- 1 Loosen the M6 screws to free the ball joint for a customized adjustment.
- 2 Secure after adjustments.

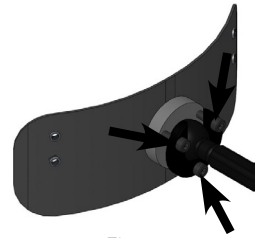
Figure 6

8. Warranty

The headrest support, a product of Physipro inc., is covered with our twelve (12) months manufacturer's warranty. It includes parts and labor on manufacturer's defects and optional parts, and all new parts replaced by your dealer or an authorized establishment from the date of delivery or replacement.

Physipro inc. will repair or replace the defective parts as long as the warranty is valid. You will have to contact Physipro inc. or an authorized dealer to obtain your warranty services. Do not return your product without prior consent. If you are not satisfied with the repairs, please send us a written complaint with your dealer's name, address, delivery date and the product's serial number to the address on this manual cover.

Maintenance

All headrest support have to be inspected at least once every three months to ensure their tightness and to detect any damage that could affect their use.

Exclusions and limitations

This warranty does not apply to products that were intentionally or accidentally altered, misused, neglected, did not have the proper maintenance or had the serial number removed or erased. Furthermore, this warranty does not apply if repairs or modifications were done without a written consent from Physipro inc., or an authorized dealer. The same restrictions apply if any other kind of damages were done without Physipro inc. involvement. This warranty will not apply for everyday use or failure to respect the rules in this manual.

Physipro inc. will not take any responsibilities for damages incurred during shipping.

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