PHYSIPRO[®]

MULTI-ADJUSTABLE HEADREST SUPPORT

Owner's Manual

This booklet contains important information about this product.
Please give to the owner upon delivery.

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INTRODUCTION

Physipro Inc. is proud to count you among its customers and would like to thank you for the confidence you have shown by purchasing our product.

This owner's manual was created to provide you with all the information needed to allow you to use our product in a safe and optimal way. For all adjustments and settings, we strongly recommend you contact your distributor.

Physipro Inc., relieves itself of all liability should physical injury or property damage occur due to the lack of maintenance or misuse of our product, or from modifications made to the product without the prior written consent of Physipro Inc.

PRODUCT INFORMATION

Date of purchase: _____

Supplier: _____

Address: _____

Telephone: _____

The product presented in this manual may not be an exact representation of your product. However, the instructions provided in this manual are still valid.

Physipro Inc. reserves the right to modify or change this manual without notice.

TABLE OF CONTENT

DESCRIPTION.....	3
COMPONENTS.....	3
RECOMMENDATIONS.....	4
WARNINGS.....	4
INSTALLATION.....	5
ADJUSTMENTS.....	6
MAINTENANCE.....	7
WARRANTY.....	8
FRANÇAIS.....	9

DESCRIPTION


Physipro's multi-adjustable headrest supports offers height and angle adjustments. Made with aluminium, the multi-adjustable headrest support features a T" locking handle for quick and easy depth adjustment. These headrest supports can be installed on a rigid backrest or on a tension bar.

This product has been assessed by the manufacturer and deemed to meet EU safety, health and environmental protection requirements and has obtained the CE marking.

COMPONENTS

Multi-adjustable - Straight

Multi-adjustable - Offset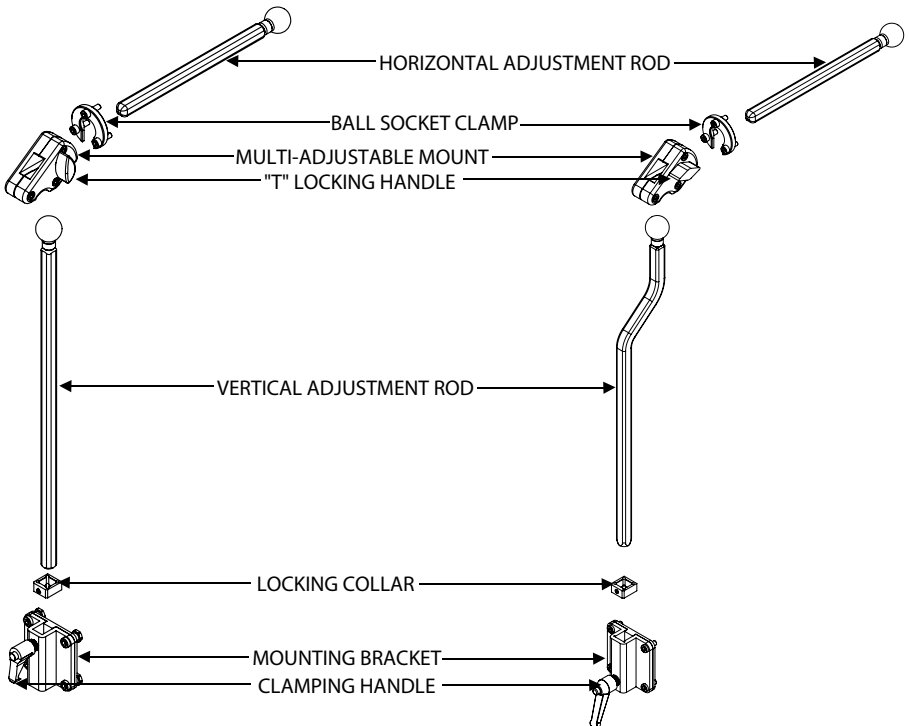

RECOMMENDATIONS

To ensure the longevity of your headrest support, the following precautions should be applied.

- Before use, it is important to verify the quality and condition of the headrest support's components.
- Physipro Inc. recommends that a health care professional such as a doctor or therapist, experienced in seating and positioning, be consulted to determine, and install the required headrest support system and make any necessary adjustments.
- Clean regularly and inspect the headrest support's components periodically.
- Remove any foreign object (crumbs, debris, etc.) from the headrest support's surface.

It is recommended to consult your doctor or therapist in the month following the installation of a new headrest support system to make sure it is efficient.

WARNINGS

- Physipro Inc. headrest supports are reserved exclusively for use on mobility aids (manual or power wheelchairs).
- Always use the provided mounting hardware to install your headrest support.
- Never push or lift wheelchair with the headrest or its support, doing so may cause the device to detach from the wheelchair which may result in damage or injury. Always lift wheelchair with the rigid parts of the frame.
- Installing a new product on a wheelchair may affect the wheelchair's center of gravity and may cause the wheelchair to become unstable. To ensure optimal stability of your wheelchair after the addition of a new component, always assess for the potential need for anti-tips or amputee axle adapter brackets to be added or adjusted.
- Before use, let the headrest get to room temperature to limit discomfort caused by excessively hot or cold surfaces.
- Avoid exposing your headrest to extreme cold or heat and avoid prolonged contact to fluids or moisture. Never place near open flame.
- Crash tests have not been conducted on this product and even if the mobility aid in which it is installed has obtained the ISO-717619, it will not be maintained.

INSTALLATION


Installing the mounting bracket on a rigid backrest

Required tool: 4 mm Allen key

1. Place the mounting bracket **A** on the backrest **B** and align the 4 holes of the mounting bracket with the 4 center holes of the backrest.
2. Attach the mounting bracket to the backrest with the screws **C** and nuts **D**.

Note: Physipro's rigid backrests are pre-drilled for headrest support installation.


If installing headrest support on a backrest that is not manufactured by Physipro, you may need to drill headrest support holes in your backrest. We recommend placing the mounting bracket to about 4" (10 cm) from the top of the backrest and then drilling the backrest by using the mounting bracket as a guide.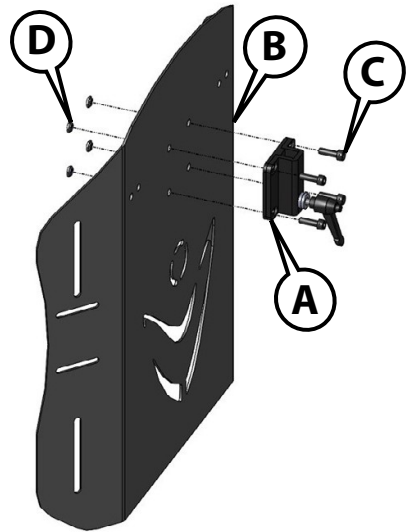

Installing the mounting bracket on a tension bar

Required tool: 4 mm Allen key

1. Place the adaptor **A** on the tension bar.
2. Align the 4 holes of the adaptor **A** with the 4 holes of the mounting bracket **B**.
3. Attach the mounting bracket to the adaptor with the screws **C**.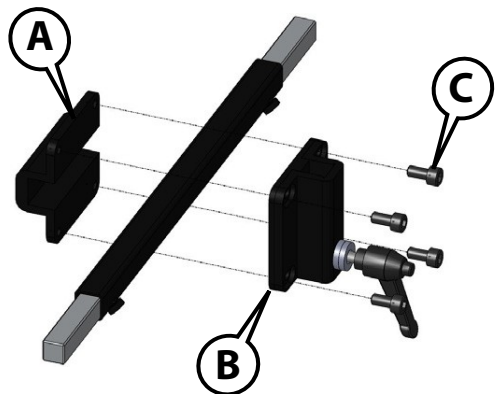


ENGLISH

ADJUSTMENTS

Height adjustment


Required tool: 3 mm Allen key

1. Loosen clamping handle **A** and insert the vertical rod **B** in the mounting bracket **C**.
2. Adjust the height of the headrest support and tighten the clamping handle **A**.
3. Slide locking collar **D** against the mounting bracket **C** and firmly tighten screw, this will ensure the headrest is positioned at the correct height if it is removed and then reinstalled.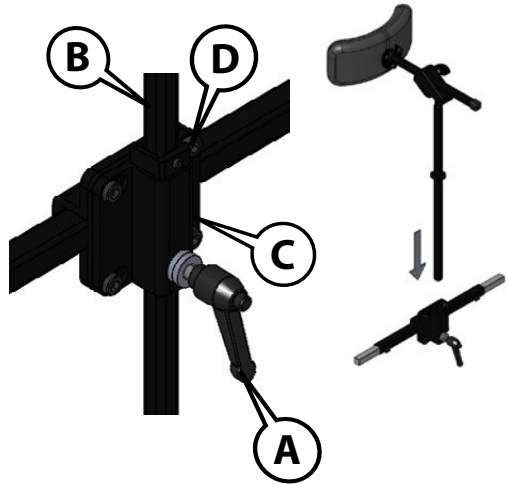

Depth adjustment


1. Loosen locking handle **A**.
2. Adjust the depth of the headrest and tighten the locking handle **A**.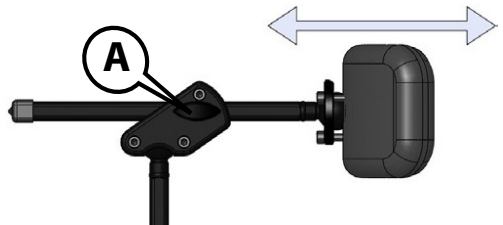

Multi-directional adjustment

Required tool: 4 mm Allen key

1. Loosen screws **A** of the ball socket clamp **B**.
2. Adjust headrest position and firmly tighten all three (3) screws **A**.

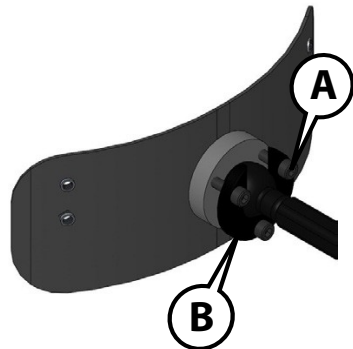MAINTENANCE

Regular cleaning eliminates the accumulation of dirt and grime and helps extend the life of your headrest support. It is recommended to visually inspect all the components of the headrest support during the cleaning process.

A weekly cleaning is recommended.

Cleaning the headrest support

1. Clean the headrest support with a cloth slightly dampened with warm water and a mild detergent.
2. Wipe with a clean dry cloth.

WARNING

- Do not submerge in water.

Storage

Headrest and its support must be stored in a dry and secure area.

WARNINGS

- Never store your headrest and/or its support in direct sunlight, near a heat source or in a high-humidity environment.
- Never place your headrest and/or its support near sharp objects or in an area where they may be struck by an object.

Life span

The estimated life span of your headrest support is five (5) years.

Disposal of Medical Devices

Do not burn or throw away medical devices outside of designated areas.

ENGLISH**WARRANTY**

The warranty period for the parts and components of Physipro Inc. Multi-adjustable Headrest Support is **12 months**.

This limited warranty covers defects in material and workmanship on all the standard or optional parts and components of this product, from the date of shipment, for the duration stated above.

Parts and components that are replaced or repaired under one of these warranties will remain subject to the warranty for the remainder of the warranty period. Physipro Inc. agrees to repair or replace, free of charge, any defective part or component for the full duration of the warranty period.

This warranty does not apply to:

- a) Products that were misused, neglected or have not been maintained and stored in accordance with the recommendations of the manufacturer.
- b) Products that were intentionally or accidentally altered, or products that have been damaged through repairs or modifications made without the prior written consent of Physipro Inc. or of an authorized retailer.
- c) Normal wear and tear or damages caused by the user such as punctures, tears, or burns.

Warranty service

If a defect is detected during the warranty period, please contact the retailer from whom you purchased your Physipro Inc. product as soon possible. Please refrain from using your product until an inspection has been made by a qualified technician.

In the event you do not receive satisfactory warranty service, please write directly to Physipro Inc. at the address listed below or send us an e-mail at **order@physipro.com**, please include the retailer's name and address, a brief description of the defect, the invoice date, and the products serial number, if applicable. **DO NOT RETURN** product prior to receiving a written return authorization from Physipro Inc.

The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date at the following address:

Physipro Inc.

370, 10th Avenue South, Sherbrooke, Quebec, J1G 2R7

Exclusions and limitations

Compliance with the safety precautions and recommendations mentioned in this manual is crucial to avoid injury to the user or other persons. Inadequate or inappropriate adjustments will increase the risk of accident. It is important to thoroughly read the manual before making any adjustments. Physipro Inc. will not be held responsible for bodily injury or property damage caused by misuse or incorrect adjustments made to its product by an unqualified person.

Physipro Inc. disclaims all liability for damages that occur during transportation or are the result of situations beyond its control.

The foregoing warranty is exclusive and in lieu of all other expressed warranties.

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