

## CUSHION GUARANTEE

We offer a 2 year warranty on Stimulite Classic/XS cushions, Contour/XS, Slimline®/XS, Silver, Sport, Pediatric Corbee® and Pediatric Contour.

This warranty includes any damage due to handling, a material defect or an error caused by the manufacturer. This warranty does not include damage resulting from daily use or tears caused by improper maintenance.

The cover of your cushion is guaranteed 90 days. This covers any damage due to handling, a material defect or an error caused by the manufacturer. Are excluded from the warranty: cases of burns by cigarettes, damage by sharp objects can cause tears or color changes caused by non-compliance with washing instructions.

## ELIMINATION OF MEDICAL DEVICES

Do not burn or throw into the nature the cushion and its protective cover outside the dedicated places.

## LIFETIME

The estimated lifespan of Stimulite cushions is 4 years.

## STORAGE

Stimulite cushions should preferably be stored on a flat surface, away from direct light and moisture.

Exclusive distributor in Canada and France



### CANADA

370, 10<sup>th</sup> Avenue South  
Sherbrooke (Quebec) J1G 2R7  
Phone: 819 823-2252/1 800 668-2252  
Fax: 819 565-3337  
info@physipro.com  
www.physipro.com

### FRANCE

Village des entrepreneurs  
461, rue St-Léonard  
49000 Angers  
Tél.: 02.41.69.38.01  
Télé.: 02.41.69.43.32  
contact@physipro.fr

Fabricant: Supracor Inc. 2050 Corporate Court San Jose, CA 95131-1753 États-Unis  
www.supracor.com

# Cushioning Products by Supracor

featuring Stimulite® honeycomb

## User manual



This booklet contains important information about this product.  
Please give it to the end user upon delivery.




**CONGRATULATIONS** on the purchase of your revolutionary Stimulite honeycomb technology!

Stimulite cushions offer full pressure management (Total Pressure Management®) - reduced pressure, reduced shearing and ventilation to control heat and moisture - the key solution for pressure ulcer prevention.

They are made of a thermoplastic urethane compound that is naturally antibacterial, antifungal and odor resistant. To help keep your skin clean and healthy, they are hand or machine washable.

### WARNING!

Before changing the seat system, it is recommended to consult your occupational therapist or health professional to test and have it installed correctly. A cushion can affect the center of gravity of your chair and cause a loss of stability. Some minor adjustments will therefore be required on your mobility aid.

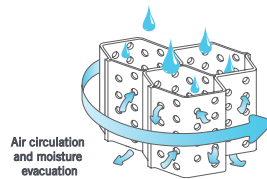

Complete aeration system

### ADVICE

In order to avoid the appearance of pressure ulcers, it is recommended to change position every three hours approximately. It is also necessary to regularly check for redness on your skin, especially around bony protuberances. The Stimulite honeycomb technology cushions are specially designed to relieve pressure. However, they can not be substituted for good skin hygiene, a proper diet and a frequent change of position.

Contact your doctor or occupational therapist immediately if redness persist.

### WEIGHT OF THE USER

Depending on the cushion measurements, the weight of the user can vary between 75 and 250 pounds - 34 to 90 kg. (The bariatric cushion supports a weight up to 650 pounds - 295 kg)

### SAFETY WARNING

Before using a seat system for a long time, it is recommended to try the cushion for a few hours and check that redness does not develop.

### WASHING INSTRUCTION

Stimulite cushions can be washed by hand or machine in cold water during delicate cycle.

- Use only a small amount of mild detergent (1 tablespoon).
- **NEVER USE BLEACHING.**
- After cleaning, shake to remove excess water.
- Dry the Stimulite sheet in the open air or in the dryer by using the tumble dryer at the minimum temperature.
- **DO NOT** dry the Stimulite sheet with wet clothing as this will cause heat retention and may damage the Stimulite sheet.

Each Stimulite cushion includes a laundry bag for machine washing and drying.

*\*Use only a front-loading washing machine. A traditional machine could damage the honeycombs.*

### PRECAUTIONS

- A cushion must be installed on a clean, flat surface.
- Check the dimensional compatibility of the cushion and seat as well as the height adjustment of the armrests during installation.
- For a better effectiveness of the support, it is important to limit as much as possible the extra thicknesses between the body and the cushion.  
vêtement de corps et d'un éventuel change complet.
- Loose cotton clothing and, where possible, without seams on support areas are preferable.
- Do not put a towel, folded sheet or other surface between the user and the cushion.
- Ensure the absence of any foreign matter such as tubing, crumbs, greasy substances...
- It is recommended to consult your occupational therapist within one month of installing this cushion to ensure its effectiveness.